# Hip Navigation

## User Guide

**VELYS™ Hip Navigation 4.0**

[www.jointpoint.com](http://www.jointpoint.com)

IFU-0902-00-905 Rev. B

© DePuy Synthes 2021. All rights reserved.  
VELYS™ Hip Navigation User Guide (Version 4.0)

## Table of Contents

Intended Use .....	5
Indications for Use.....	5
Hardware Specification .....	6
VELYS™ Hip Navigation Windows .....	6
VELYS™ Hip Navigation for iOS .....	6
System Requirements.....	7
Hardware .....	7
Software .....	7
Network.....	7
Support and Contact Information .....	7
Logging in for the First Time.....	8
Password Recovery.....	8
Change Password .....	9
Logout .....	9
Main Display.....	10
Header Bar .....	10
Navigation Bar .....	10
Footer Bar.....	10
Calendar Use .....	11
New Case Creation .....	12
Case Type Selection.....	12
Enter Case Details.....	12
Workflows .....	13
Adding an X-Ray.....	14
Camera .....	14
Direct Import.....	14
Case X-Rays.....	14
DICOM/Image File .....	14
X-Ray Options .....	16
Invert Image .....	16
View.....	16
Delete .....	16
Templates & Analysis.....	17
Preoperative Hip Workflow .....	18

Creating a Preoperative Hip Workflow .....	19
1. Select PreOp Hip Template .....	19
2. Import X-Ray .....	19
3. Scale the X-Ray. ....	20
4. Create the Preoperative Hip Template .....	21
5. Final Preoperative Template .....	29
OneTrial™ Workflow .....	30
Creating a OneTrial™ Workflow .....	30
1. Select OneTrial™ .....	30
2. Import Non-Operative X-Ray .....	30
3. Greater Trochanter .....	31
4. Draw Circle On Femoral Head .....	31
5. Align Femoral Axis Line Tool Down Canal .....	32
6. Mark Pelvic Reference Line .....	32
7. Mark Teardrop .....	33
8. Cup Position Analysis .....	33
9. Enter Implant Sizes .....	34
10. Import Operative X-Ray .....	37
11. Mark Greater Trochanter .....	38
12. Enter Component Size and Place Circle Around Acetabulum .....	38
13. Mark Shoulder of Trial/Implant .....	39
14. Mark Pelvic Reference Line .....	39
15. Mark Teardrop .....	40
16. Rotate Template to Match Cup .....	40
17. Align Femoral Template in Canal .....	41
18. Align Femurs .....	41
19. Surgeon Check Point .....	42
20. OneTrial™ Analysis View .....	42
21. Analyze New Image .....	45
Contralateral Overlay Workflow .....	46
Creating a Contralateral Overlay Workflow .....	46
1. Select Contralateral Overlay .....	46
2. Import Non-Operative X-Ray .....	46
3. Mark Greater Trochanter .....	47
4. Draw Circle around Femoral Head .....	47
5. Mark Teardrop Brim Line .....	48

6. Import Operative X-Ray .....	48
7. Mark Greater Trochanter.....	49
8. Draw Circle On Acetabular Component .....	49
9. Align Femoral Axis Tool in Canal.....	50
10. Mark Teardrop Brim Line.....	50
11. Overlay Analysis .....	51
Cup Check.....	54
Creating a Cup Check Workflow .....	54
1. Select Cup Check .....	54
2. Import AP Pelvis X-Ray .....	54
3. Mark the Trans-teardrop or Neutral Axis .....	55
4. Mark Operative Hip Teardrop Brim Line.....	55
5. Import Operative X-Ray .....	56
6. Mark Teardrop Brim Line.....	56
7. Cup Check Analysis.....	57
Preoperative Knee Workflow.....	58
Creating a Preoperative Knee Workflow .....	58
1. Select the PreOp Knee Template.....	58
2. Import AP X-Ray .....	58
3. Scale the AP X-Ray.....	59
4. Add Optional Lateral X-Ray.....	60
5. Import Lateral X-Ray.....	60
6. Scale the Lateral X-Ray .....	61
7. Create the Preoperative Knee Template.....	62
8. Final Preoperative Template.....	65
Appendix A - Symbol Glossary.....	66

The names “JointPoint” and “Velys Hip Navigation” may be used interchangeably to describe the software solution throughout the documentation.

## Intended Use

VELYS Hip Navigation is a non-invasive software system intended to provide preoperative templating for orthopaedic procedures and intraoperative data for total hip arthroplasties (THA).

VELYS Hip Navigation is designed to provide the following:

- Accurate digital templating by allowing 'real time' templating in the OR using direct measurement and template rescaling.
- Intraoperative surgical guidance, including offset, leg length, anteversion and abduction angle data. These data will be determined using a completely noninvasive imaging approach, and do not rely on hardware associated with computer assisted navigation.

## Indications for Use

VELYS™ Hip Navigation is an image-processing software indicated to assist in the positioning of total hip replacement components. It is intended to assist in precisely positioning total hip replacement components intra-operatively by measuring their positions relative to the bone structures of interest provided that the points of interest can be identified from radiology images. VELYS™ Hip Navigation is also indicated for assisting healthcare professionals in preoperative planning and postoperative analysis of orthopedic surgery in Total Hip Replacement, Total Knee Replacement, and Intertrochanteric Fracture Reduction. The device allows for overlaying of prosthesis templates on radiological images and includes tools for performing measurements on the image and for positioning the template. Clinical judgment and experience are required to properly use the software. The software is not for primary image interpretation. The software is not for use on mobile phones.

## Hardware Specification

VELYS™ Hip Navigation markets two versions of its software platform – VELYS™ Hip Navigation for Windows and VELYS™ Hip Navigation for iOS. The VELYS™ Hip Navigation for iOS system is an iPad-based system, and the VELYS™ Hip Navigation for Windows system operates in a Windows environment.

The VELYS™ Hip Navigation for Windows system is provided on the hardware with specifications provided below. In addition, the preoperative planning module in the VELYS™ Hip Navigation for Windows software may be used on any Windows machine that meets the minimum hardware and software requirements.

### VELYS™ Hip Navigation Windows

**The VELYS™ Hip Navigation Windows product specifications include the following:**

- 24" Medical Grade All-in-One PC with Battery
- Variable Height Roll Stand for Flat Panel Display with mounting bracket
- Wheel Base for pole roll stand attachment with 4 wheels
- Dual (Side by Side) Power Supply Mount with Cord Cleat for 2" Posts
- Medical Grade Power Strip with 4 Hospital-Grade Outlets
- "L" Bracket with Keyboard Plate
- Roll Stand Handle
- Medical Grade Silicone Keyboard
- PC mounting bracket adapter
- Manual Tilt Clamp (Included in Wheel base box)
- Direct Import Setup
  - BNC to VGA + HDMI Converter
  - HD USB video grabber with DVI to VGA cable and USB 3.0 cable
  - BNC cable – Male to Male extension connector
- Pre-loaded VELYS™ Hip Navigation Software License for Intraoperative and Preoperative use, including acetabular component analysis module, contralateral and ipsilateral analysis modules for leg length and offset, OneTrial™ technology, and preoperative templating.
- Preoperative planning and analysis module (enterprise license) is included with license.

### VELYS™ Hip Navigation for iOS

The VELYS™ Hip Navigation for iOS system can be installed to run on an iPad Pro that meets the minimum hardware requirements.

## System Requirements

Intraoperative use of VELYS™ Hip Navigation for Windows requires use of the hardware specification above. Intraoperative use of VELYS™ Hip Navigation for iOS requires that the system be installed on an iPad Pro 12.9" running a minimum of iOS 14.

The following configuration is required for preoperative use of VELYS™ Hip Navigation:

### Hardware

- 2GB RAM or more
- Processor: i5/i7 (compatible) 2.8 GHz or higher
- Hard Drive:
  - Up to 80MB for the software
  - Up to 200GB for data storage
- Minimal screen resolution: 1280X720
- 2GB graphics card

### Software

- Windows 10 with all current patches/updates
- Microsoft .NET Framework 4.7.2
- TeamViewer for remote support

### Network

- Unrestricted internet access to \*.jointpoint.com on port 443
- TLS 1.2 or higher

## Support and Contact Information

If you require assistance with use of VELYS Hip Navigation, you can reach our Customer Support via phone or email:

Support Phone: +1 (800) 689-0746

Support Email: VELYSHipNav\_CustomerSupport@ITS.JNJ.com

## Logging in for the First Time

Once your account is setup and the application is installed, you will need to login to begin using VELYS™ Hip Navigation. When you login for the first time, be sure the Offline/Online switch is set to Online.

Offline Mode will allow you to use VELYS™ Hip Navigation for a period of 5 days without an internet connection. After the 5 days has expired, you will be required to login again with Online Mode.

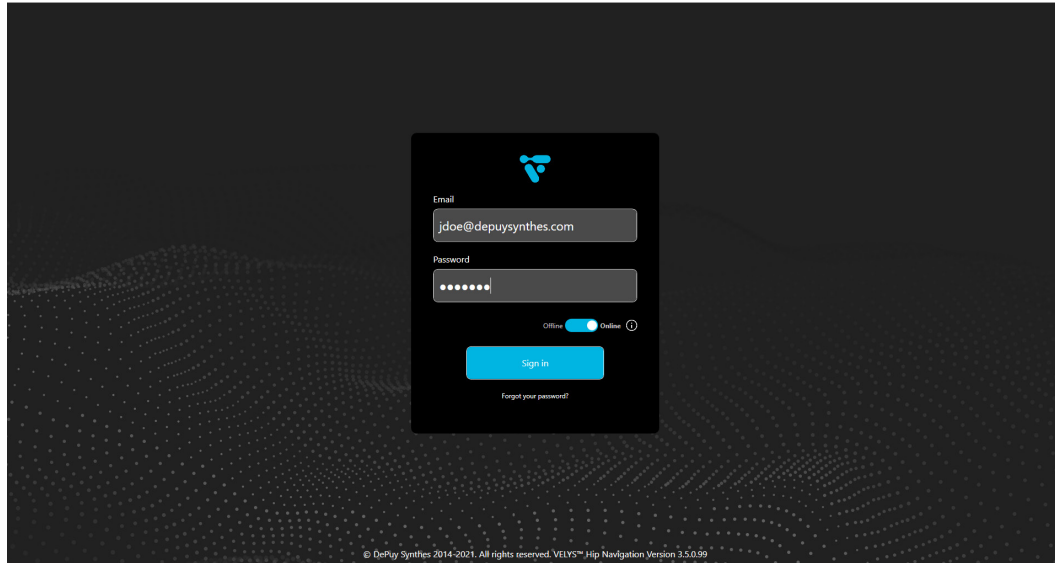
Figure 1: Login Screen

## Password Recovery

If you forget your password, you may reset it by selecting the “Forgot your password?” option. The following dialog is presented. Selecting “Send Reset Link” will send an email to your account with a link that allows you to reset your password.

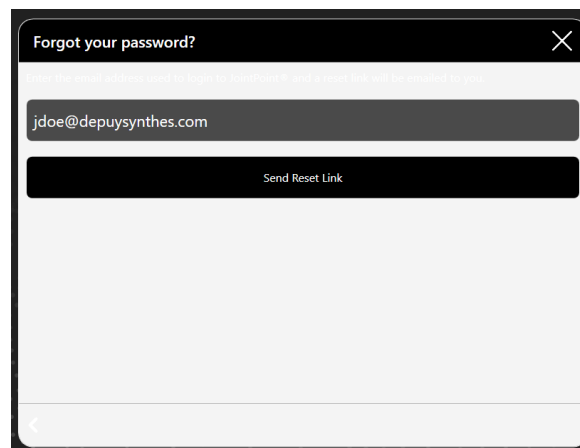
Figure 2: Password Recovery Screen

## Change Password

If you wish to change your password, once you login, select the gear icon at the bottom of the screen and select “Change Password”.

Figure 3: Settings Icon

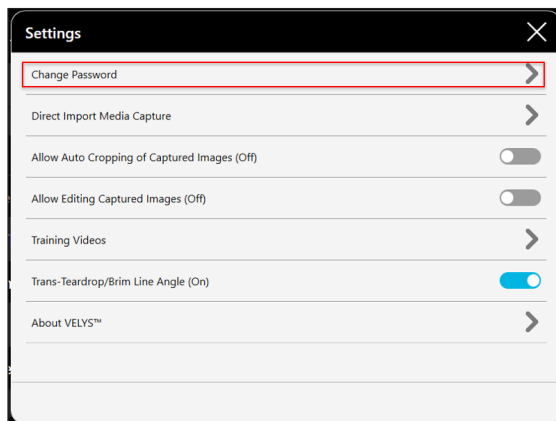
Figure 4: Settings Menu

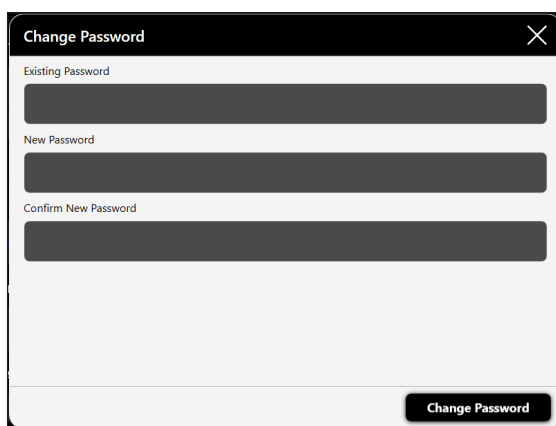
Figure 5: Change Password Menu

## Logout

To logout of VELYS™ Hip Navigation click on Account at the top right of screen and select Logout as depicted below.

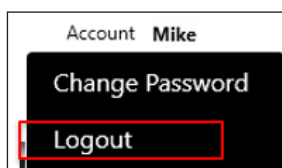
Figure 6: Account Drop-Down

## Main Display

### Header Bar

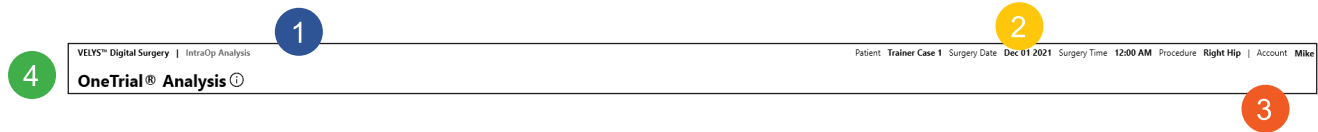
Figure 7: Header Bar

1. Current workflow (1) Patient and Procedure Details (2)
2. Currently logged in user (3)
3. Current Step Title (4)

### Navigation Bar

Figure 8: Navigation Bar

1. Previous – Moves the workflow back a step
2. Next/Finished – Either moves the workflow forward or finishes the workflow and exits
3. Step progress and menu – Allowing navigation to other steps
4. Current Step in workflow

### Footer Bar

Figure 9: Footer Bar

Icons from left to right:

- Previous – Moves back a screen
- Remote Support Settings
- Home/Schedule
- Start workflow or next step in workflow

## Calendar Use

After logging in, you will be presented with a schedule view. The schedule presents cases from the current day forward. To view past cases, use the Search feature at the top.

The components of the calendar view are as follows:

- Create New Case button allows the creation of new cases.
- Search button used for patient name or to search by date (Figure 11).
- Cases are grouped by day and include surgery date, patient name and surgeon. Each row also displays the procedure type and body side. Grouped rows can be expanded and collapsed.
- Select button takes you into the case to view information, X-rays, and workflows.
- Cases can be downloaded to a local device using download icon on bottom middle of screen.
- Cases can be deleted by selecting and using the trash can icon.

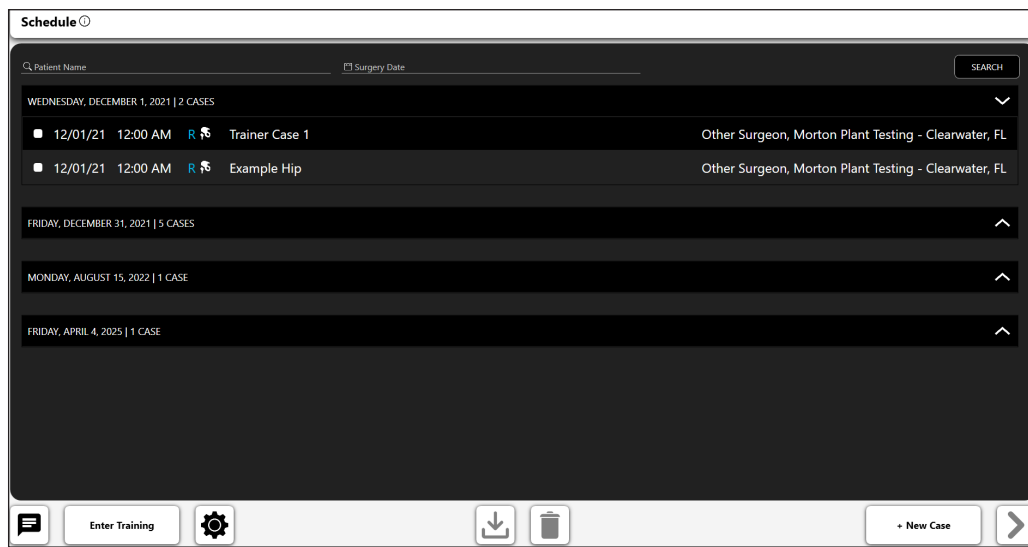
Figure 10: Schedule Screen

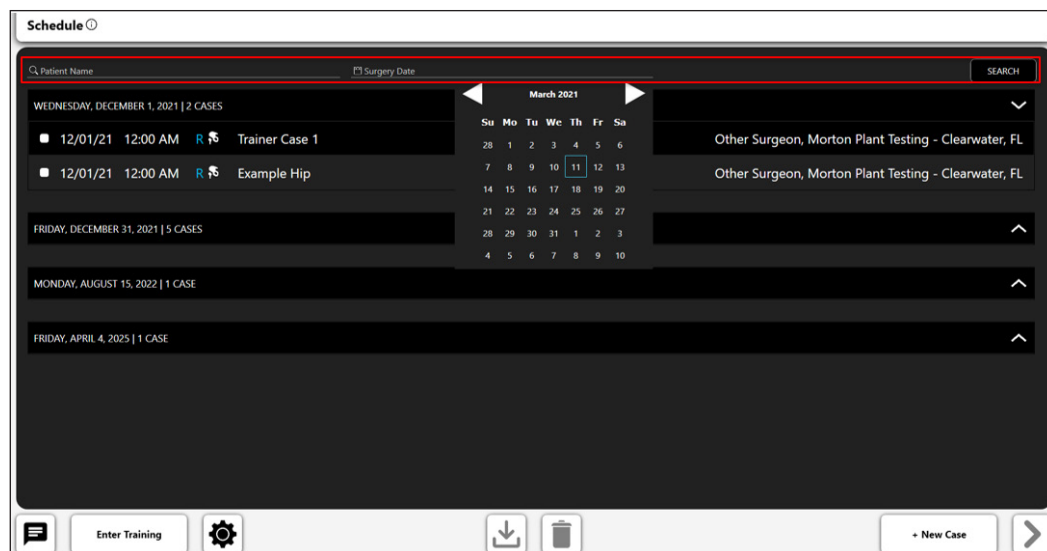
Figure 11: Schedule Screen - Search Detail

## New Case Creation

### Case Type Selection

To create a new case, select the New Case button. A dialog will be displayed to allow you to choose the type of case to create.

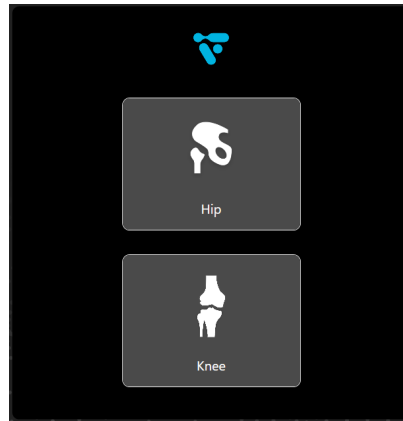
Figure 12: Case Type Selection Detail

### Enter Case Details

Once an option is selected you will be prompted to fill in some basic information. Patient first and last name, surgery date, operative side, gender, and surgeon are required. You may share the case with any users currently configured on your account for that purpose. Notes may be added to provide additional information on the case.

A screenshot of the "Enter Case Details" screen. The screen has a dark background with white text and icons. At the top, it says "Enter Case Details" with a close icon. Below this are several input fields: "First Name \*", "Hospital" (with a dropdown arrow), "Notes" (with a plus icon), "Last Name \*", "Surgeon \*" (with a dropdown arrow), and "Share Case" (with a plus icon). There are also date pickers for "DOB" (MM, DD, YYYY), "Surgery Date \*" (MM, DD, YYYY), and "Surgery Time" (HH, MM, AM/PM). At the bottom, there are buttons for "Operative Side \*" (Right, Left) and "Gender \*" (Male, Female, Not Specified). A navigation bar at the very bottom contains icons for back, chat, settings, home, and forward.

Figure 13: Enter Case Details Screen

## Workflows

Selecting the “Next” arrow along the bottom will advance you to the Case Overview screen.

Figure 14: Next Arrow Button

The Case Overview screen allows you to create, edit or remove workflows on the case, view a case summary, and add or remove X-rays to the case image library.

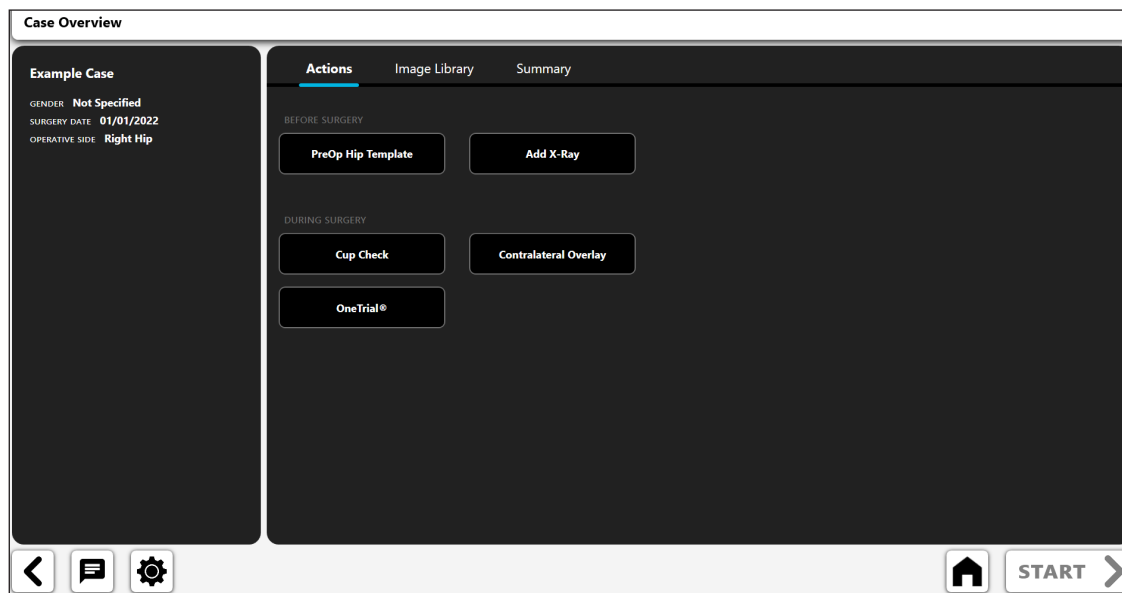
Figure 15: Case Overview Screen

1. Actions – add x-ray or create a case analysis
2. Image Library – saved x-rays
3. Summary – analysis that was previously created and saved

To get started select an Action and then the “START” button. The following dialog is presented. The options allow you to add X-rays or create one of the VELYS™ Hip Navigation Workflows.

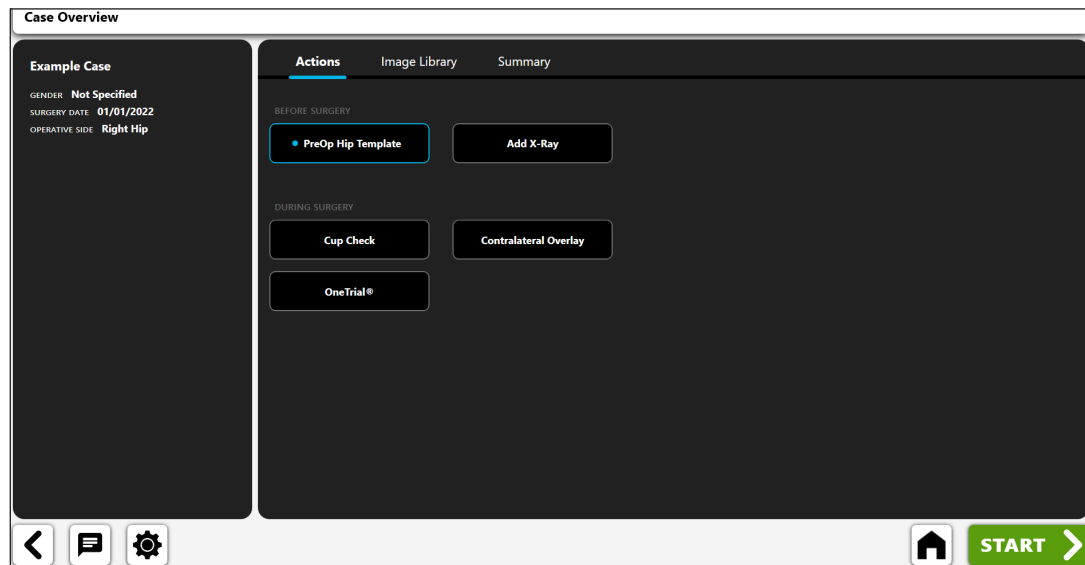
Figure 16: Case Overview Screen – Workflow Options

*PreOp Hip Template Action selected as shown, start button can be selected to enter the template analysis workflow*

## Adding an X-Ray

When you select the “Add X-Ray” button from the dialog above, you will be presented with the following dialog. The camera is currently only available on the iOS version and Direct Import is only available for the Windows version.

### Camera

On the iOS version, this feature allows you to capture X-ray images using the iPad camera.

### Direct Import

On the Windows version, this feature allows you to capture the output from a C-Arm.

### Case X-Rays

This feature allows you to use an X-ray already associated with the case.

### DICOM/Image File

This feature allows you to load an image from a DICOM file or local image file.

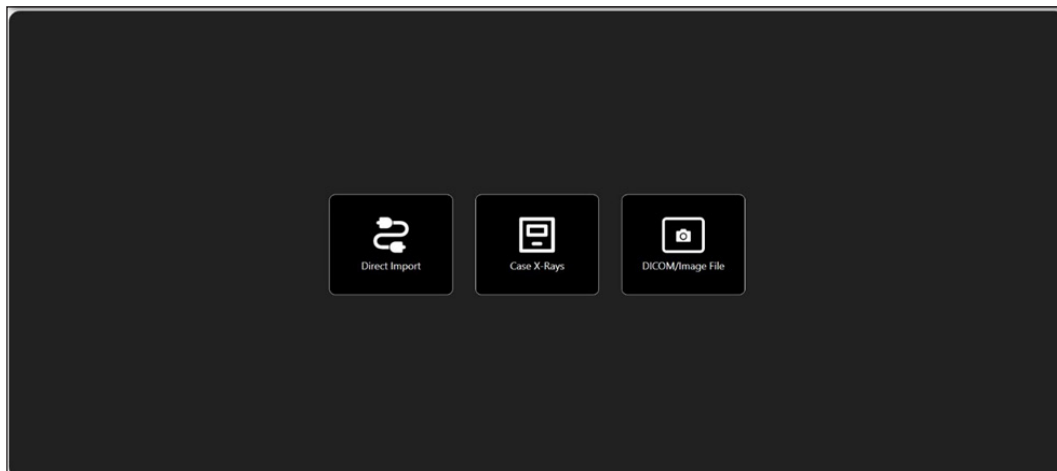
Figure 17: Import X-Ray Screen

Once an X-Ray is added it will appear under the “Image Library” heading.

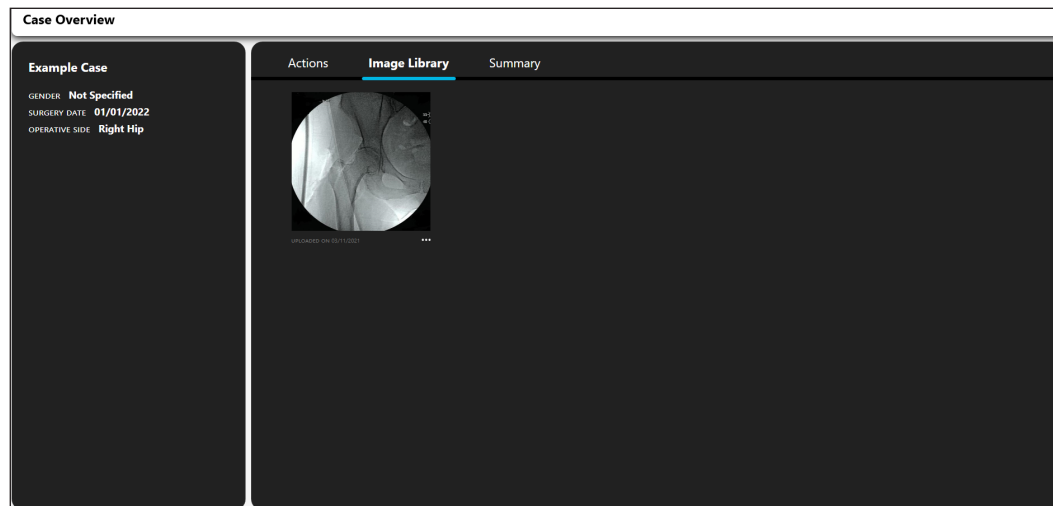
Figure 18: Case Overview Screen - Image Library Tab

## X-Ray Options

The collection of X-rays added to a case have some additional options. To view the options, select the ellipsis (...) from the bottom right corner.

Figure 19: X-Ray Options Drop-Down

### Invert Image

This option will cause a copy of the X-ray to be mirrored horizontally.

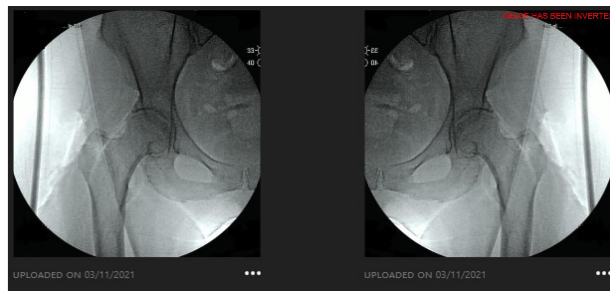
Figure 20: Invert Image Detail

### View

This option will open the X-ray in a larger window.

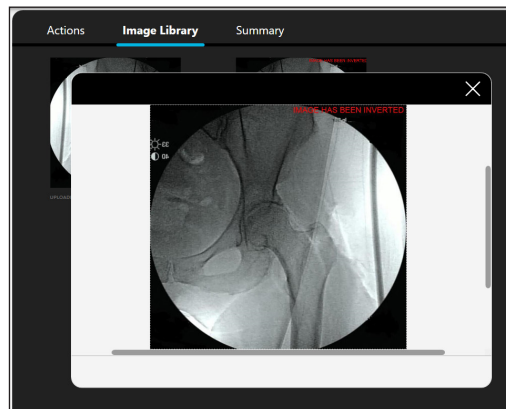
Figure 21: X-Ray View Detail

### Delete

This option removes the X-ray.

## Templates & Analysis

Workflows are located under the “ Actions” heading. When you add new workflows, they will appear under the Summary heading and allow you to view/edit or remove them.

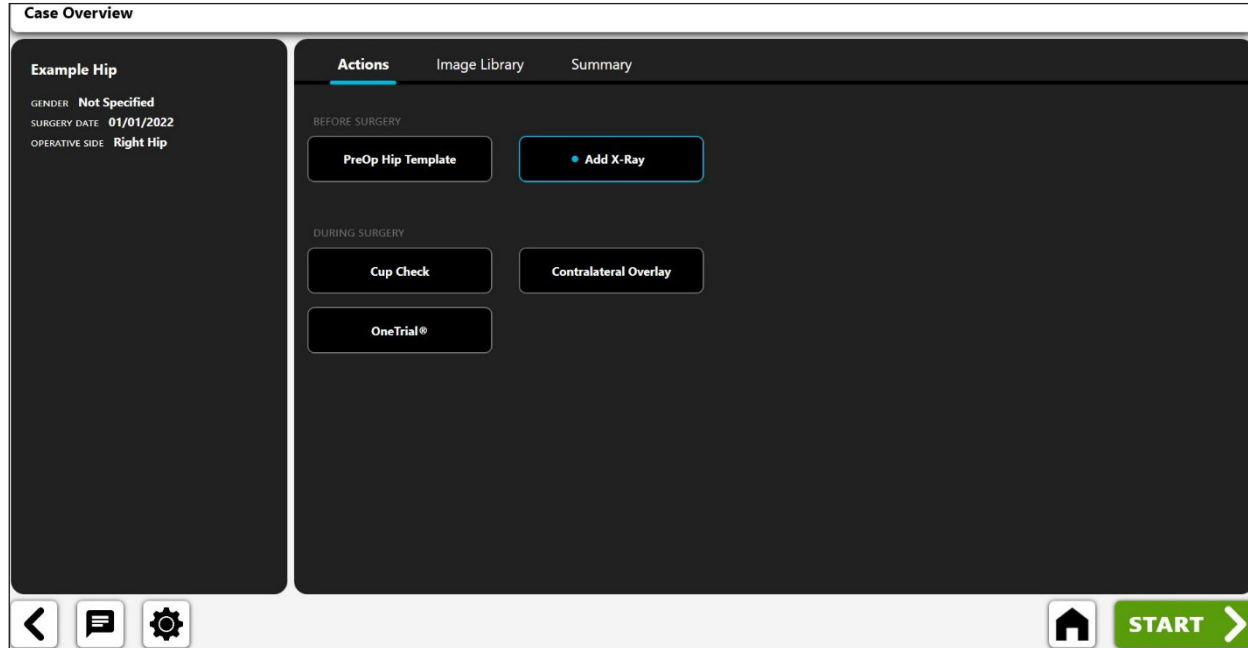
Figure 22: Case Overview Screen - Actions Tab

Summary To open a workflow, select the image or select the ellipsis (...) in the bottom right corner and choose “View”. To remove a workflow, select the ellipsis (...) and choose “Delete”. Patient details and previously saved analysis notes appear on the left side of the screen for quick reference.

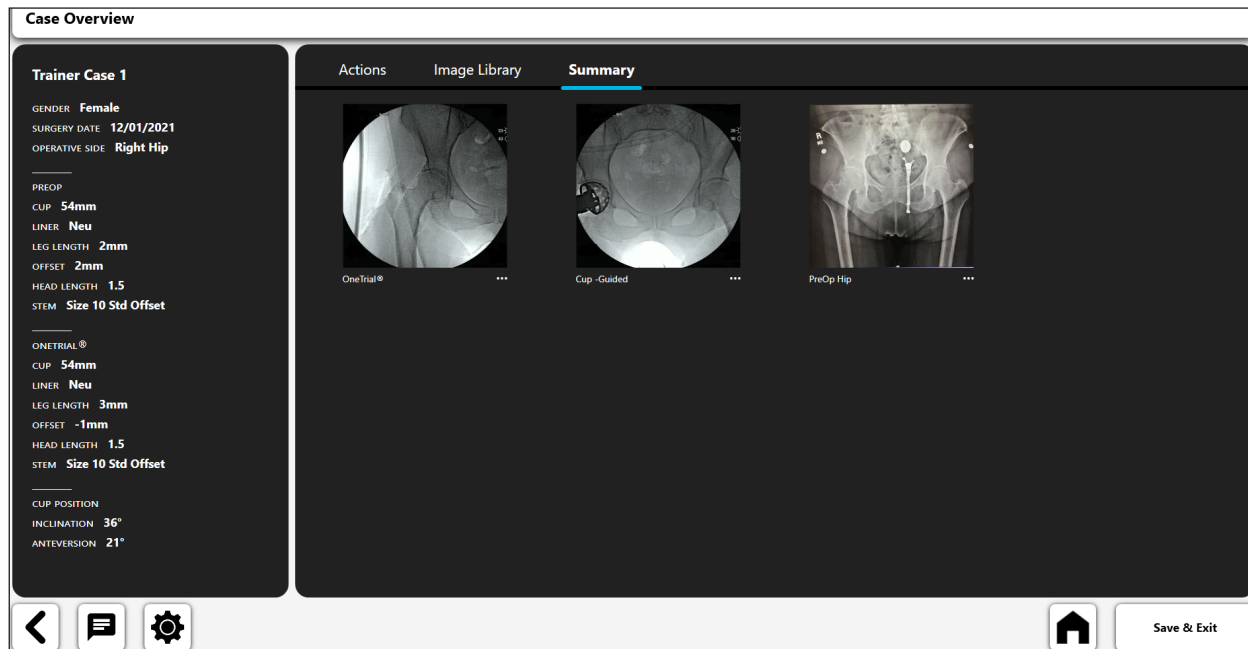
Figure 23: Case Overview Screen - Summary Tab

## Preoperative Hip Workflow

VELYS™ Hip Navigation's preoperative templating workflow provides all the features traditionally found in preoperative templating systems, including:

- Image acquisition
- Image calibration using a line or ball marker
- Template selection of the femoral stem and acetabular cup
- Digital Annotations (i.e. digital lines and circles)
- Leg length Analysis tool

Additionally, VELYS™ Hip Navigation's **portability and dynamic rescaling** technology also provides unique capabilities compared to other preoperative templating systems.

### Portability in the OR

VELYS™ Hip Navigation's portability enables you to create a template in the OR, during surgery but before implant insertion. This may be valuable in revision surgeries when existing hardware isn't known until the time of surgery.

One possible workflow for creating a preoperative template in the OR follows:

- A fluoro image is taken. User captures this image in VELYS™ Hip Navigation using the device camera
- Surgeon begins surgery and gains direct exposure to the existing acetabular cup, at which point they can visually identify the cup size.
- A user operates VELYS™ Hip Navigation outside of the sterile field and creates a new preoperative template using the fluoroscopic image.
- The VELYS™ Hip Navigation user calibrates the preoperative template image using the known size of the visually identified acetabular cup.
- The user continues with their preoperative templating exercise.
- The surgeon visually verifies the results and uses the information provided in the template.
- If desired, the template can also be displayed on a monitor in the OR using a display cable.

### Dynamic Rescaling

Surgeons have historically relied upon ball markers or other sizing devices as they template. However, a ball marker measurement is sometimes imprecise because of patient anatomy, positioning, or image parallax.

To address this, VELYS™ Hip Navigation **provides a unique capability to dynamically rescale an existing template**. This functionality can be valuable in the OR during a revision surgery when direct observation of a pre-inserted prosthetic such as an acetabular cup may be used during surgery to re-scale a preoperative image more precisely.

Use of this feature involves initially using a sizing device to create a preoperative template. Afterward, you can rescale the entire preoperative plan according to the pre-inserted acetabular component. This will enable the surgeon to update their plan with more precisely scaled data.

## Creating a Preoperative Hip Workflow

### 1. Select PreOp Hip Template

On the Actions Screen, select “PreOp Hip Template” then the “START” button.

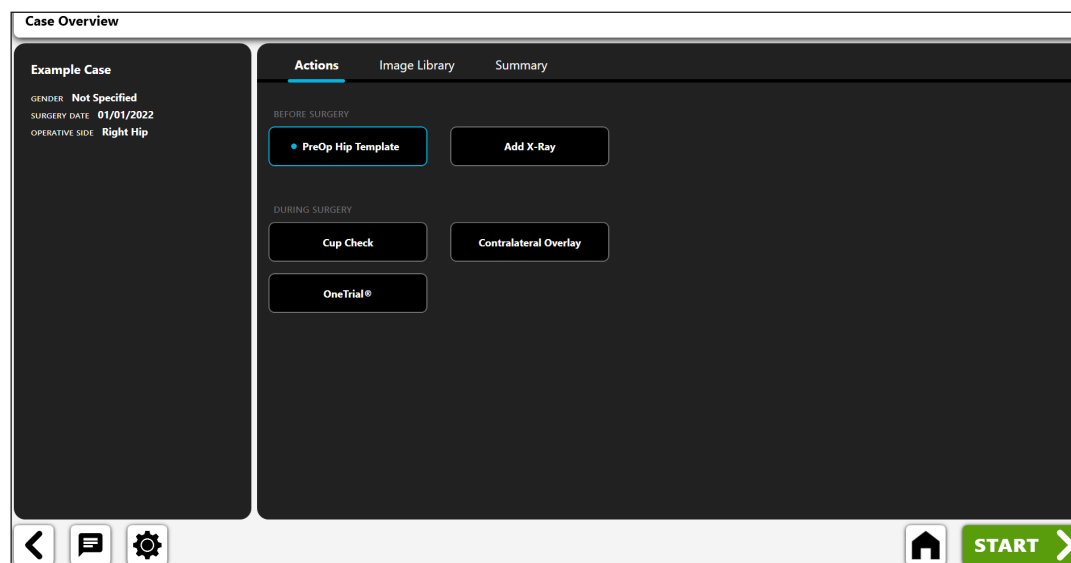
Figure 24: Case Overview Screen - PreOp Hip Template

### 2. Import X-Ray

Capture an X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

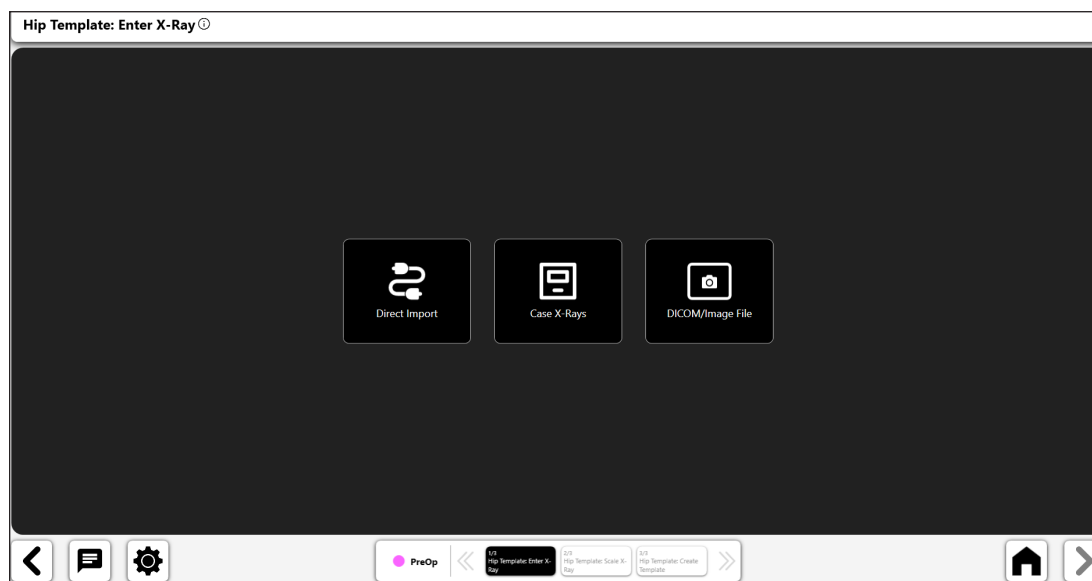
Figure 25: Import X-Ray Screen

### 3. Scale the X-Ray.

You begin the preoperative templating process by scaling the image using an object of known size.

- Choose whether you will use a ball marker or a straight line using the Toggle control (No. 1).
- If you use a ball marker to calibrate, a digital circle will appear in the screen (No. 2). VELYS™ Hip Navigation's image recognition will try to auto-identify a ball marker on the image and position the circle around it.
- Modify the position and size of the circle using the control handles so that it matches the object used to calibrate the image.
- Enter the known size of the object in the data entry box in the top left of the screen (No. 3). This is a diameter measurement if using a ball marker, and the actual line size if using 'ruler' calibration.
- **TIP: The standard ball marker size is 25.4 mm. However, you should always verify the ball marker size before calibrating an image.**
- **TIP: If you create your preoperative template in the OR (but before implant insertion), you can calibrate the image using the extracted femoral head. You can do this by drawing a 'ruler' line across the width of the femoral head, and entering the size measured using calipers.**

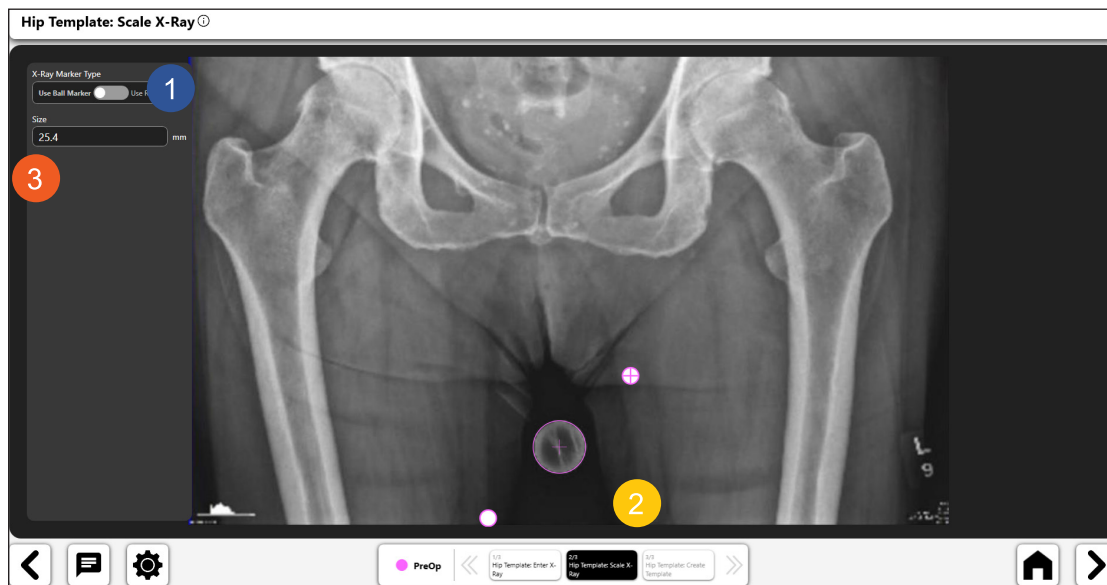
Figure 26: Scale X-Ray Screen

#### 4. Create the Preoperative Hip Template

You can create a preoperative template after completing calibration. You can create the template using the variety of tools available from the templates. The tools are described below.

*Leg Length Analysis Tool:* 

- Select the leg length icon on the toolbar to add the leg length tool (No. 1) to the image.
- You can modify the leg length control using its navigation controls.
- You should position the blue dots on the central line to define the horizontal access of the pelvis. (For example, you can position them on the teardrop points or ischial tuberosity.)
- Position the red lines to touch the top of the lesser trochanters (or other identifiable femoral landmarks). The system uses these lines to calculate leg length inequalities.
- Leg length information is displayed on screen (No. 2). You can move the information display by selecting and dragging it.

**TIP:** Using the tear drop for the horizontal pelvic line (as shown in the image) is often more precise.

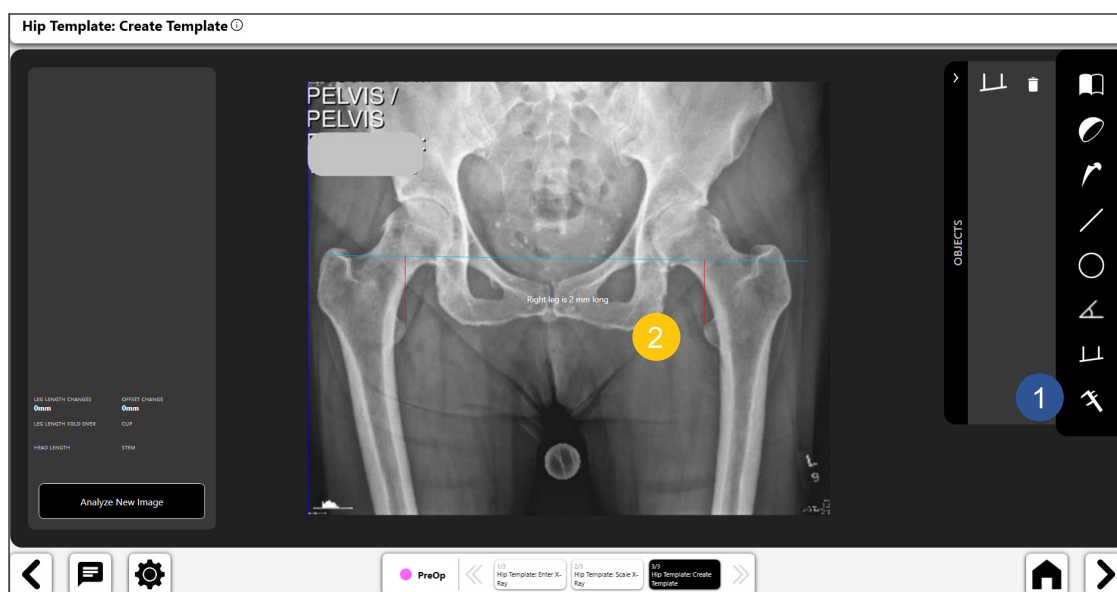
Figure 27: Create Template Screen

### Acetabular Cup Template Tool:

- Select the acetabular cup template icon to display the template selection panel (No. 1) below.
- Select the desired cup model in the panel .
- The acetabular cup template will display above the image, as shown (No 1) below. Move the template by dragging it into position over the femoral head and acetabulum. Then, use the rotation handle to rotate the template into position.
- Update size, offset, and head information for the acetabular template using the input boxes on the top left of the form (No. 2).
- Figure 28, No. 3 shows a 'toggle icon' for the acetabular template, showing that it is activated. You can delete the template by selecting the trash can icon displayed within the toggle icon.

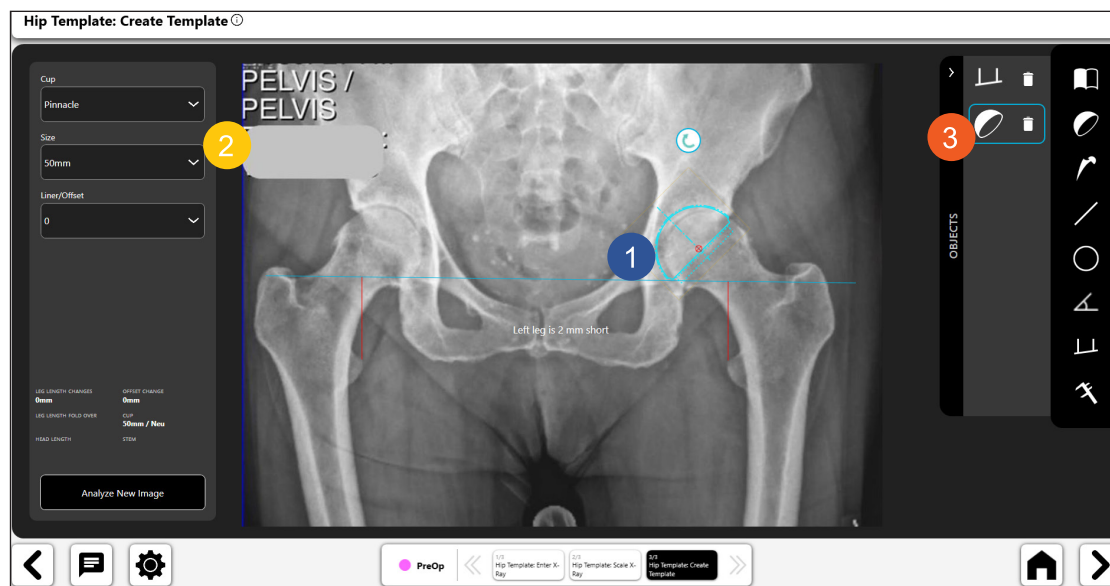
Figure 28: Create Template Screen - Acetabular Cup Tool

### Femoral Stem Template Tool:

- Select the femoral stem template icon to display the template selection panel, shown below.
- Select the femoral stem model.
- The femoral stem template will display above the image, as shown in (No 1) below. Move the template by dragging it into position along the femoral canal. You can use the rotation handle to rotate the template into position.
- Update size, offset, and head information for the femoral stem template using the input boxes on the left of the screen (No. 2).
- No. 3 shows a selected 'toggle icon' for the femoral stem template, showing that it is activated. You can delete the template by selecting the trash can icon displayed within the toggle icon.
- **TIP: You can change the active object by selecting the toggle icon for the object you want to activate.**
- **TIP: Press the Rescale button (No. 4)  to recalibrate an image. You can use this during surgery when the surgeon measures the extracted femoral head.**

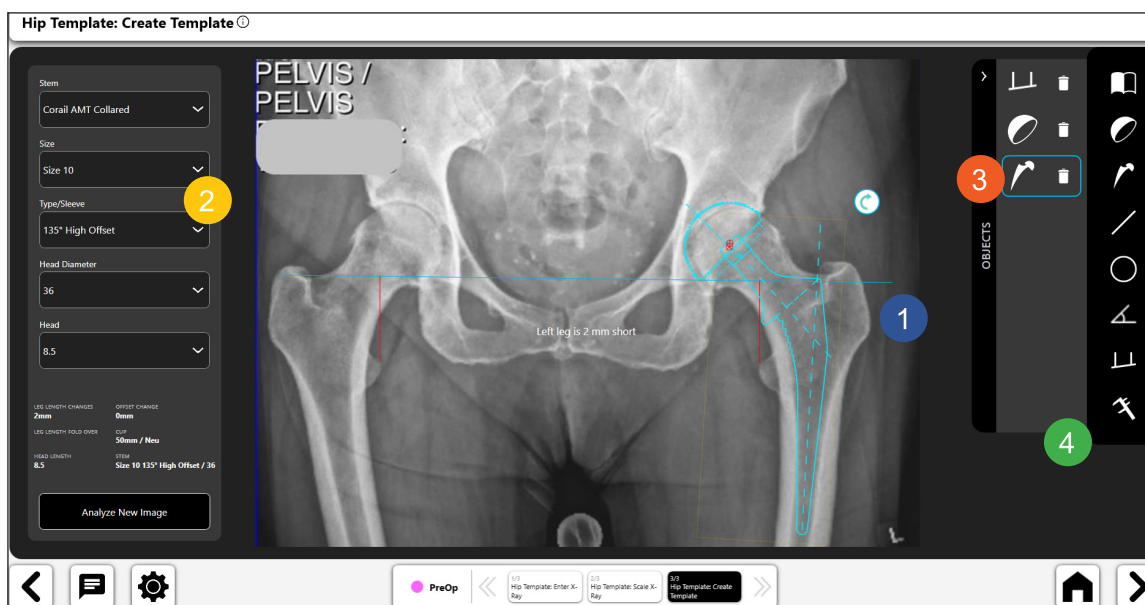
Figure 29: Create Template Screen - Femoral Stem Tool

#### Digital Line Tool:

- Select the digital line icon to create a digital line (No. 1).
- You can manipulate the length and position of the line using the navigation handles.
- The line length calculation will automatically update as you change the line length.
- **TIP: You can create multiple lines. Each line will have a different color.**

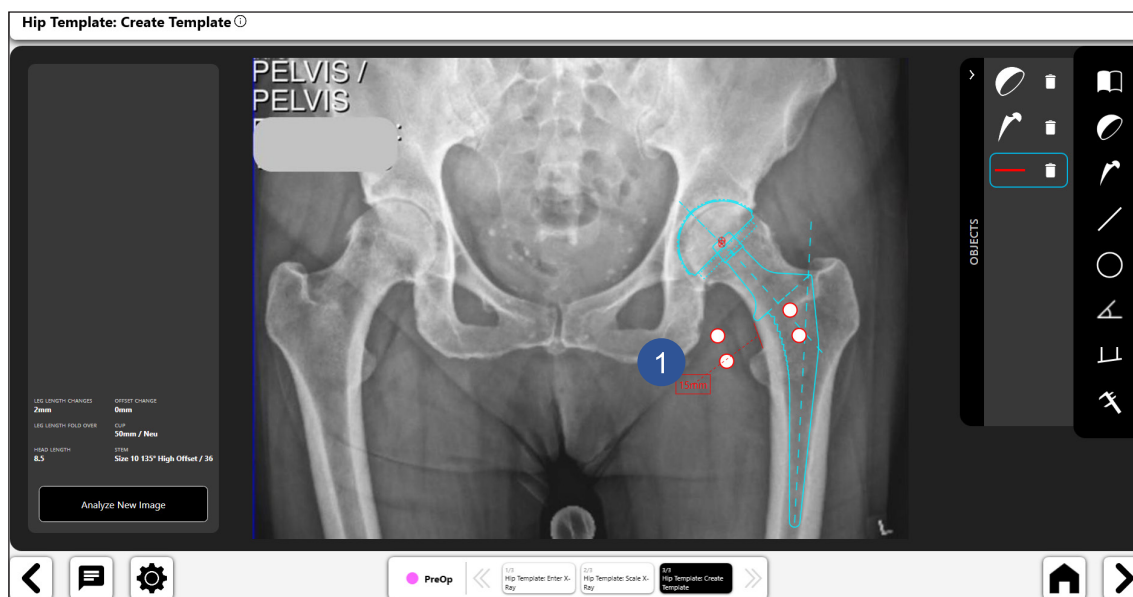
Figure 30: Create Template Screen - Digital Line Tool

### Digital Circle Tool:

- Select the digital circle icon to create a digital circle (No. 1).
- You can manipulate the size and position of the line using the navigation handles.
- The displayed circle size is the diameter of the circle.
- You can create multiple circles.

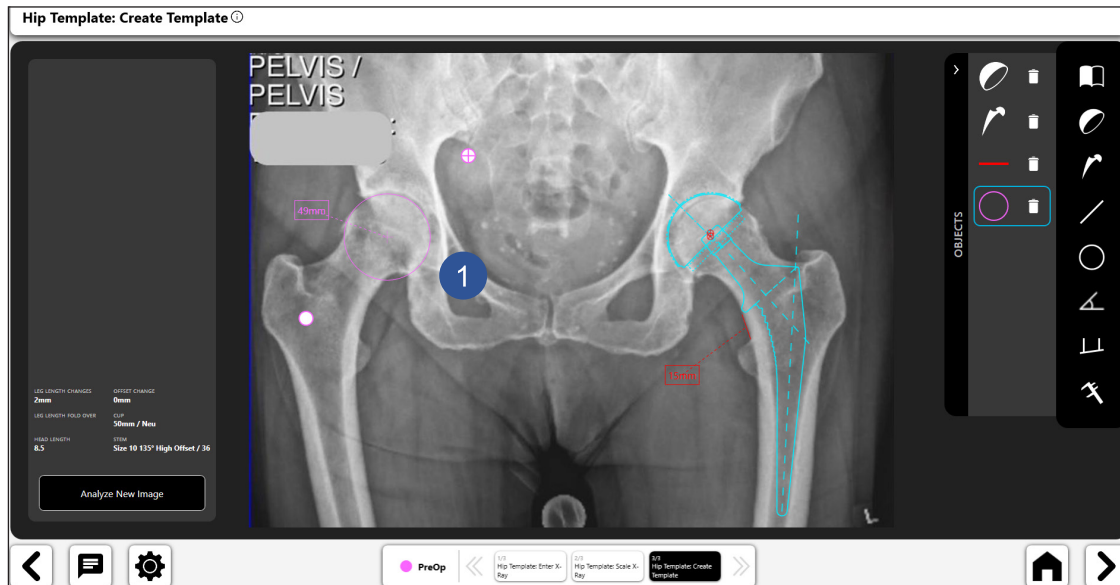
Figure 31: Create Template Screen - Digital Circle Tool

### Digital Angle Tool:

- Select the digital angle tool icon to create a digital angle (No. 1).
- You can manipulate the two lines and the angle is calculated.
- The two lines can be moved into various positions to calculate different angles.

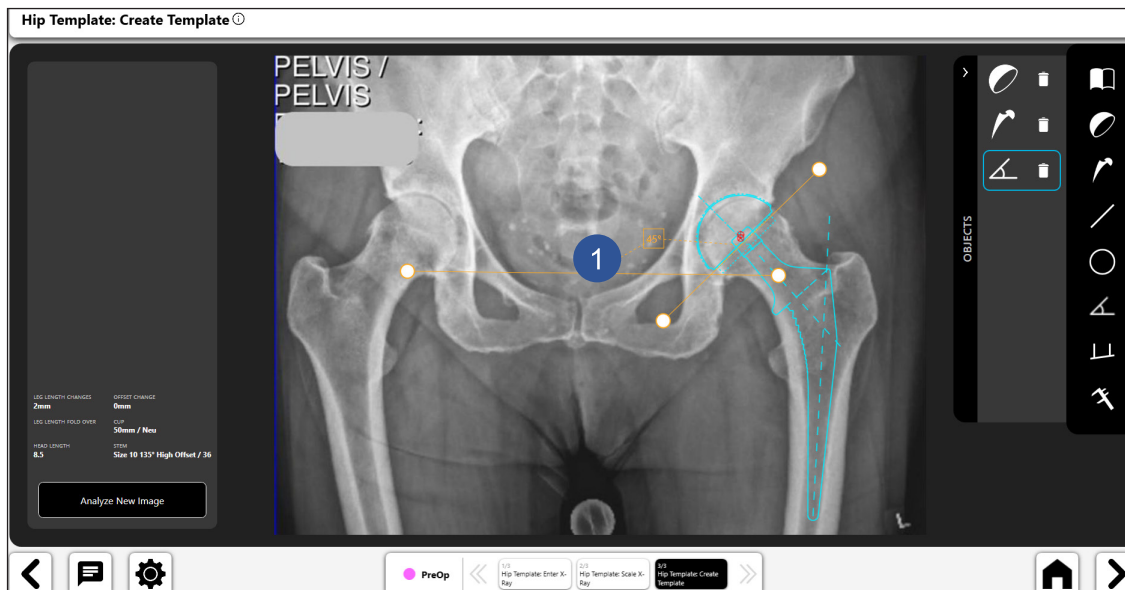
Figure 32: Create Template Screen - Digital Angle Tool

**Preoperative Fold Tool:** 

- Select the preoperative fold icon to create a Preoperative Analysis between the operative and non-operative sides of the AP pelvis X-ray.
- The process requires placing anatomical landmarks on the AP pelvis X-ray as follows:
  - Operative Side
    - Mark greater trochanter

Figure 33: Mark Greater Trochanter (Op Side)

- Draw circle around the femoral head

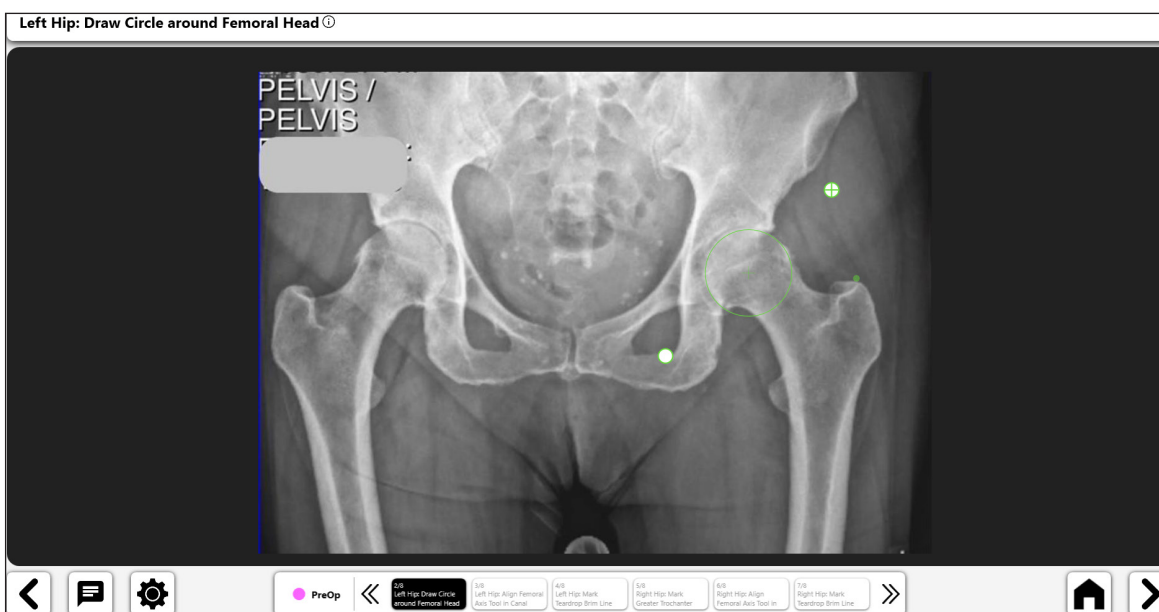
Figure 34: Femoral Head Circle (Op Side)

- Align femoral axis tool in canal

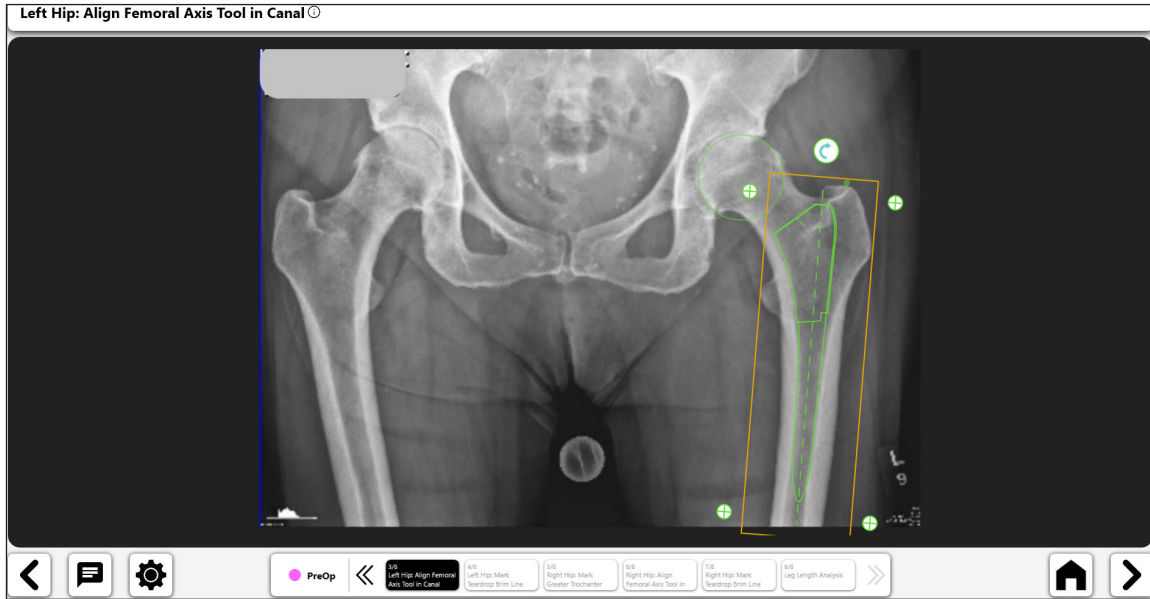
Figure 35: Align Femoral Axis Tool (Op Side)

- Mark teardrop brim line

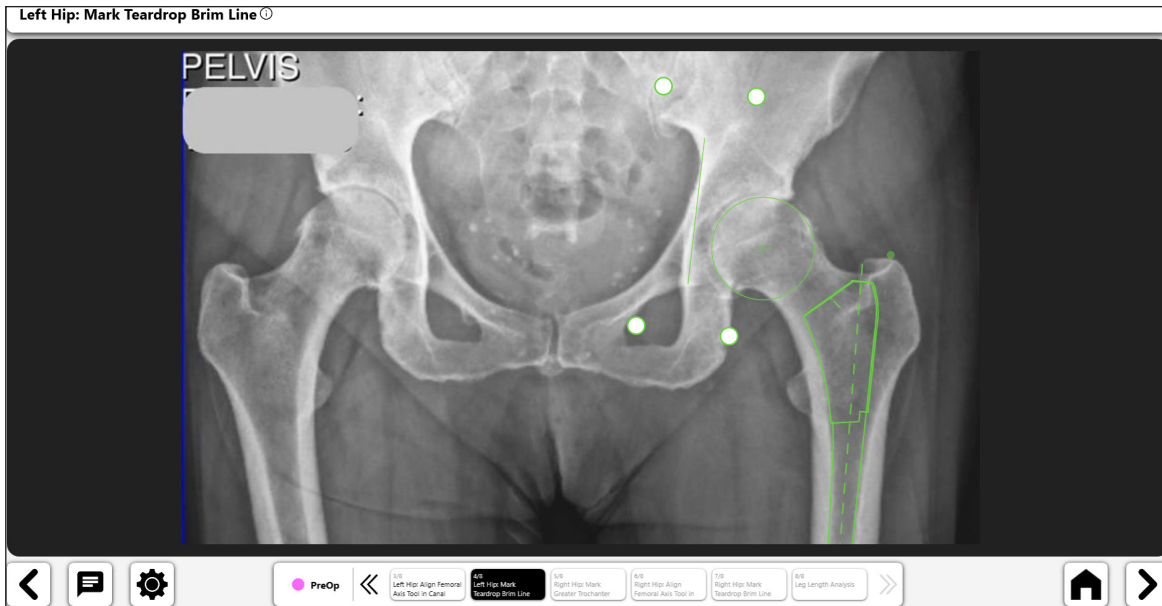
Figure 36: Mark Teardrop Brim Line (Op Side)

- Non-operative Side
  - Mark greater trochanter

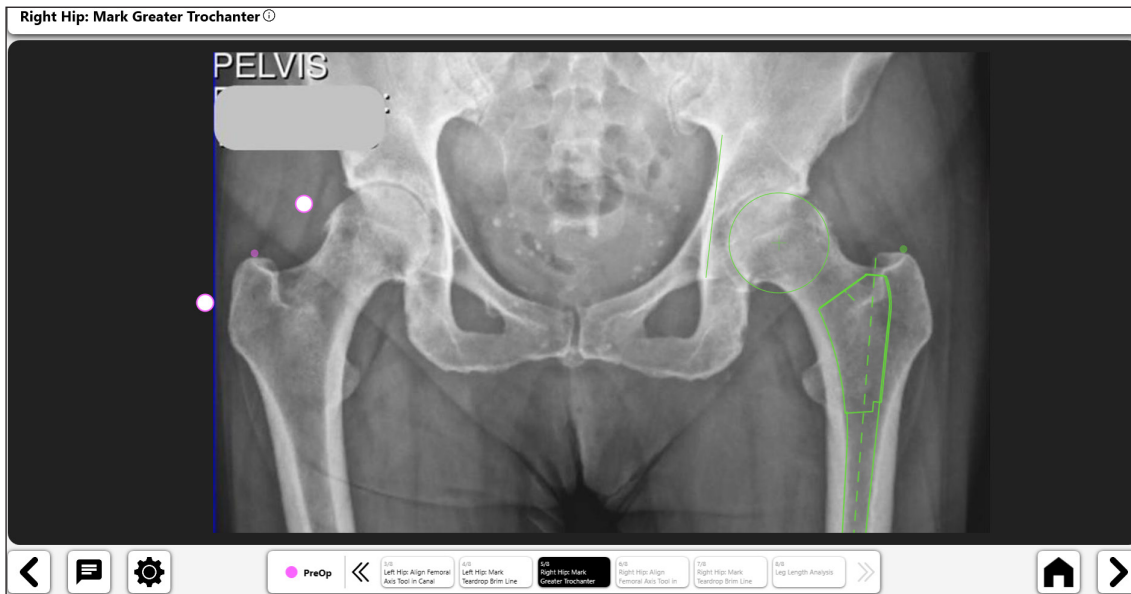
Figure 37: Mark Greater Trochanter (Non-Op Side)

- Align femoral axis tool in canal

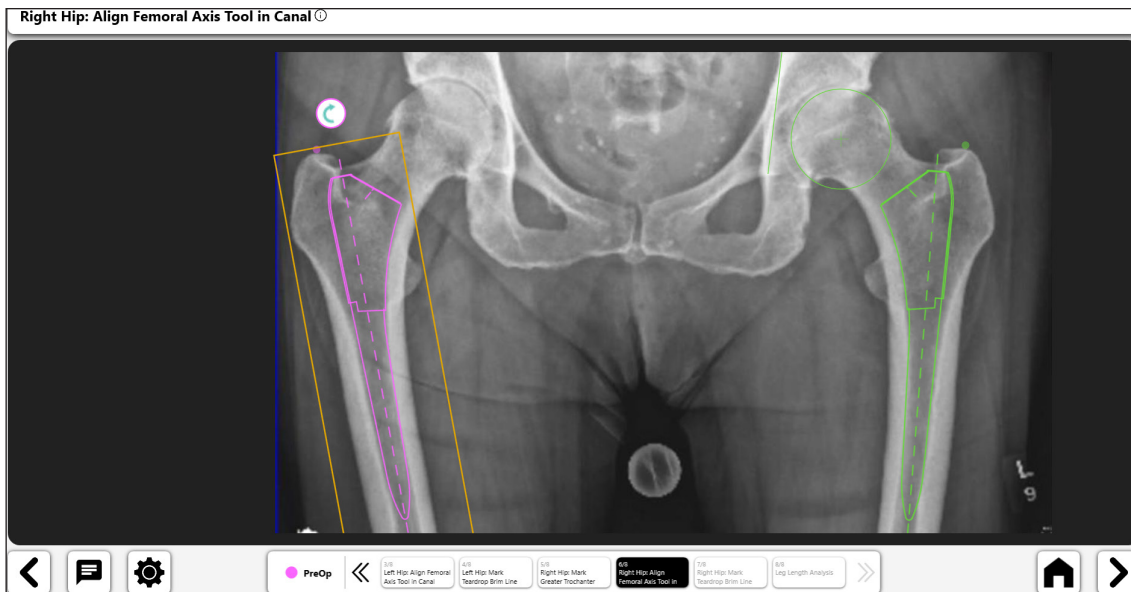
Figure 38: Align Femoral Axis Tool (Non-Op Side)

- Mark teardrop brim line

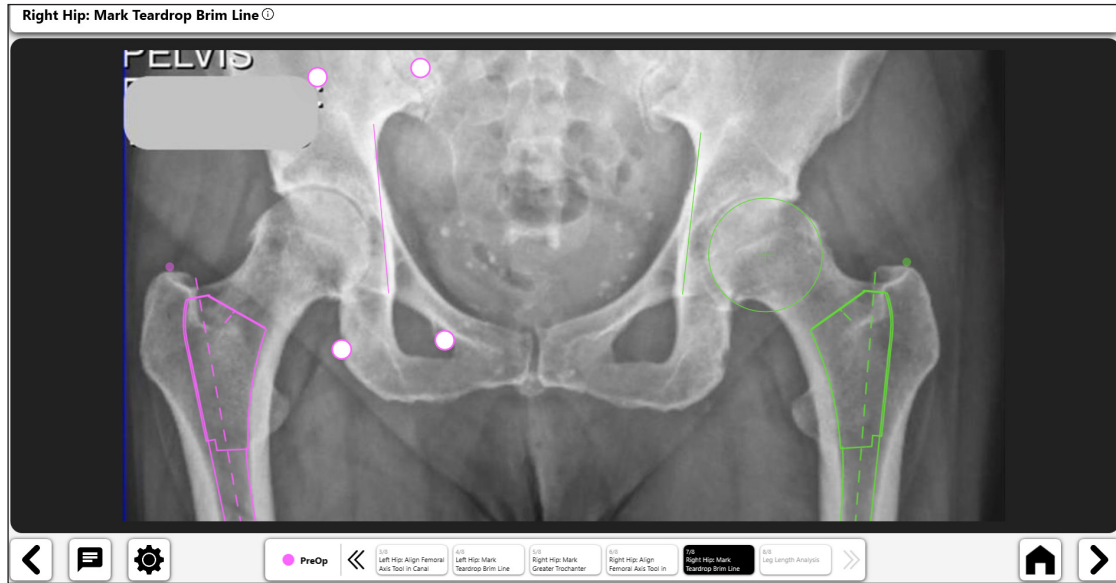
Figure 39: Mark Teardrop Brim Line (Non-Op Side)

- The final analysis displays the leg length in relation to the operative and non-operative sides. The “Femoral Alignment Correction” switch will display details about leg length differences.

Figure 40: Leg Length Analysis Screen

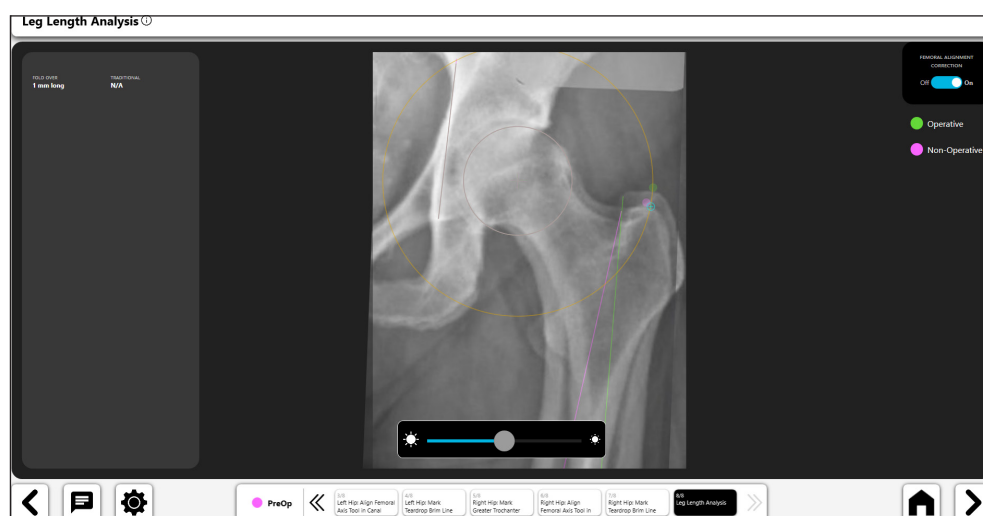
Figure 41: Leg Length Analysis Screen - Femoral Alignment Correction

## 5. Final Preoperative Template

The preoperative template provides various features and information. Figure 42 below shows an example of a completed template with associated information.

- **Toggle icons (No. 1)** allow you to activate controls on the page. Only one control can be active at a time. You can activate a control by selecting the toggle button for that control.
- The Heads-Up Display (“HUD”) shown in No. 2 shows the template data that is being used for the preoperative template as well as leg length and offset data.
- Upon finishing a preoperative template, you can save the template for later use by selecting the next Arrow near the bottom right of the screen (No. 3).

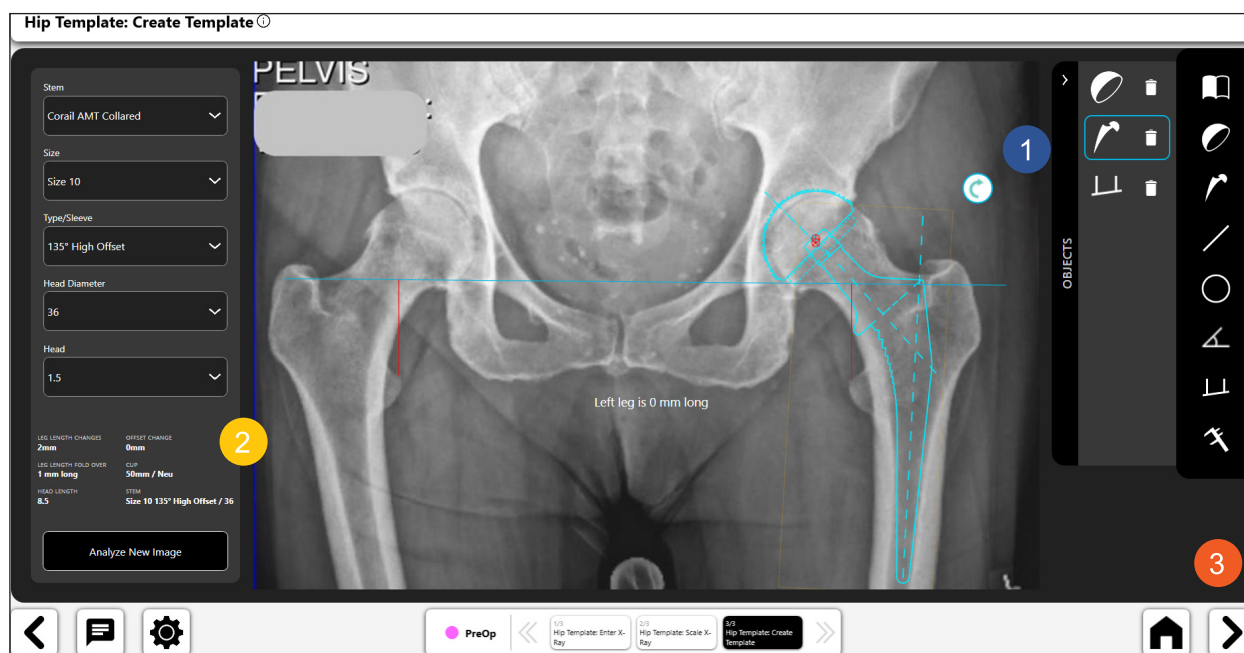
Figure 42: Final PreOp Template

## OneTrial™ Workflow

### Creating a OneTrial™ Workflow

#### 1. Select OneTrial™

On the Case Overview Screen, choose “OneTrial™” from the available options on the “Actions” tab.

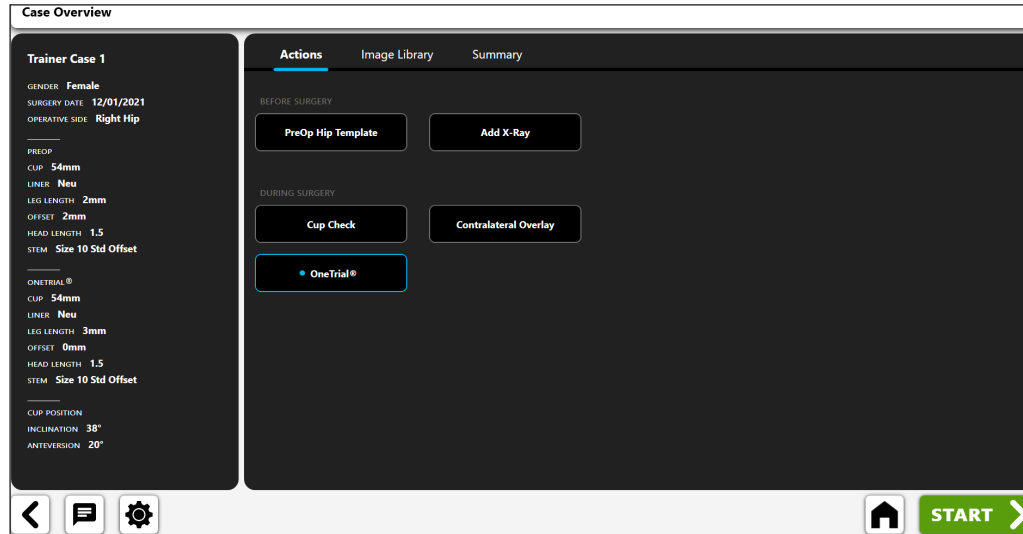
Figure 43: Case Overview Screen - OneTrial™

#### 2. Import Non-Operative X-Ray

- Center acetabulum on screen
  - Image must include: greater trochanter, femoral diaphysis, pubic symphysis, and pelvic bone above acetabulum
- Place leg in 10 Degrees of internal rotation and rainbow C-Arm 10 Degrees over the top to show patient's true offset

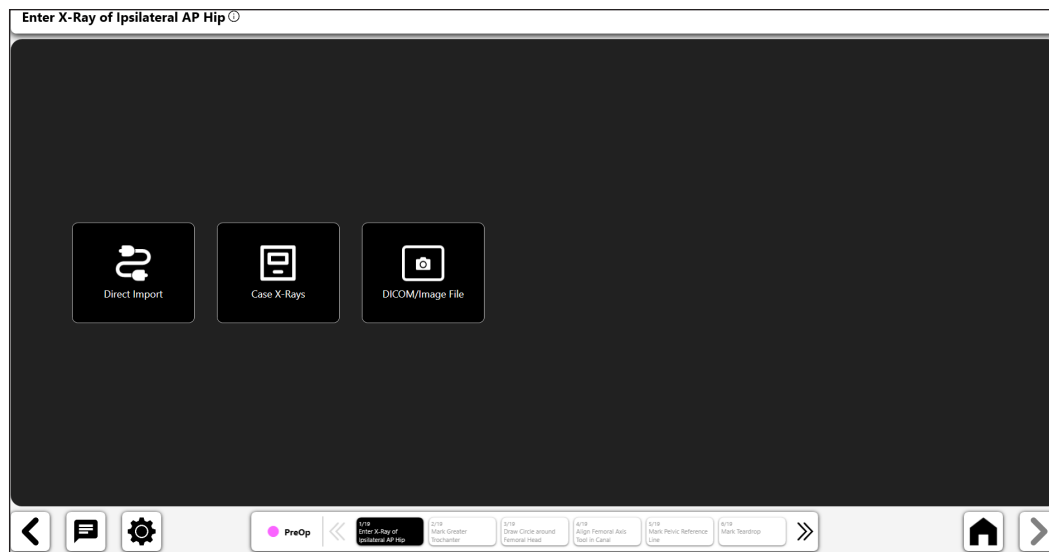
Figure 44: Import X-Ray Screen – NonOp Side

### 3. Greater Trochanter

- Best point is generally either a dark, well-defined edge or highest point on the trochanter
- Point can be placed above or directly on the landmark

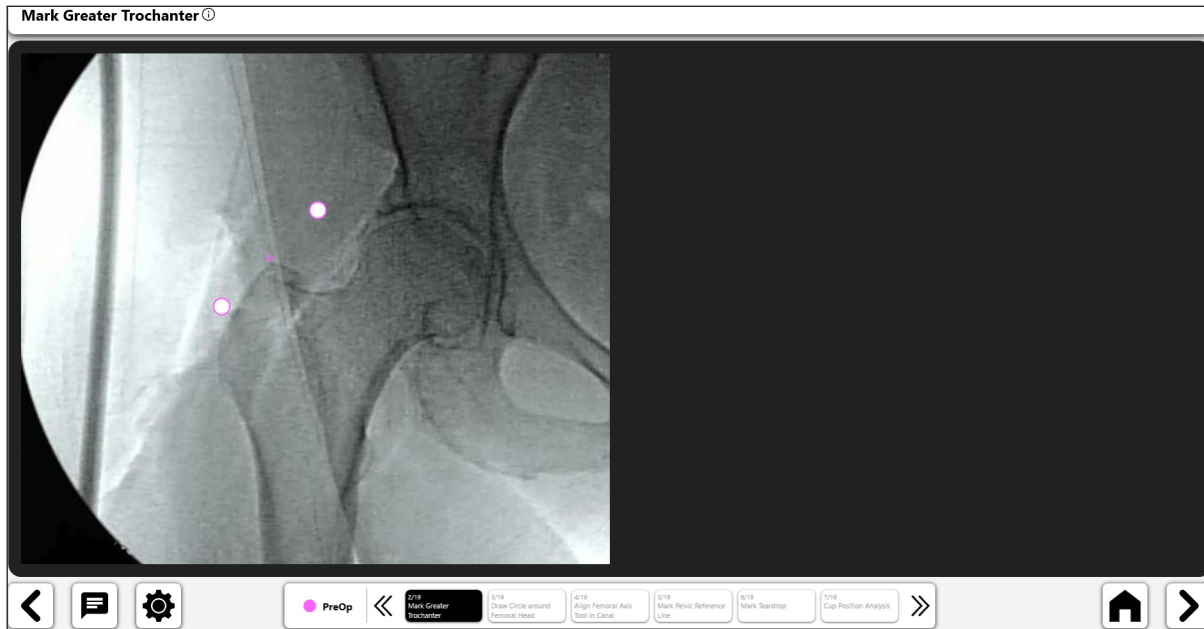
Figure 45: Mark Greater Trochanter (NonOp Side)

### 4. Draw Circle On Femoral Head

- This circle approximates the center of rotation and will help define femoral offset
- In cases of head collapse approximate the current head center of rotation

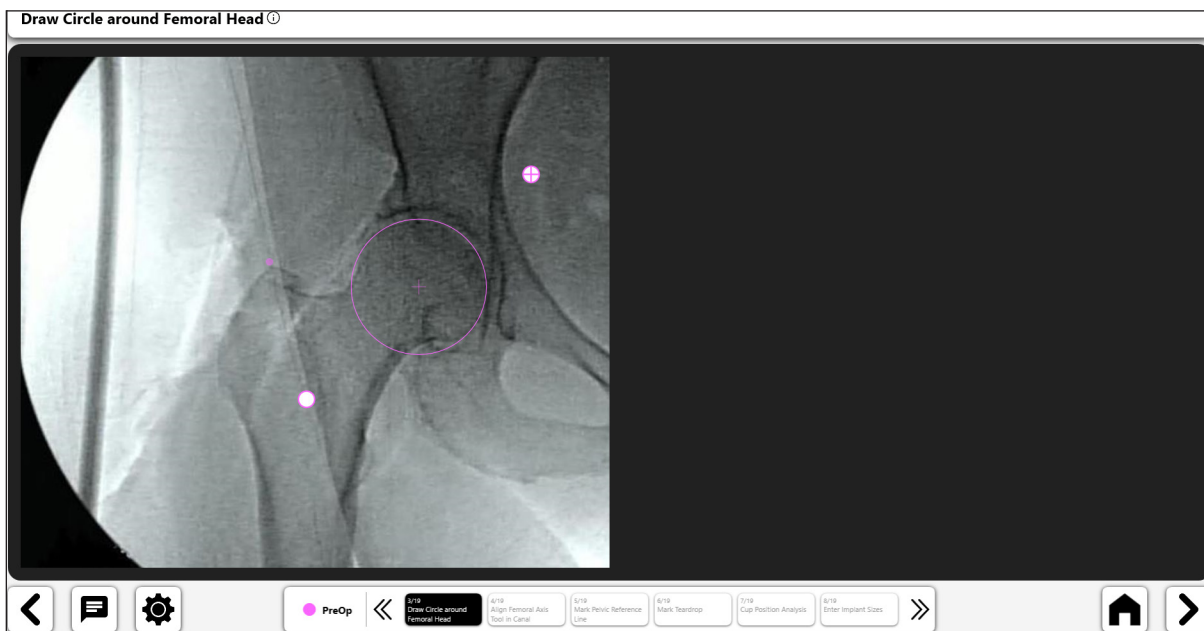
Figure 46: Draw Circle On Femoral Head (NonOp Side)

## 5. Align Femoral Axis Line Tool Down Canal

- Size the tool to fill the medullary canal and then align with the long axis of femur

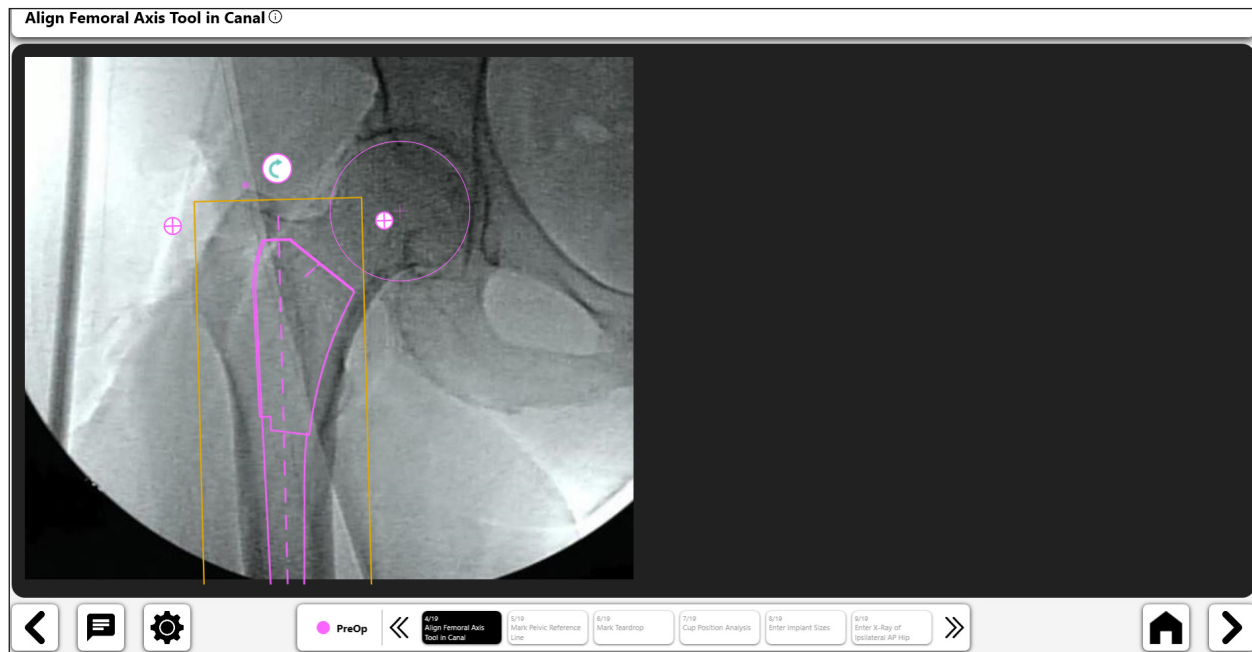
Figure 47: Align Femoral Axis Tool (NonOp Side)

## 6. Mark Pelvic Reference Line

- Draw a Line from Pubic Symphysis to Identifiable Point Above Acetabulum

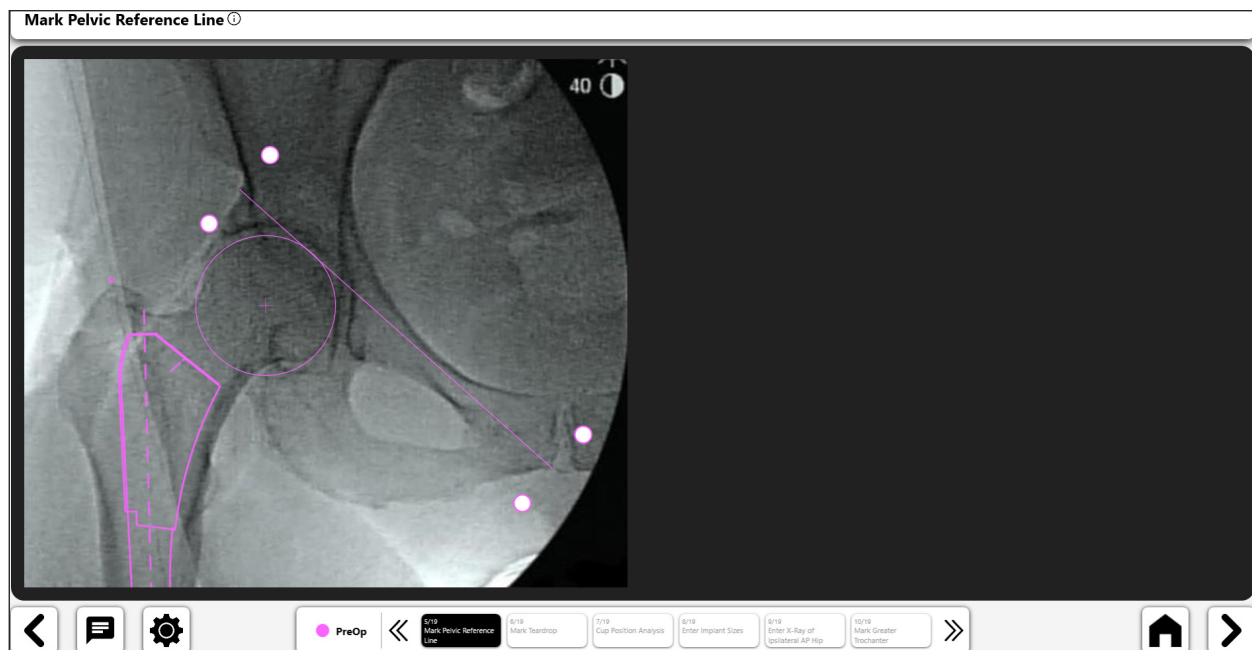
Figure 48: Mark Pelvic Reference Line (NonOp Side)

## 7. Mark Teardrop

- Best Point is often surrounded by the teardrop Edges or Dark Border
- For Hard to Identify Teardrop, use Superior Edge of Obturator Foramen

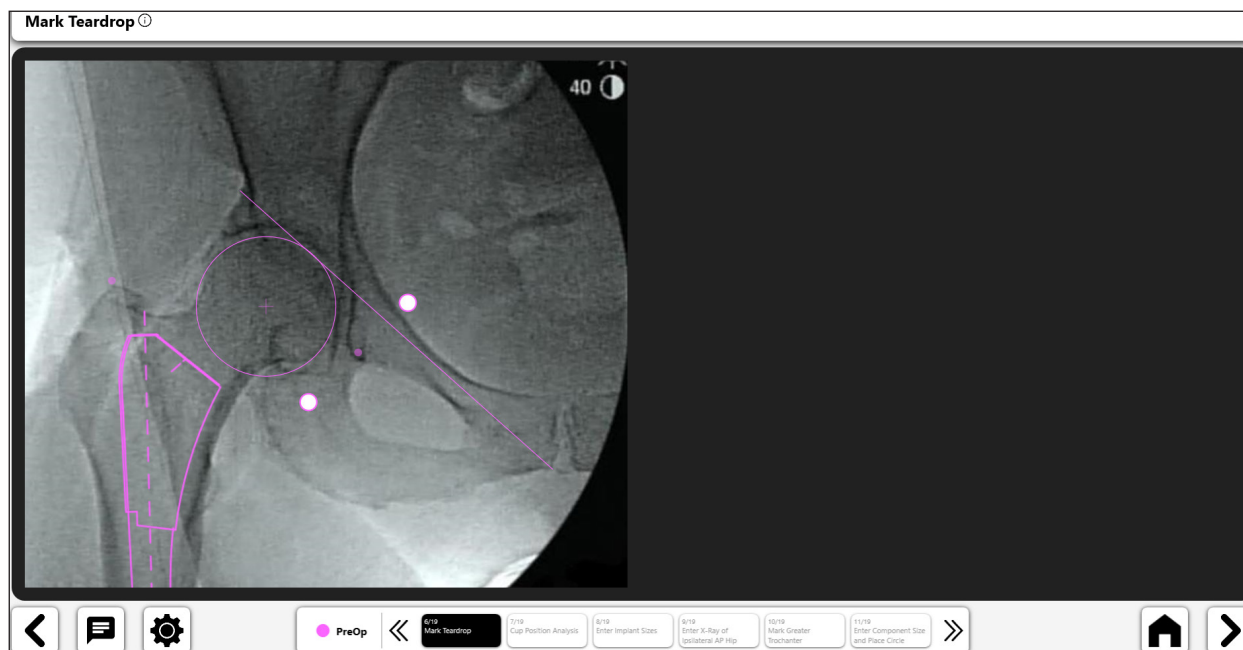
Figure 49: Mark Teardrop (NonOp Side)

## 8. Cup Position Analysis

At this point in the process you may choose to Analyze cup position (inclination and anteversion). See the Cup Check workflow process for more information.

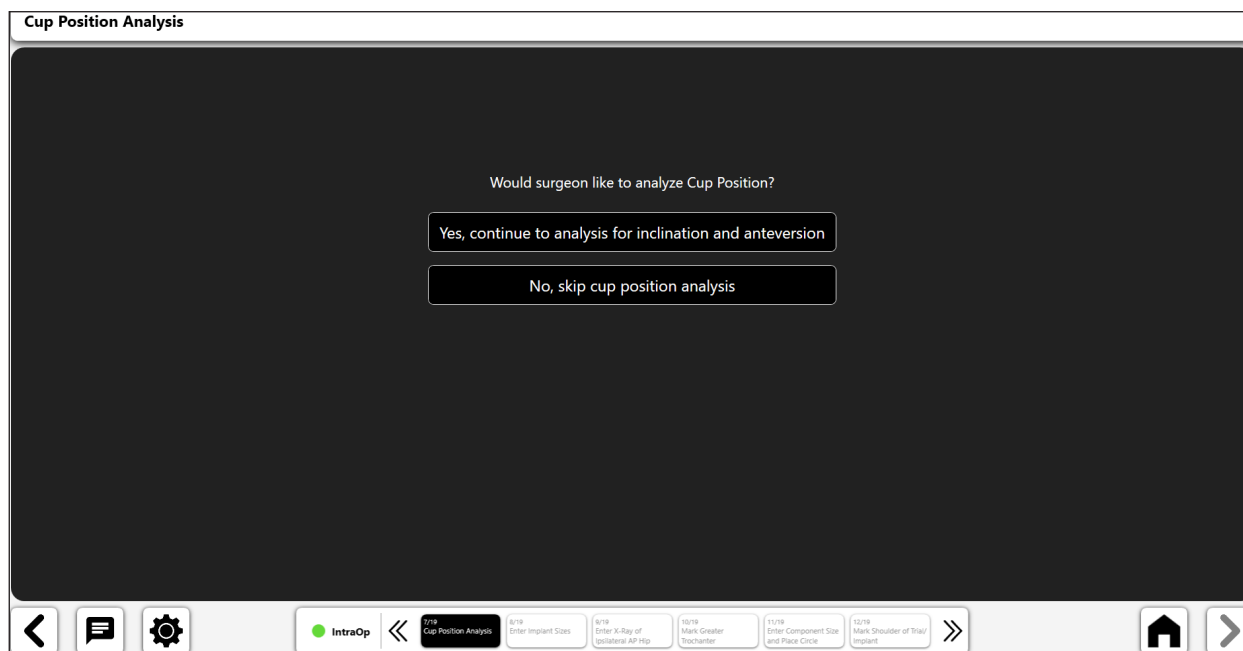
Figure 50: Cup Position Analysis Screen

### 9. Enter Implant Sizes

If you complete or skip the cup position analysis the next step in OneTrial™ is to select the implant components being used in surgery.

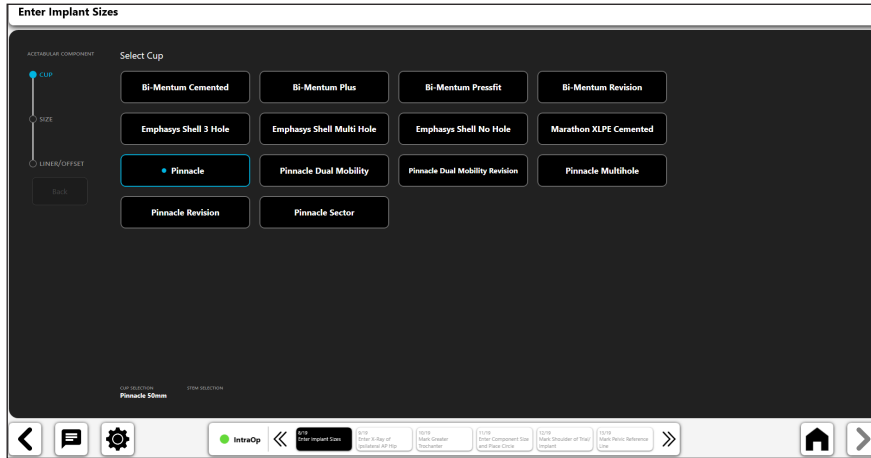
Figure 51: Enter Implant Sizes - Select Cup

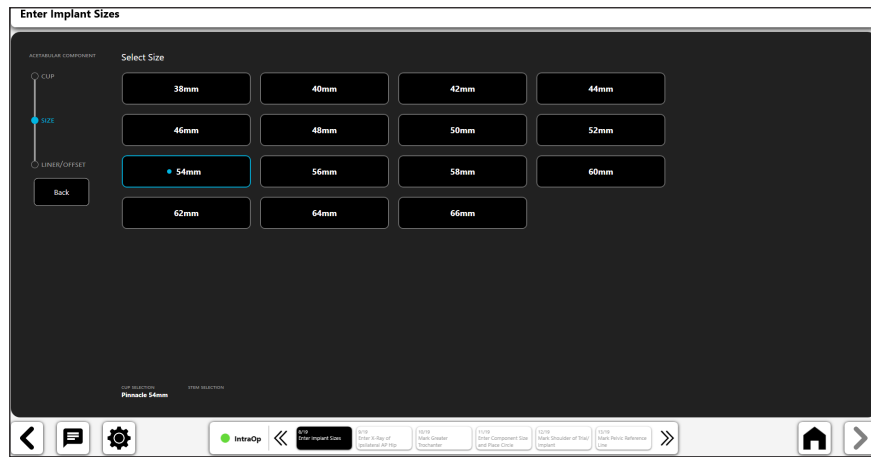
Figure 52: Enter Implant Sizes - Select Cup Size

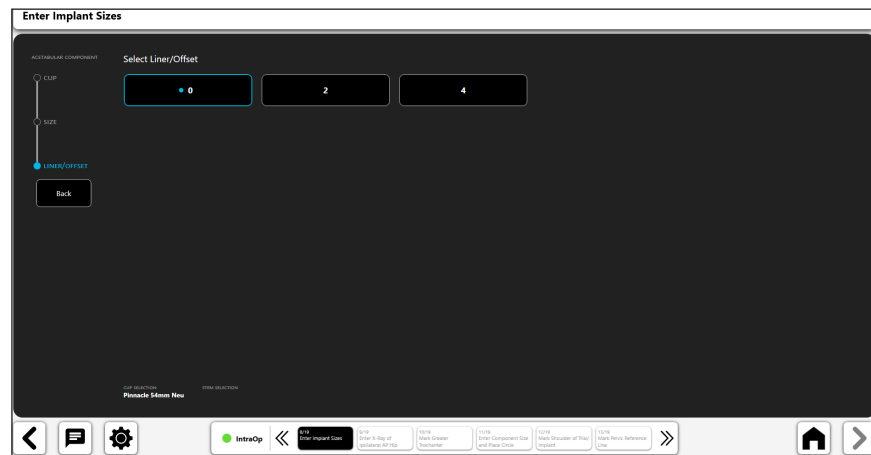
Figure 53: Enter Implant Sizes - Select Liner/Offset

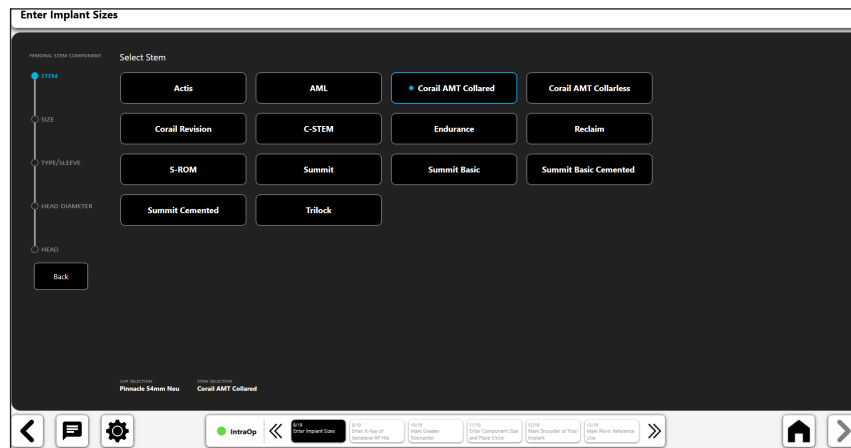
Figure 54: Enter Implant Sizes - Select Stem

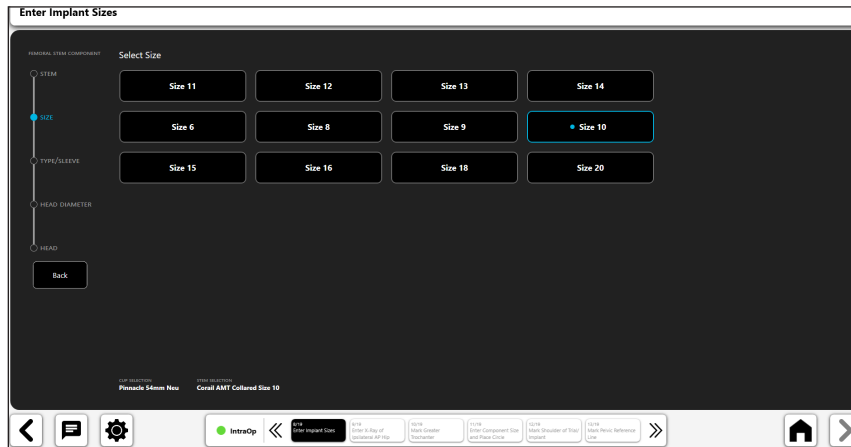
Figure 55: Enter Implant Sizes - Select Stem Size

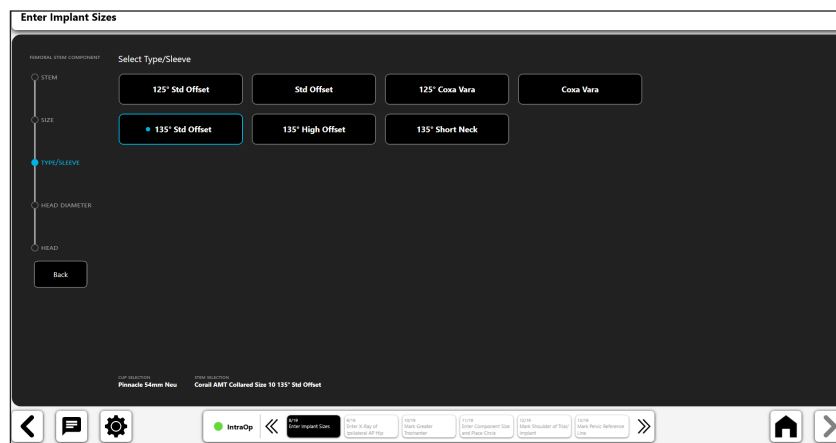
Figure 56: Enter Implant Sizes - Select Type/Sleeve

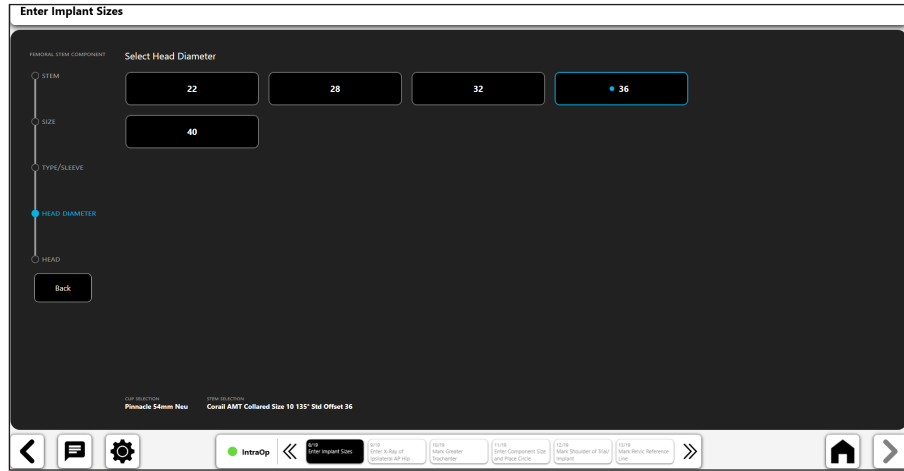
Figure 57: Enter Implant Sizes - Select Head Diameter

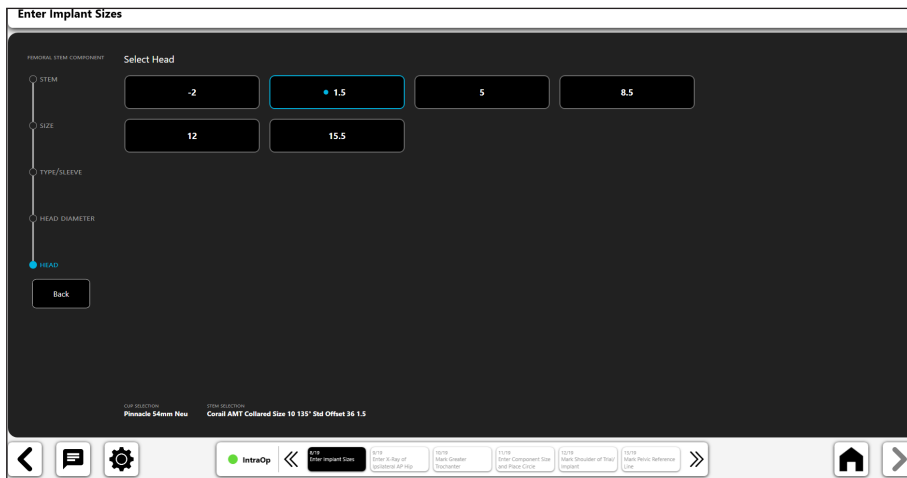
Figure 58: Enter Implant Sizes - Select Head

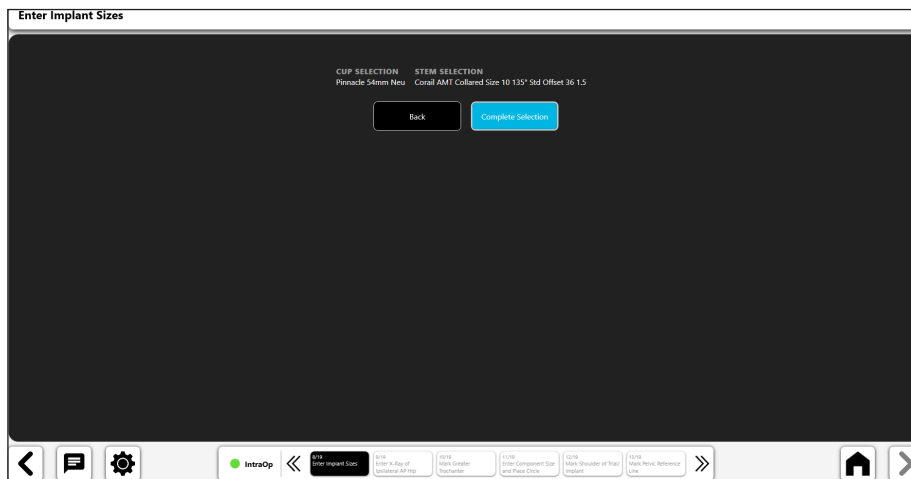
Figure 59: Enter Implant Sizes - Complete Selection

## 10. Import Operative X-Ray

- Capture an X-ray image from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.
- Maintain the same C-Arm position used in the pre-operative hip x-ray when taking the intraoperative hip x-ray
- Make sure to include all required anatomic points, including: Greater Trochanter, Femoral Diaphysis, Pubic Symphysis, Pelvic Bone above Acetabulum)
- Center the acetabulum on the x-ray
- Internally rotate the leg 10 Degrees and rainbow the C-Arm 10 Degrees over the top to help show the patient's true offset and match the femoral rotation from the pre-operative x-ray.

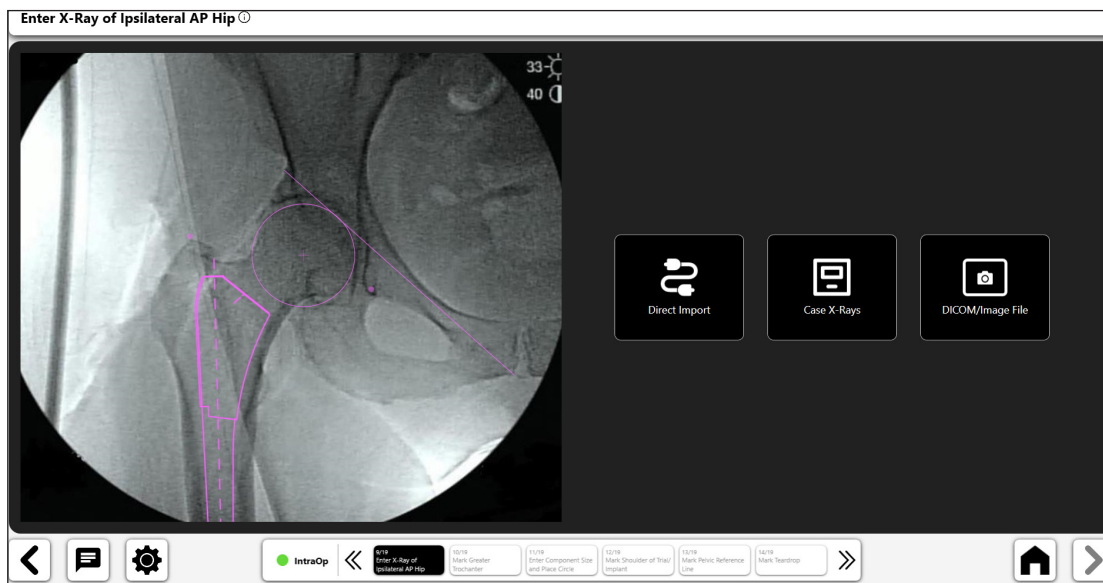
Figure 60: Import X-Ray Screen – Operative Side

## 11. Mark Greater Trochanter

- Place point on tip of the Greater Trochanter, matching the Preop point placement.
- If Greater Troch looks different, assess femoral rotation (visually compare greater and lesser trochanters)
- Modify preop greater troch point placement, if necessary, by selecting 'PreOp: Mark Greater Trochanter' from the breadcrumbs at bottom of screen (No. 1).

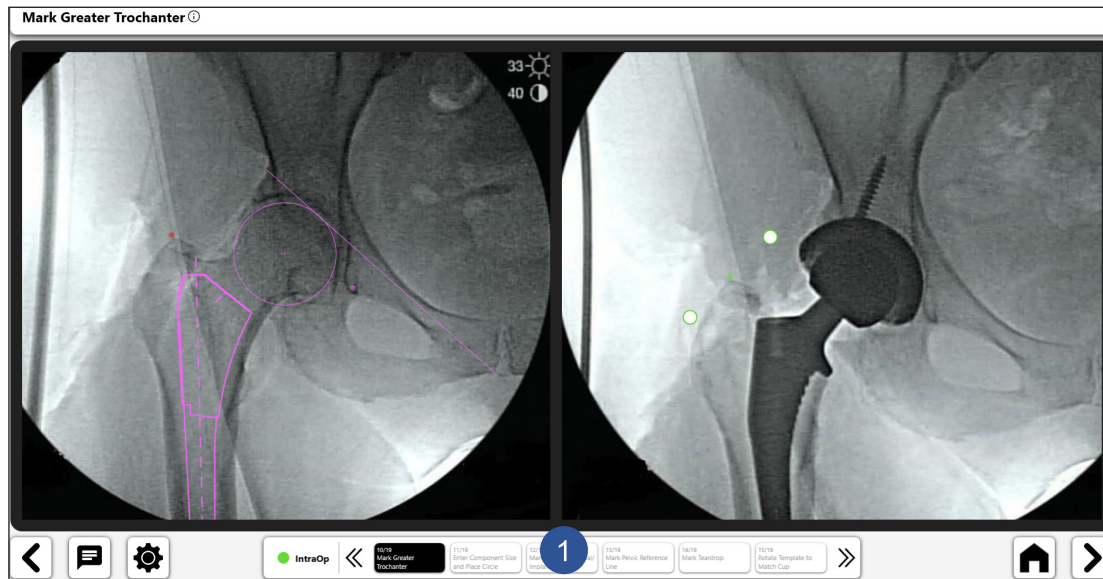
Figure 61: Mark Greater Trochanter (Op Side)

## 12. Enter Component Size and Place Circle Around Acetabulum

- Trace the Perimeter of Acetabular Component Using the Circle

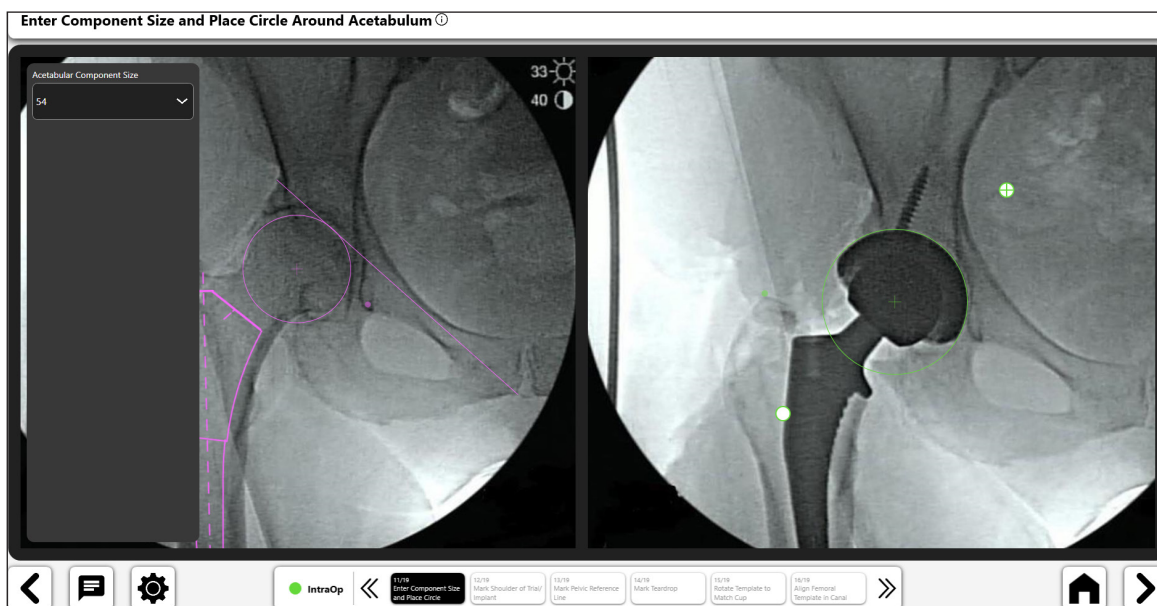
Figure 62: Enter Component Size and Circle Acetabulum (Op Side)

### 13. Mark Shoulder of Trial/Implant

- Place point on lateral shoulder of femoral component.
- Point should be placed on Superior Lateral edge of femoral broach or implant.
- Center of point should be on the edge of component shoulder

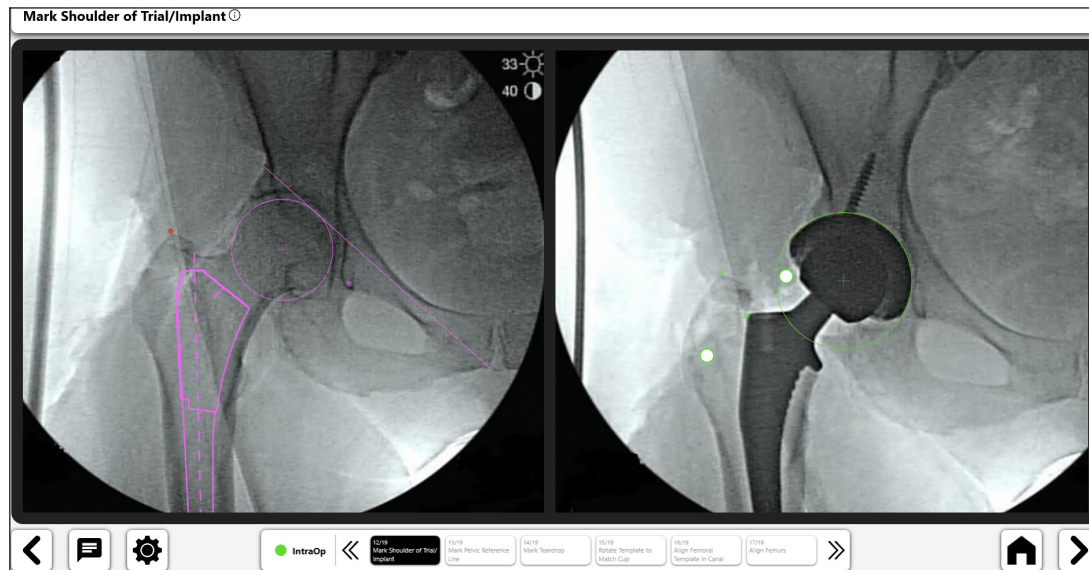
Figure 63: Mark Implant/Trial Shoulder (Op Side)

### 14. Mark Pelvic Reference Line

- Create a Pelvic Reference Line reproducing the identical line from the PreOp X-ray.
- Difference in the Line End Points can adversely affect Leg Length and Offset data.
- Identifiable Points must appear in both the PreOp and IntraOp X-rays.
- To edit the original Preop reference line, select 'PreOp: Mark Pelvic Reference Line,' from the breadcrumbs at the bottom of the screen and reposition line (No.1).

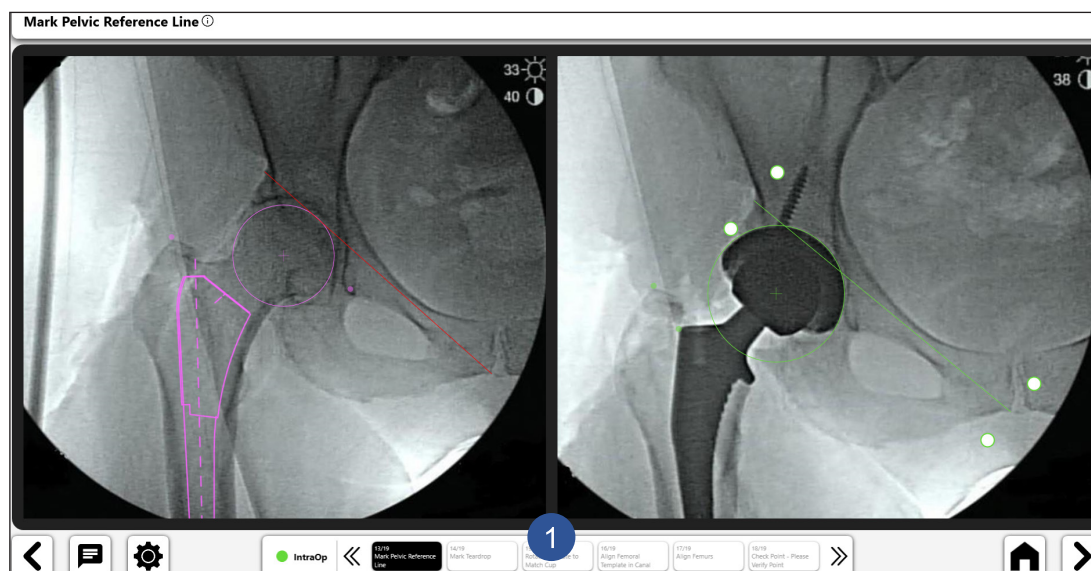
Figure 64: Mark Pelvic Reference Line (Op Side)

## 15. Mark Teardrop

- Place Point on Teardrop, matching the Preop X-ray Placement.
- Replicating teardrop point placement is important for leg length and offset calculations.
- Differences in teardrop appearance can signal a change in pelvic rotation or tilt.
- If previous point is no longer visible, select 'PreOp: Mark Tear Drop' from the breadcrumbs at the bottom of the screen to place a new teardrop point that appears on both of the PreOp and IntraOp X-rays.

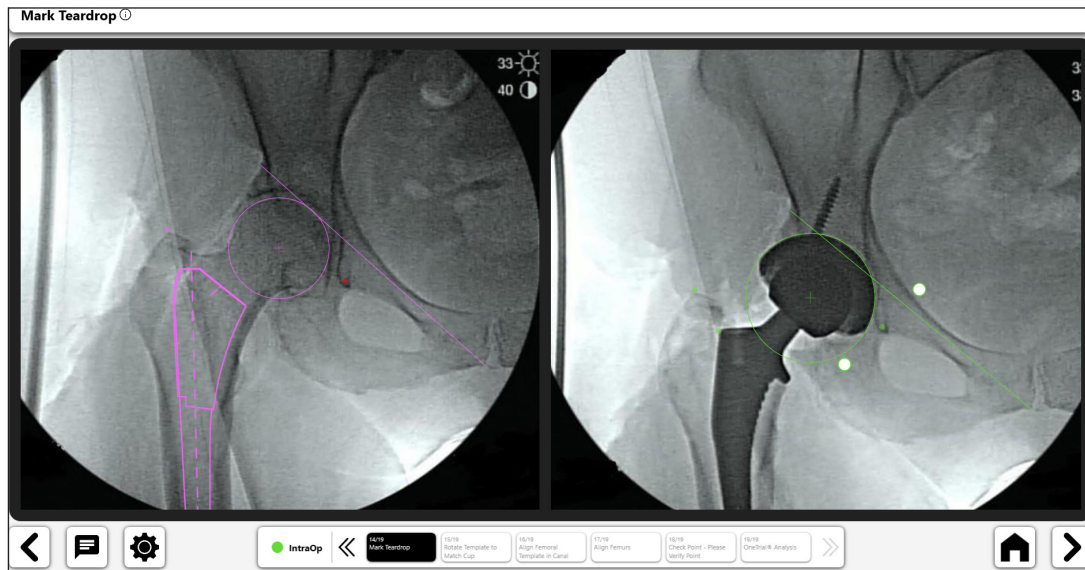
Figure 65: Mark Teardrop (Op Side)

## 16. Rotate Template to Match Cup

- Rotate the Acetabular Template to Match Component Position.
- Lateral edge of template should match lateral edge of component.

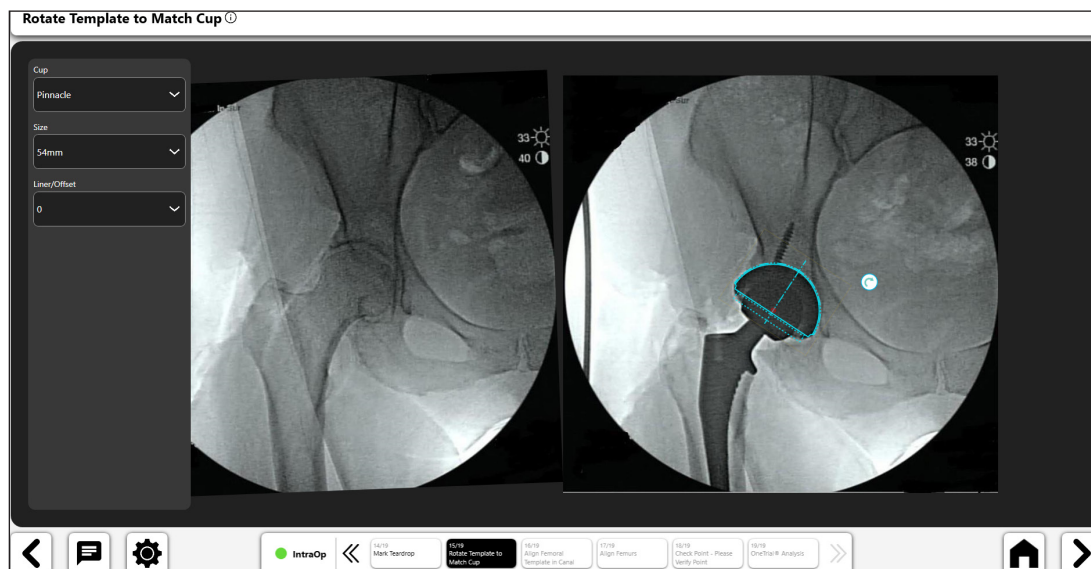
Figure 66: Rotate Template to Match Cup (Op Side)

## 17. Align Femoral Template in Canal

- Rotate Template so that it is Positioned Down the Femoral Canal and recreates the femoral components position.
- Template will ideally match the femoral component, but if not, center the template down the canal. If femoral template not aligning, check the femoral rotation (greater and lesser trochanters)
- Confirm neck and head trial are completely seated when analyzing, so as not to add error.

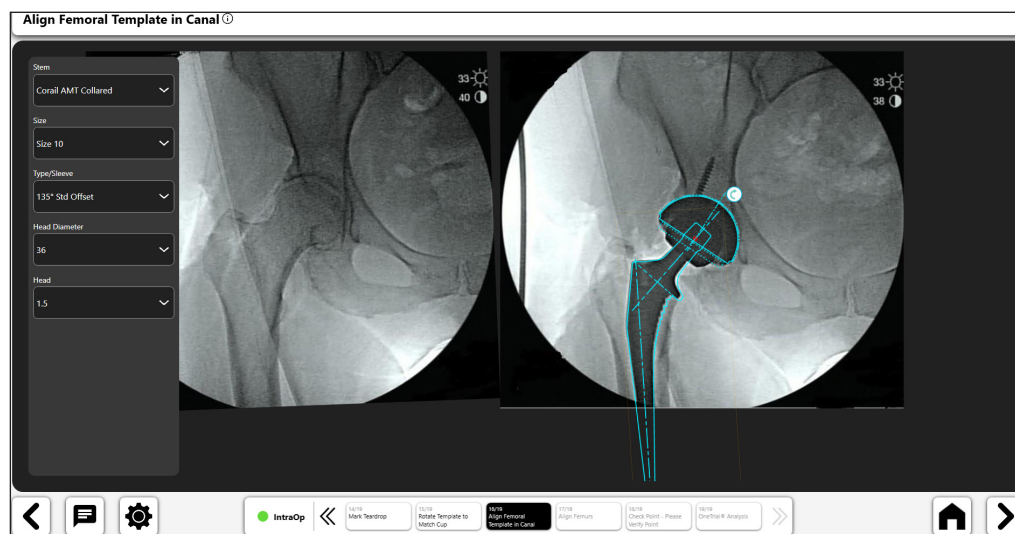
Figure 67: Align Femoral Template in Canal (Op Side)

## 18. Align Femurs

- Rotate Femurs to match Femoral Axis Alignment.
- Use the femoral shaft to align femurs on the x-rays
- If only metaphyseal region of femur visible, align along medial calcar.
- Use the transparency slider to confirm femurs are aligned (No.1)

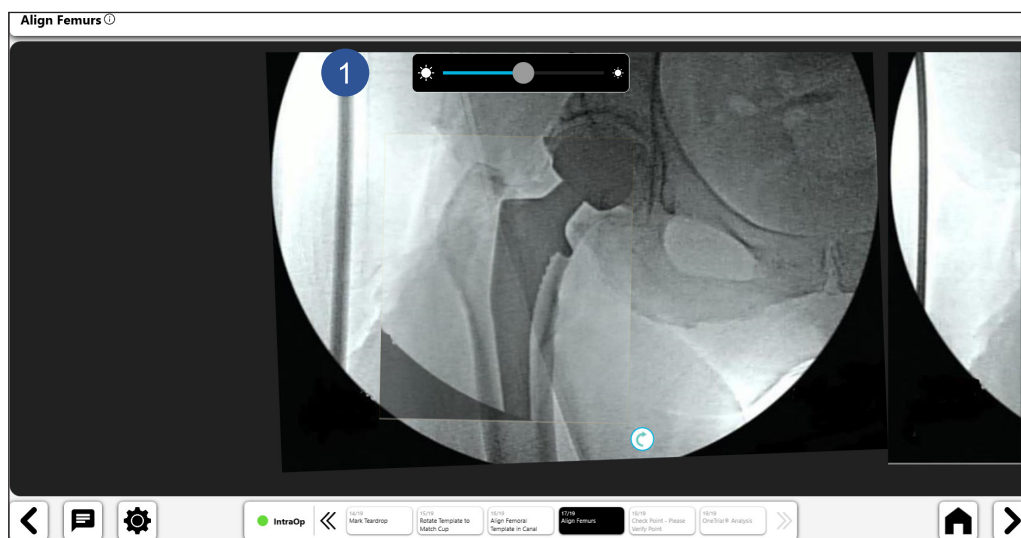
Figure 68: Align Femur

## 19. Surgeon Check Point

- Surgeon to Confirm point placement is consistent between the two x-rays.
- Changes in point of line positions are made through the breadcrumbs at bottom of screen.
- If Tear Drop Error Message appears, verify Pelvic Reference Line end points and teardrop points are the same. Tear Drop Error may signal that the X-rays are not similar.

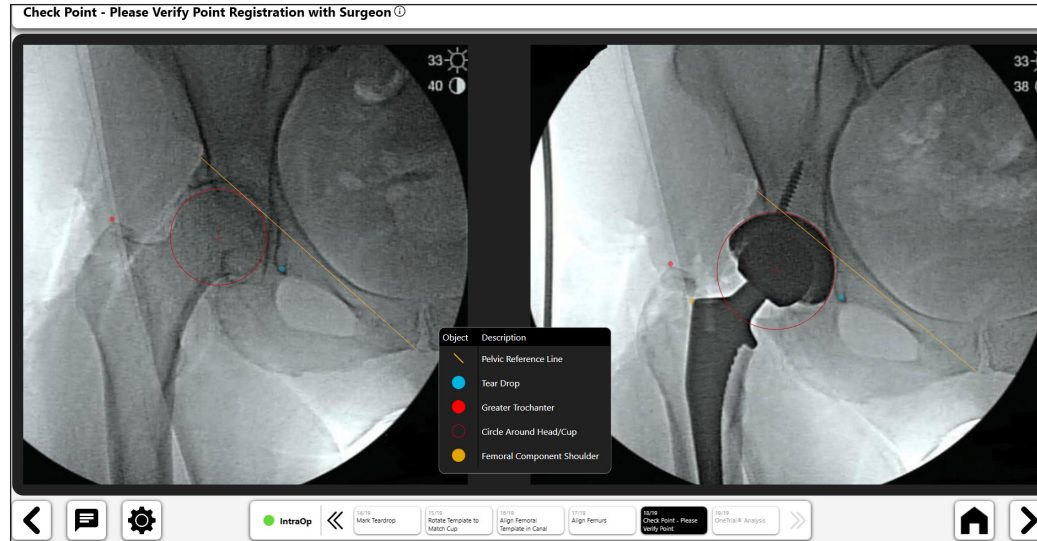
Figure 69: Verify Point Registration

## 20. OneTrial™ Analysis View

- Two user views as shown in Figures 70 and 71
- Toggle between the views with the switch (No. 1)
- Current components (No. 2)
- Leg Length filter tool (No. 3)
- Total and Femoral Offset selector (No. 4)

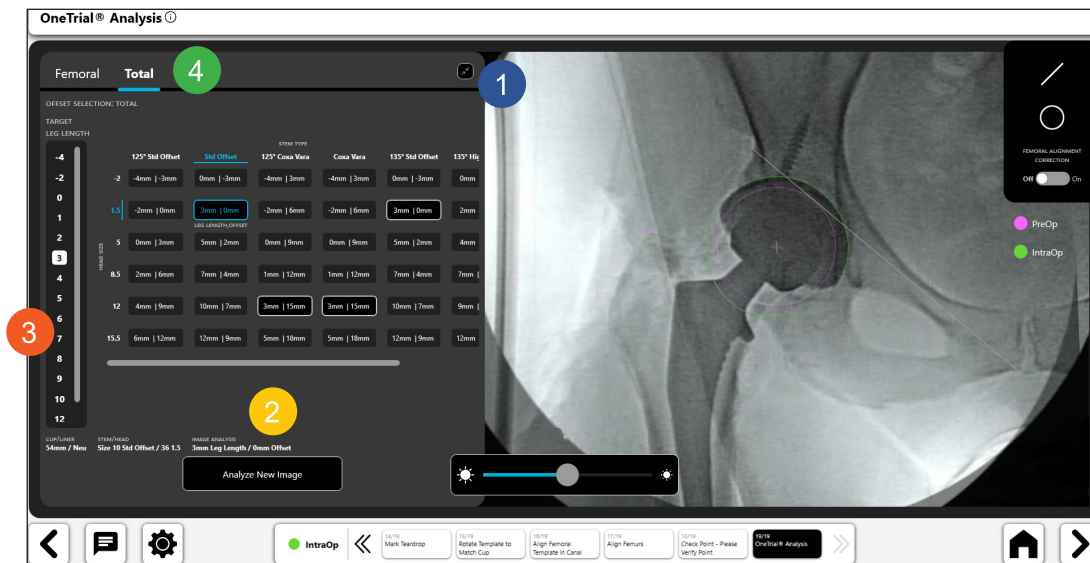
Figure 70: OneTrial™ Analysis – Expanded Chart

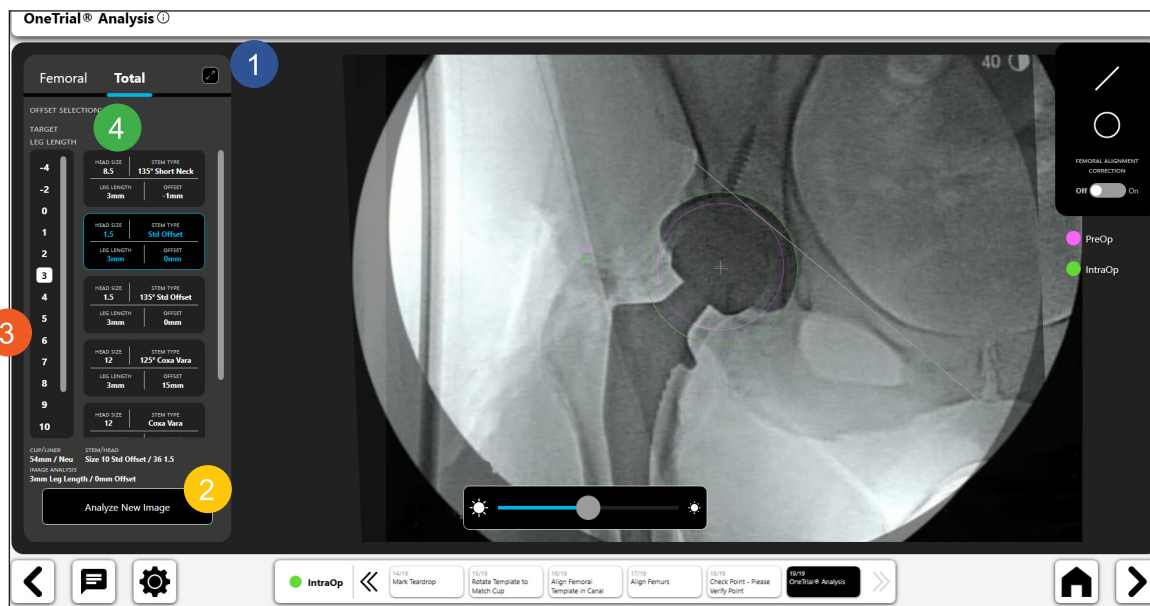
Figure 71: OneTrial™ Analysis - Condensed Chart

### Digital Line Tool:

- Select the digital line icon to create a digital line (No. 1).
- You can manipulate the length and position of the line using the navigation handles.
- The line length calculation will automatically update as you change the line length.
- **TIP: You can create multiple lines. Each line will have a different color.**

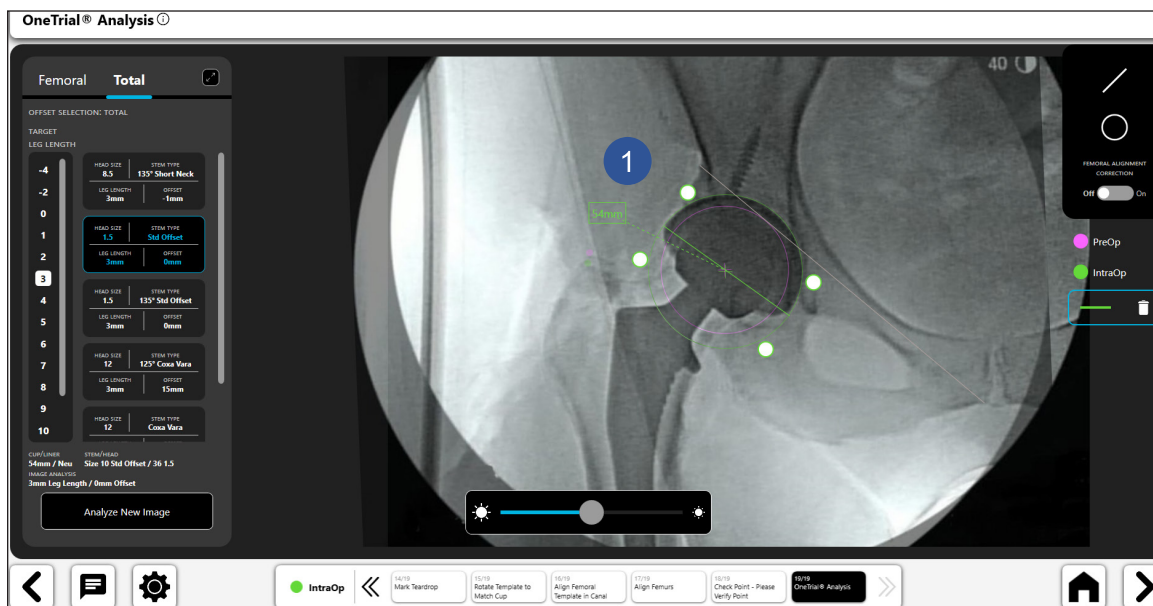
Figure 72: OneTrial™ Analysis - Digital Line Tool

### Digital Circle Tool:

- Select the digital circle icon to create a digital circle (No. 1).
- You can manipulate the size and position of the line using the navigation handles.
- The displayed circle size is the diameter of the circle.
- You can create multiple circles.

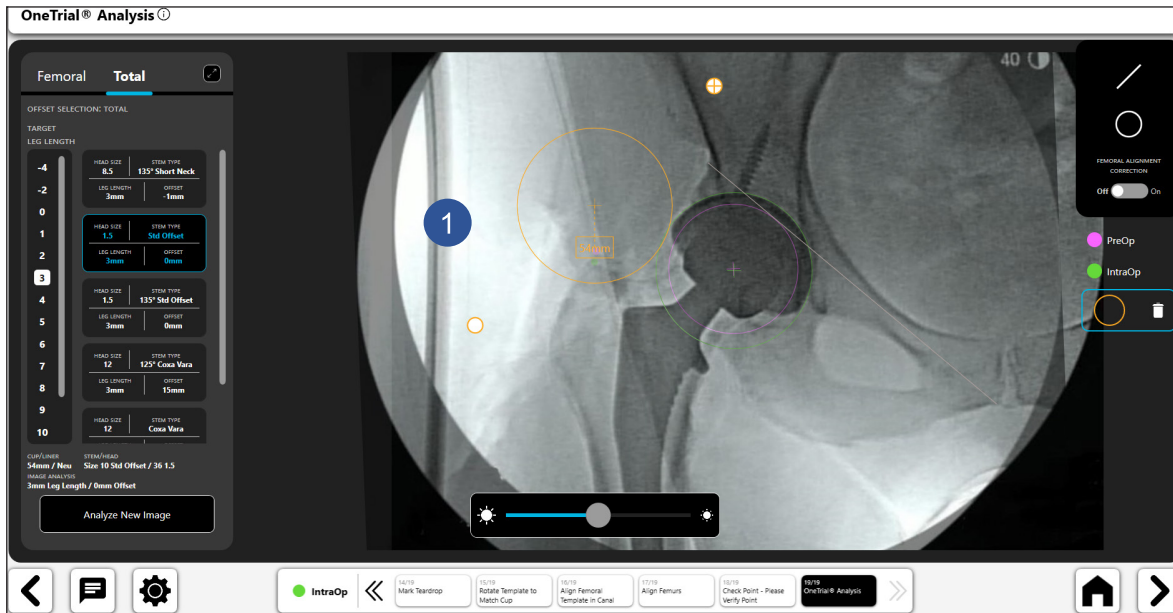
Figure 73: OneTrial™ Analysis - Digital Circle Tool

### Femoral Alignment Correction

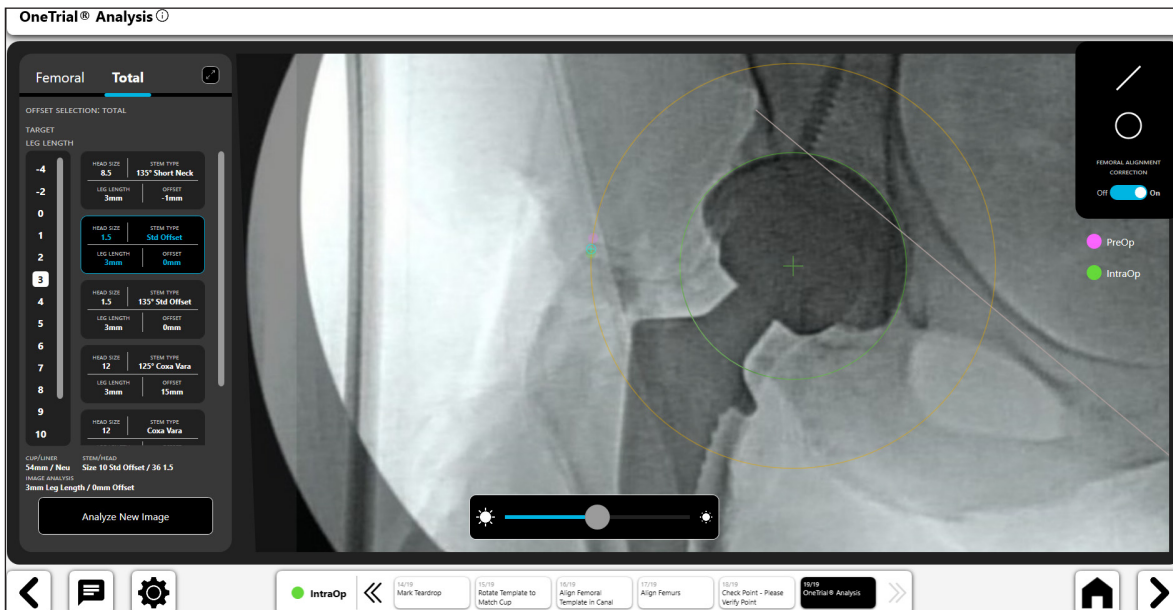
Figure 74: OneTrial™ Analysis - Femoral Alignment Correction

### Transparency Slider

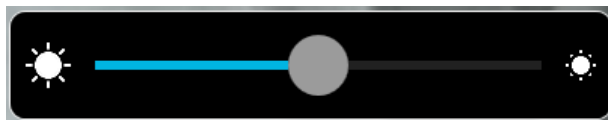
Figure 75: Transparency Slider Tool

### 21. Analyze New Image

After analyzing the trial results and the surgeon has made their decision on which implant size, offset and head to use the “Analyze New Image” button will start the intraoperative process over again. Follow all the steps to produce OneTrial™ results for the final implant.

Figure 76: Analyze New Image Button

## Contralateral Overlay Workflow

### Creating a Contralateral Overlay Workflow

#### 1. Select Contralateral Overlay

On the Case Overview Screen, select “Contralateral Overlay” and choose “Start”.

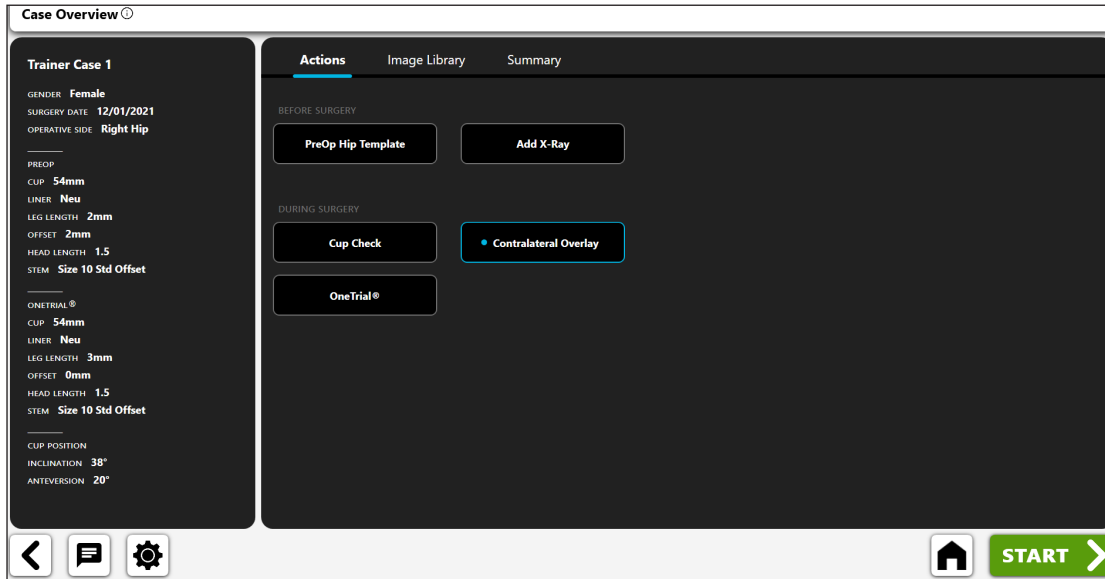
Figure 77: Case Overview Screen - Contralateral Overlay

#### 2. Import Non-Operative X-Ray

Capture a non-operative AP Hip X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

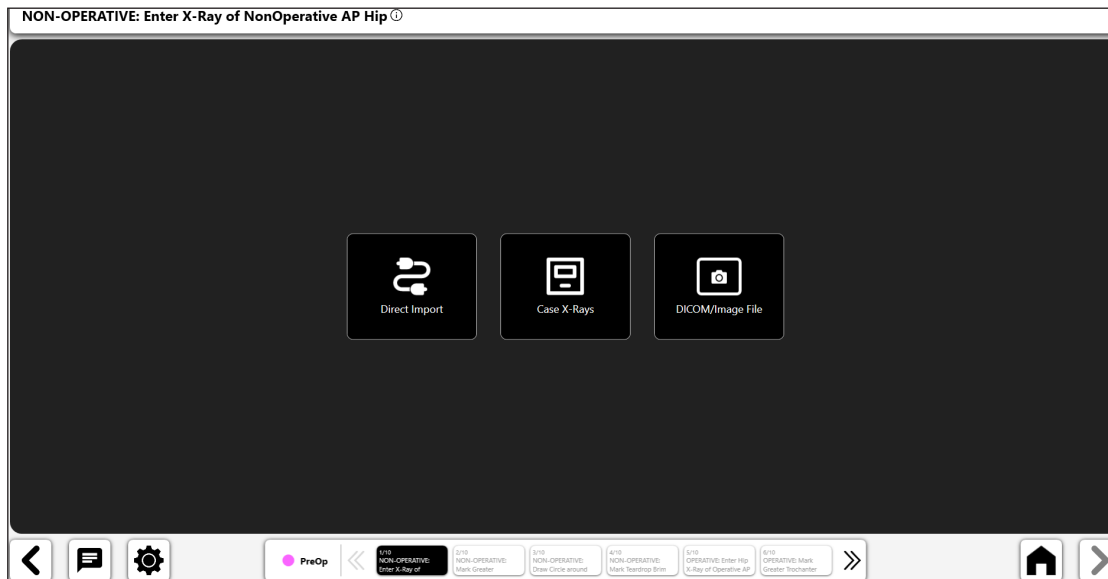
Figure 78: Import X-Ray Screen – NonOp Side

### 3. Mark Greater Trochanter

- Place point on tip of the Greater Trochanter.
- Best point is generally either a dark, well-defined edge or highest point on the trochanter.
- Point can be placed above or directly on the landmark.

Figure 79: Mark Greater Trochanter (NonOp Side)

### 4. Draw Circle around Femoral Head

- Place circle on the border of femoral head.
- This circle approximates the center of rotation and will help define femoral offset.
- In cases of head collapse approximate the current head center of rotation

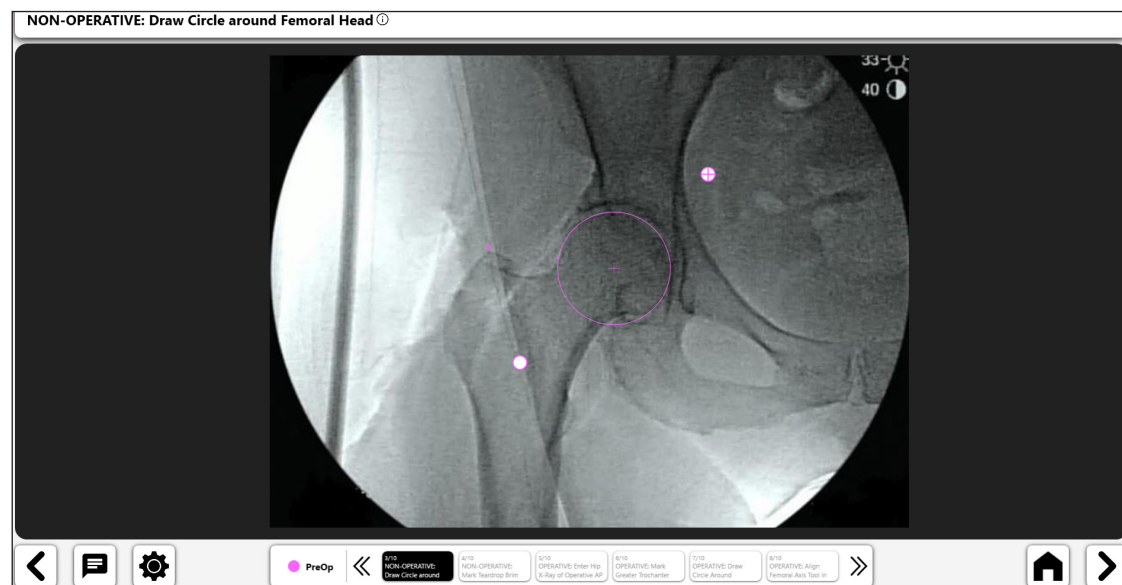
Figure 80: Draw Circle Around Femoral Head (NonOp Side)

## 5. Mark Teardrop Brim Line

- Draw Line Starting at Tear Drop and Extending Superior Along Pelvic inlet.
- Length of line is not important. No specific end point is needed for superior end of line.

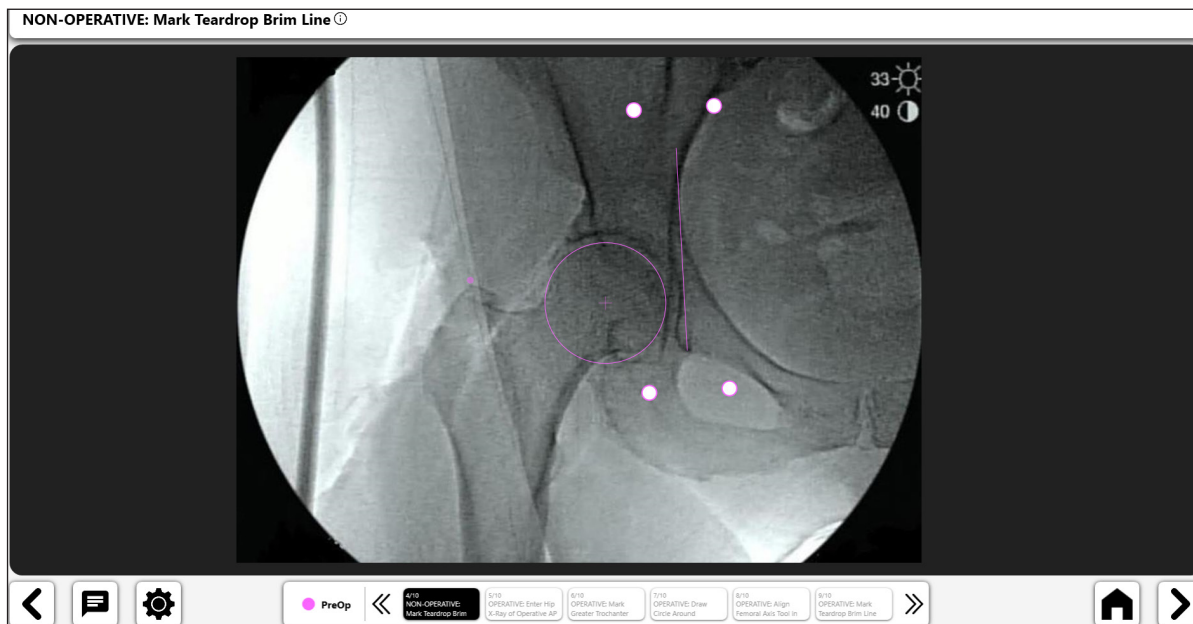
Figure 81: Mark Teardrop Brim Line (NonOp Side)

## 6. Import Operative X-Ray

Capture an operative AP Hip X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

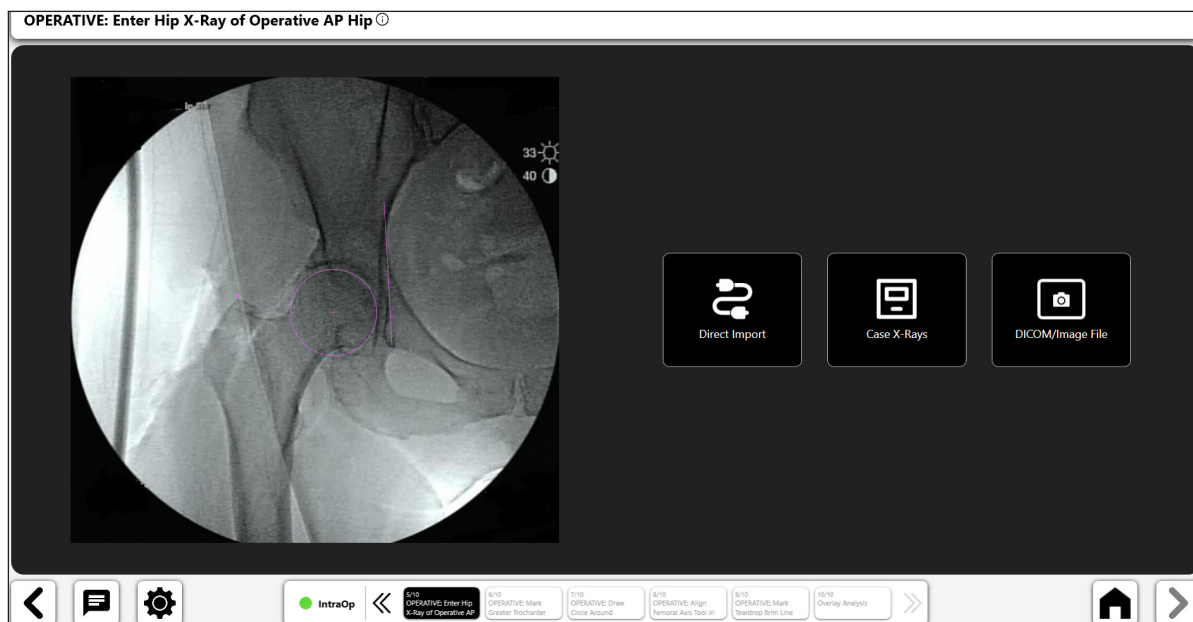
Figure 82: Import X-Ray Screen – Operative Side

## 7. Mark Greater Trochanter

- Place point on tip of the Greater Trochanter, matching the non-operative image placement.
- If Greater Troch looks different, assess femoral rotation (visually compare greater and lesser Trochanters).
- Modify preop greater troch point placement, if necessary, by selecting 'PreOp: Mark Greater Trochanter' from the breadcrumbs at bottom of screen (No. 1).

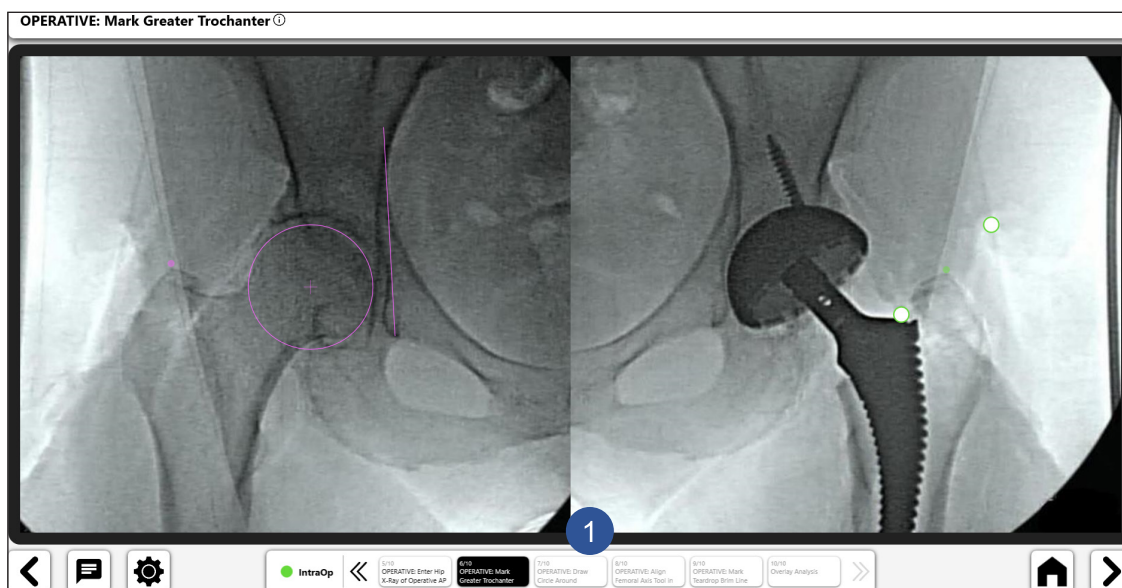
Figure 83: Mark Greater Trochanter (Op Side)

## 8. Draw Circle On Acetabular Component

- Place circle on the border of the acetabular component.
- This circle approximates the center of rotation and will help define femoral offset

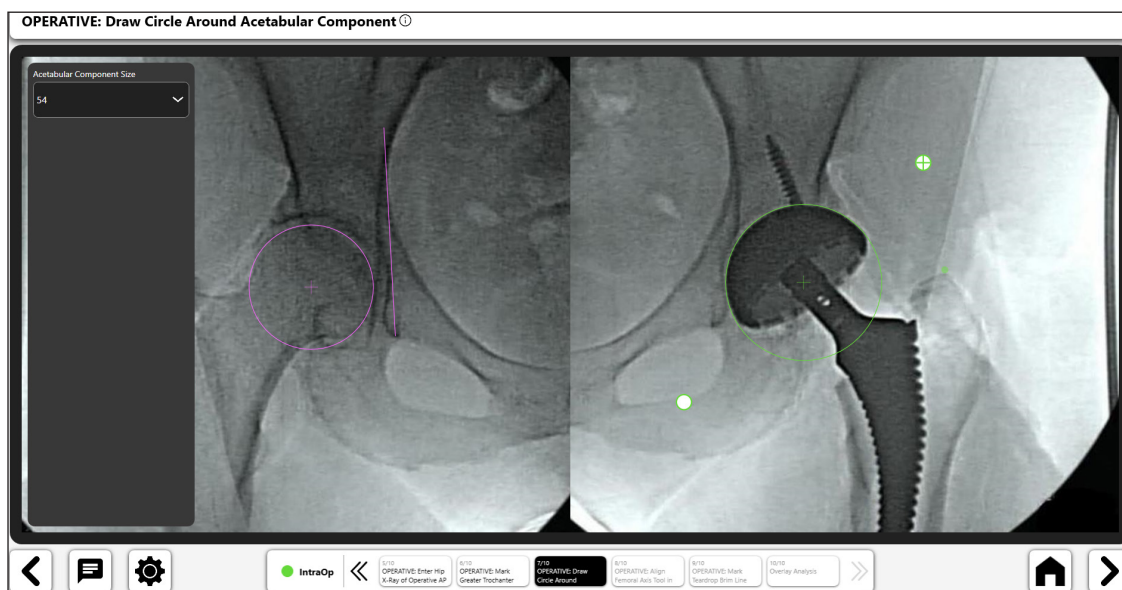
Figure 84: Draw Circle Around Femoral Head (Op Side)

## 9. Align Femoral Axis Tool in Canal

- Size the tool to fill the medullary canal and then align with the long axis of femur

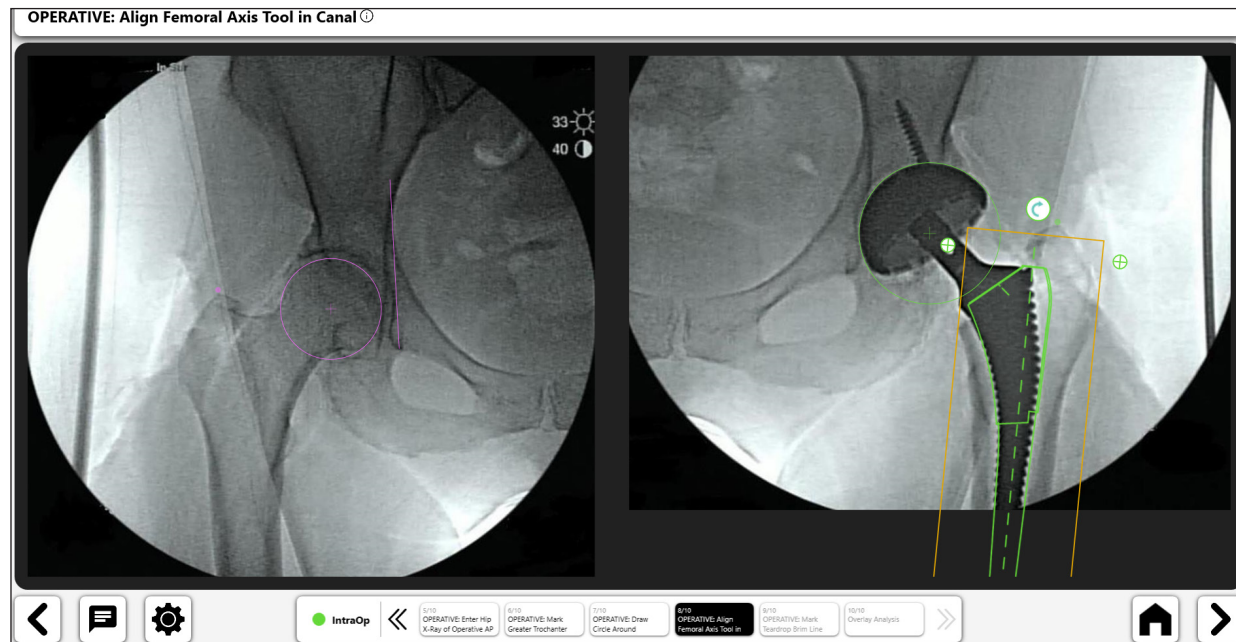
Figure 85: Align Femoral Axis Tool (Op Side)

## 10. Mark Teardrop Brim Line

- Draw Line Starting at the same Tear Drop point and Extend Superior Along Pelvic inlet to recreate the teardrop brim line from the non-operative hip.
- Length of line is not important. No specific end point is needed for superior end of line.

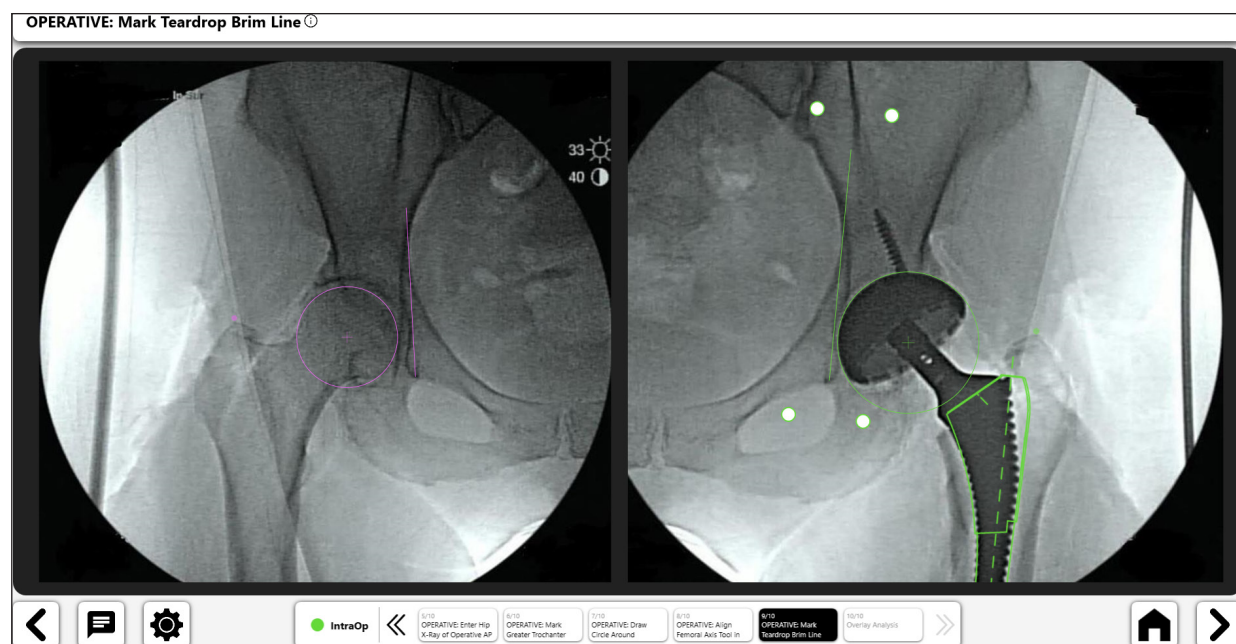
Figure 86: Mark Teardrop Brim Line (Op Side)

## 11. Overlay Analysis

Contralateral Overlay reports on the leg length between the operative and non-operative X-rays. To properly visualize the operative image needs to be rotated so that the two femurs are parallel (Figure 88).

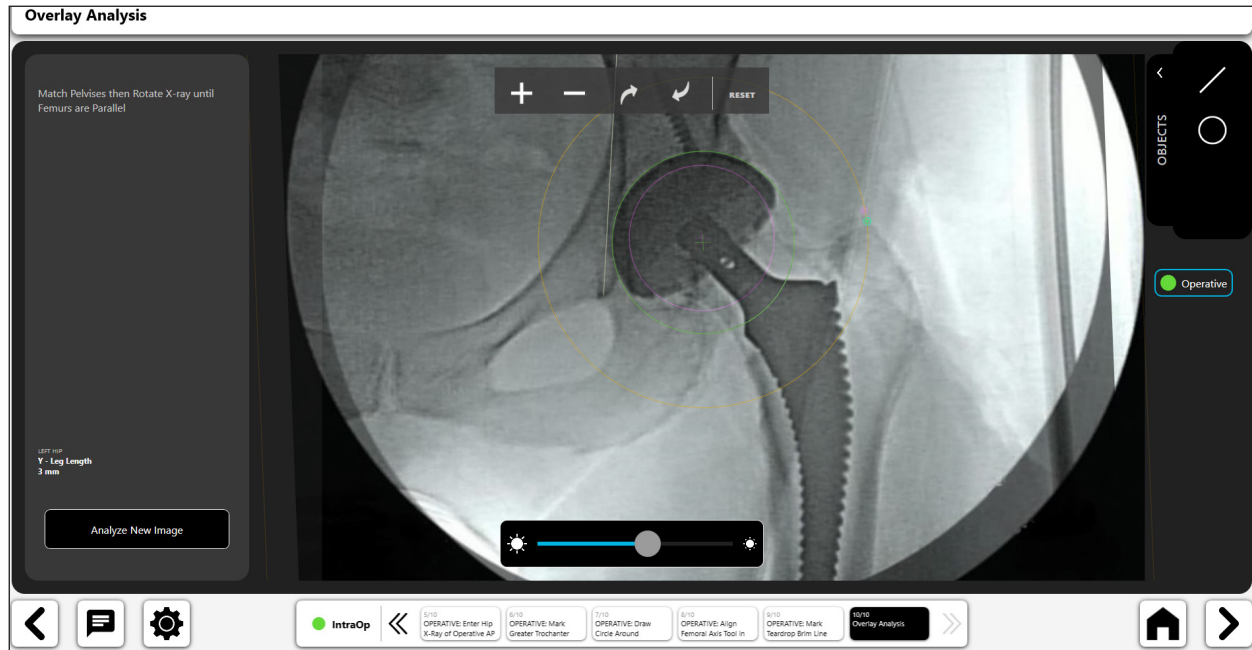
Figure 87: Overlay Analysis

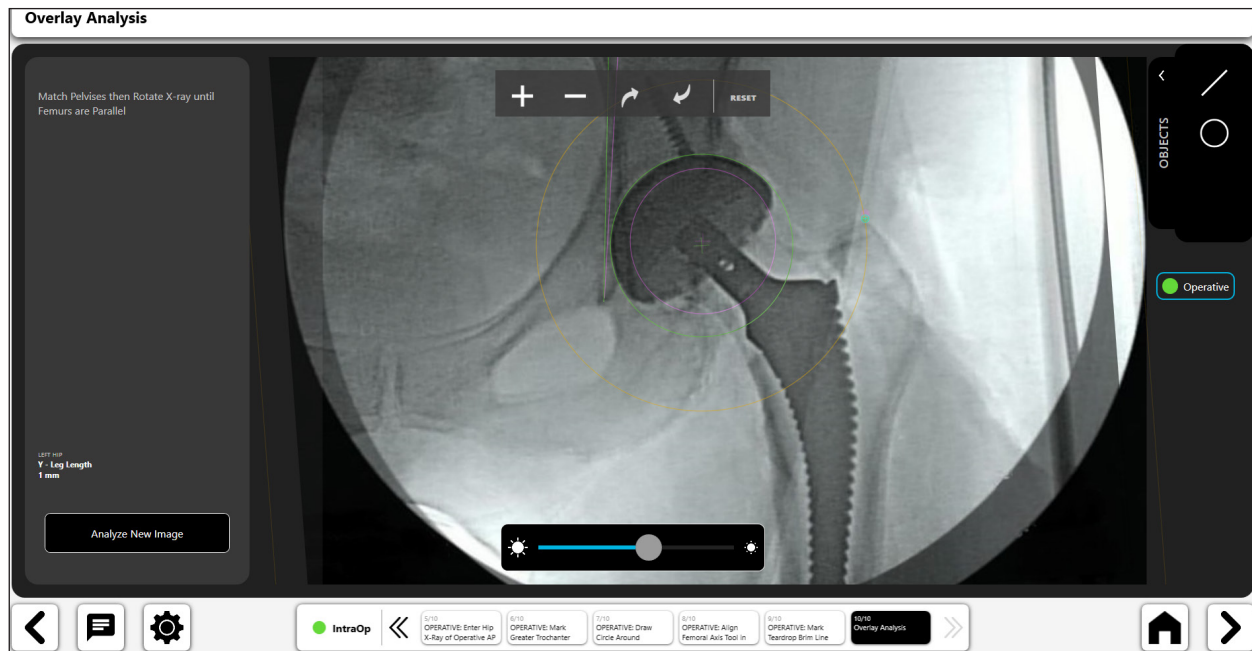
Figure 88: Overlay Analysis – After Rotation of Operative Image

### Digital Line Tool:

- Select the digital line icon to create a digital line (No. 1).
- You can manipulate the line using the navigation handles.
- The line length calculation will automatically update as you change the line length.

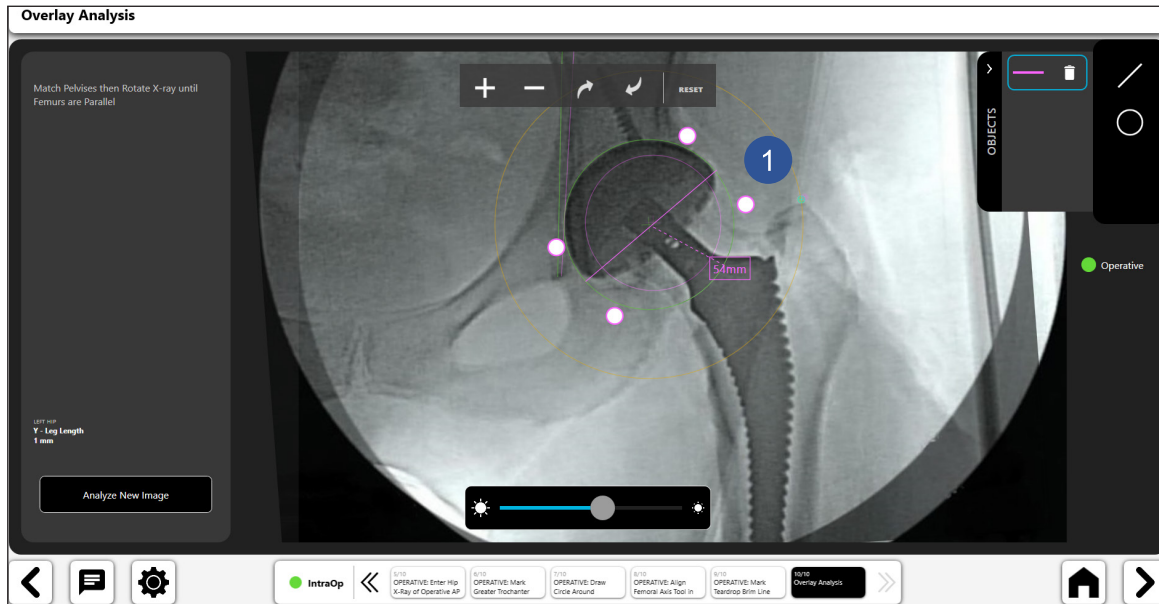
Figure 89: Overlay Analysis - Digital Line Tool

### Digital Circle Tool:

- Select the digital circle icon to create a digital circle (No. 1).
- You can manipulate the size and position of the line using the navigation handles.
- The displayed circle size is the diameter of the circle.

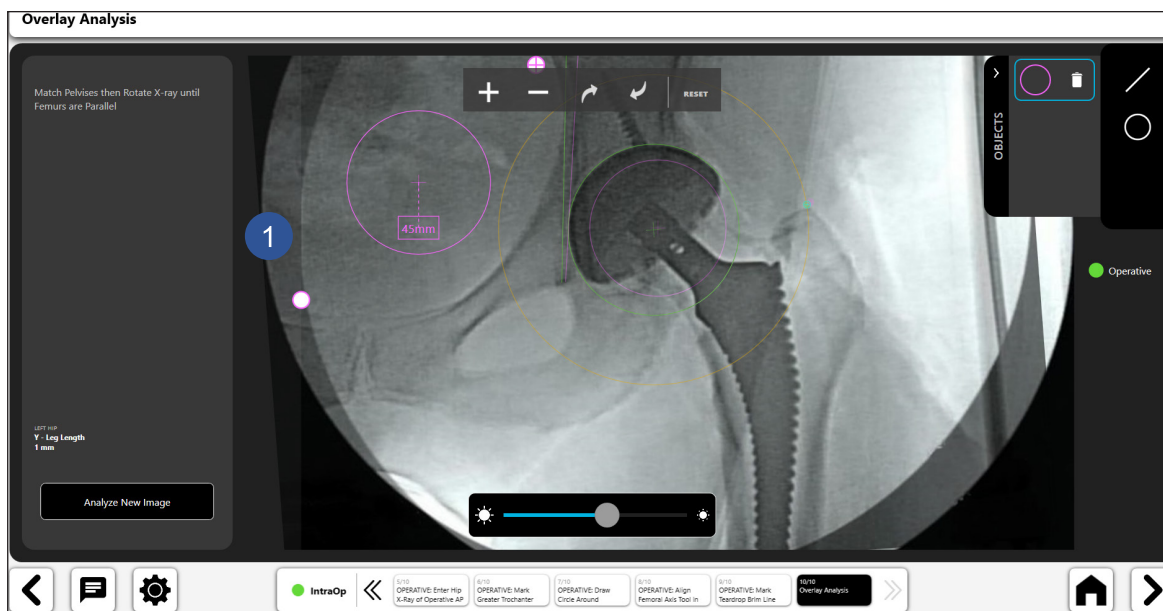
Figure 90: Overlay Analysis - Digital Circle Tool

### Zoom and Rotation

Figure 91: Zoom and Rotate Toolbar

### Analyze New Image

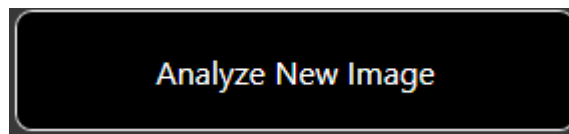
Figure 92: Analyze New Image Button

### Transparency

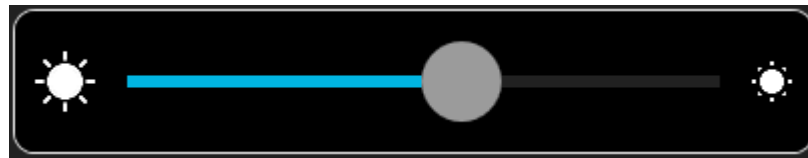
Figure 93: Transparency Slider Tool

## Cup Check

### Creating a Cup Check Workflow

#### 1. Select Cup Check

On the Workflows Screen, select the “START” button and choose “Cup Check” from the available options.

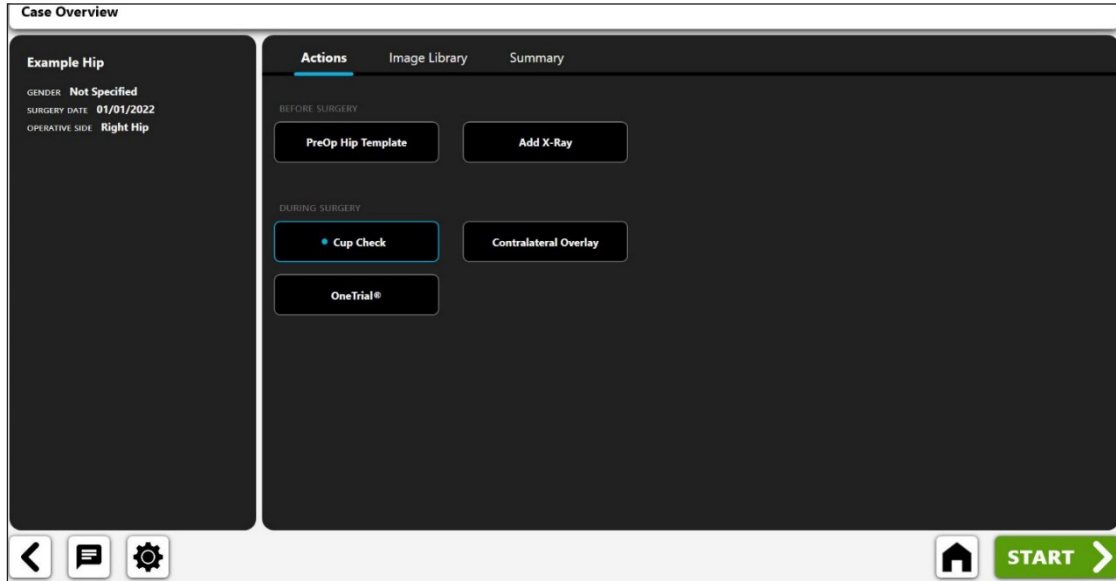
Figure 94: Case Overview Screen - Cup Check Workflow

#### 2. Import AP Pelvis X-Ray

Capture an AP Pelvis X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

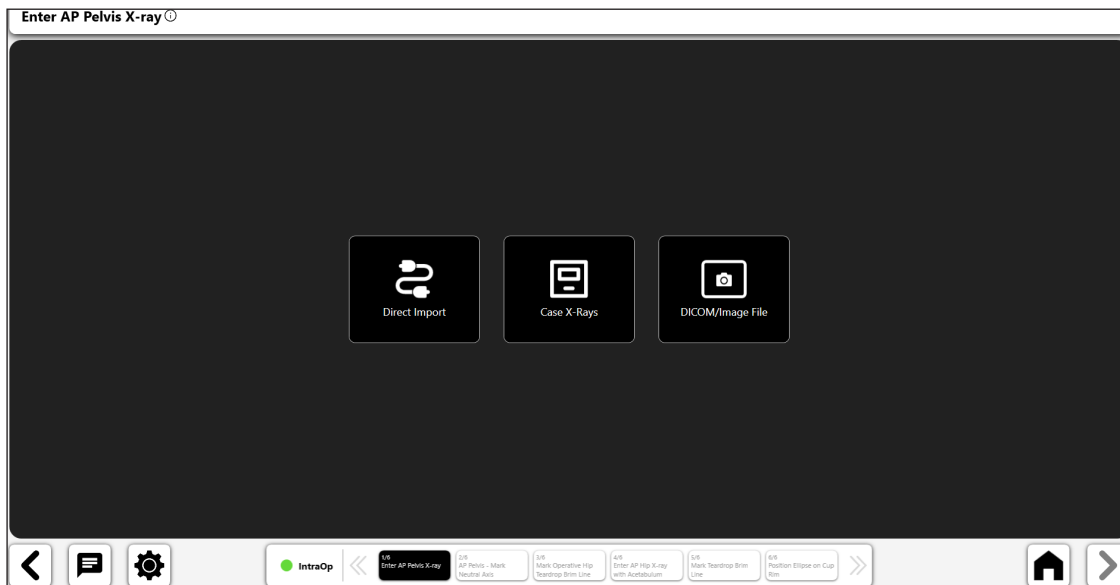
Figure 95: Import X-Ray Screen – AP Pelvis

### 3. Mark the Trans-teardrop or Neutral Axis

- Draw a line between the most inferior edges of both teardrop points.

Line start and end points at the inferior midline of each teardrop are recommended.

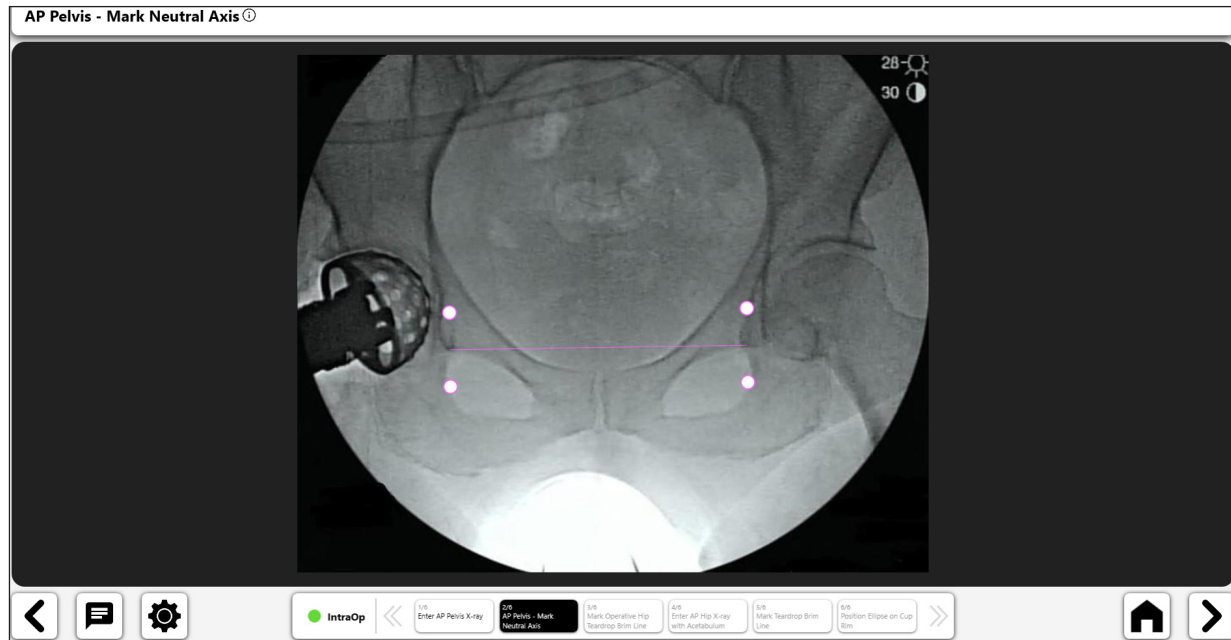
Figure 96: Mark Neutral Axis

### 4. Mark Operative Hip Teardrop Brim Line

- Draw line on operative side from tear drop along the pelvic inlet.
- Length of line is not important. No specific end point is needed for superior end of line

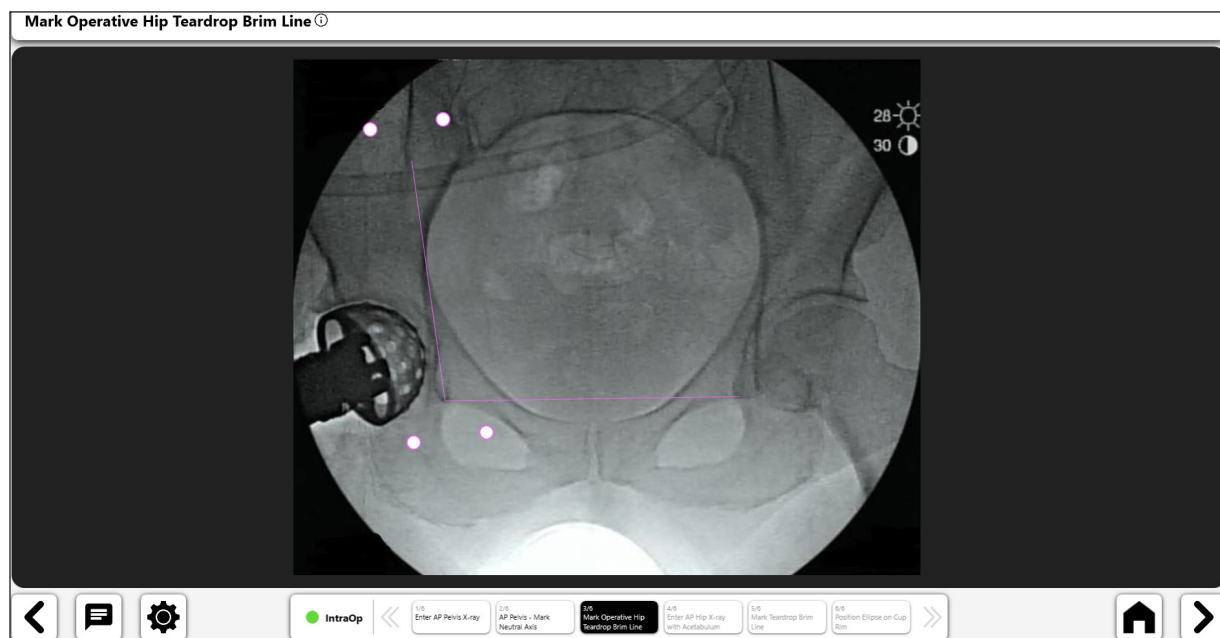
Figure 97: Mark Teardrop Brim Line

## 5. Import Operative X-Ray

Using the same c-arm position used for the AP Pelvis x-ray, capture an X-ray with the acetabulum centered on the screen from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

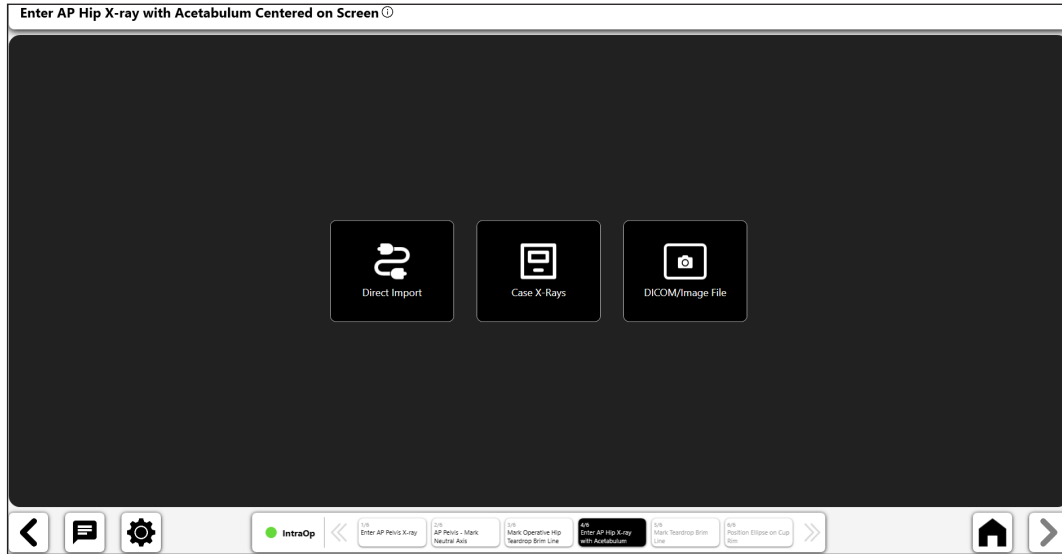
Figure 98: Import X-Ray Screen - IntraOp

## 6. Mark Teardrop Brim Line

- Start with same tear drop point and extend along pelvic inlet to recreate the teardrop brim line from the AP Pelvis.
- Line does not have to same length as the previous teardrop brim line.

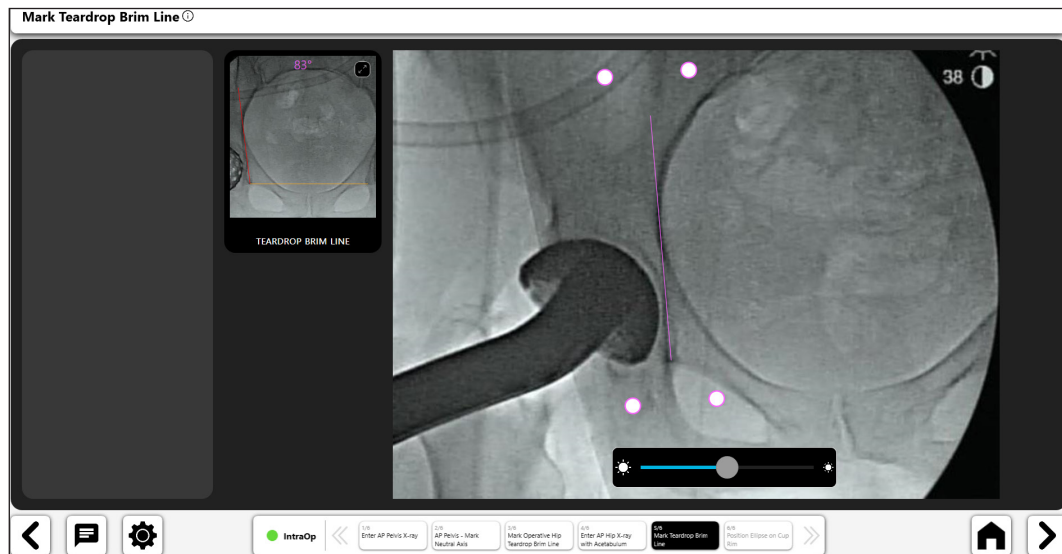
Figure 99: Mark Teardrop Brim Line

## 7. Cup Check Analysis

- Position the ellipse, resizing and moving as required, to place it over the cup rim
- Image brightness may be adjusted for a better view (No. 1)
- Ellipse angles can display either Anatomic or Radiographic (No. 2)
- Adjusting the Inclination will rotate the ellipse (No. 3)
- Adjusting the Anteversion will expand or contract the ellipse opening (No. 4)
- The AP Pelvis image is available for comparison (No. 5)

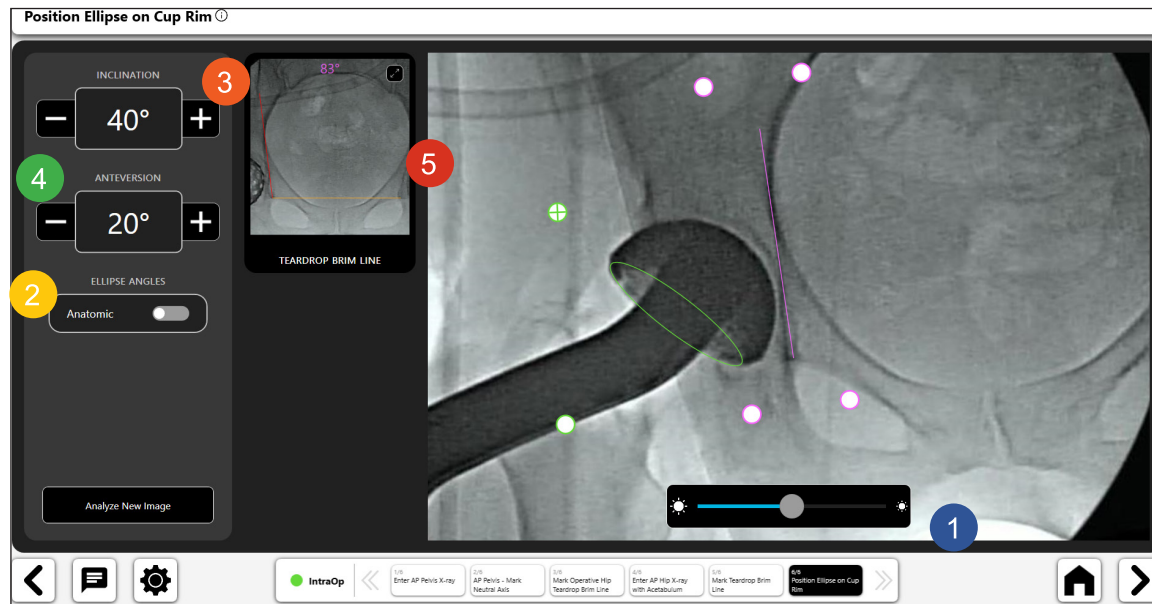
Figure 100: Position Ellipse on Cup Rim

## Preoperative Knee Workflow

### Creating a Preoperative Knee Workflow

#### 1. Select the PreOp Knee Template

On the Case Overview Screen, select the “PreOp Knee Template” and select “Start.”

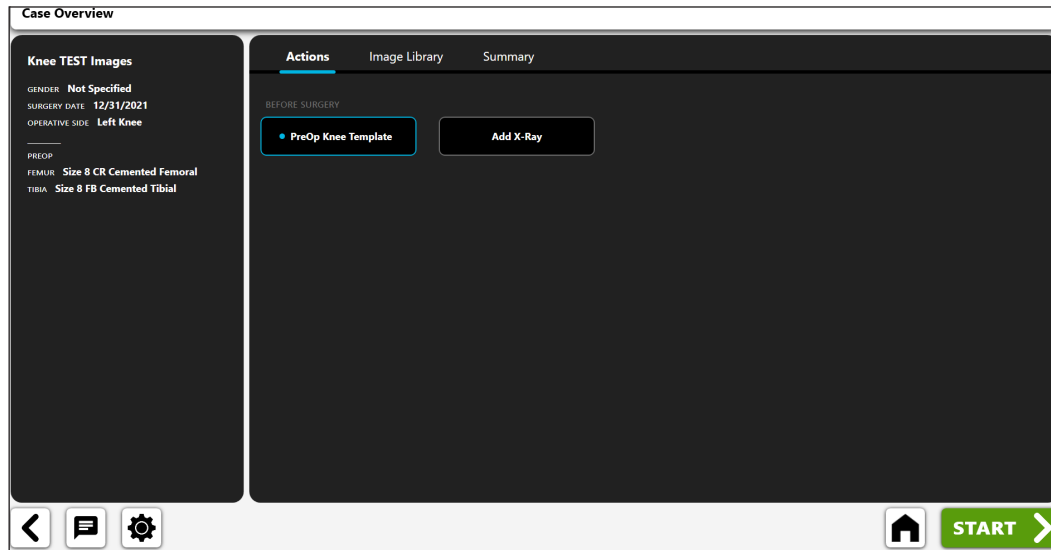
Figure 101: Case Overview Screen - PreOp Knee Template

#### 2. Import AP X-Ray

Capture an AP X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

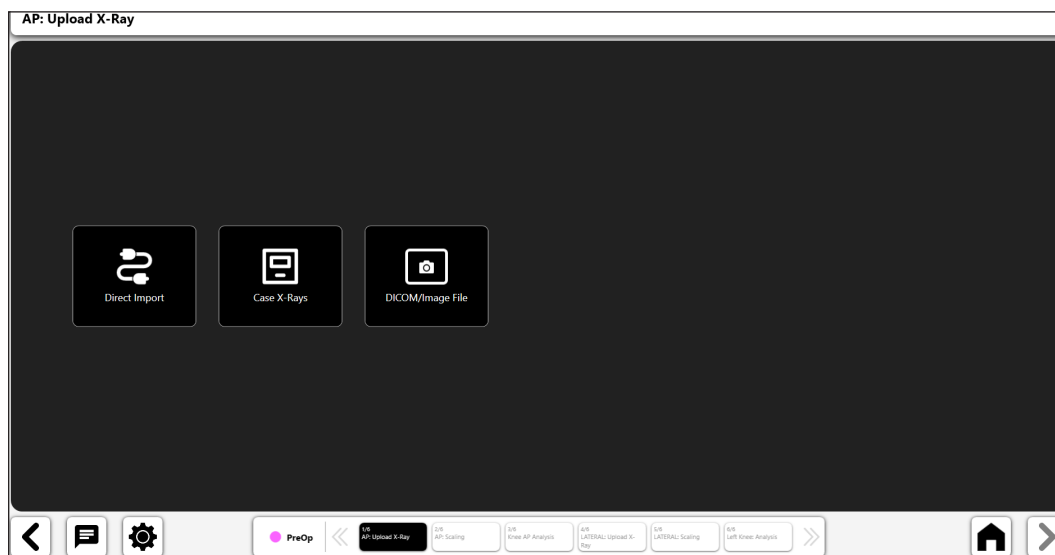
Figure 102: X-Ray Import Screen

### 3. Scale the AP X-Ray

Begin the preoperative templating process by scaling the AP image using an object of known size.

#### If you've included an AP image in your template:

- Choose whether you will use a ball marker or a straight line using the Toggle control (No. 1).
- **If you use a ball marker to calibrate**, a digital circle will appear in the screen (No. 2). Image recognition will try to auto-identify a ball marker on the image and position the circle around it.
- **If you use a ruler to calibrate**, a digital line will appear (instead of the circle) which you can manipulate to measure an object of known size.
- Modify the position and size of the circle using the control handles so that it matches the object used to calibrate the image.
- Enter the known size of the object in the data entry box in the top left of the screen (No. 3). This is a diameter measurement if using a ball marker, and the actual line size if using 'ruler' calibration.
- **TIP: The standard size of a ball marker is 25.4 mm, but you should verify this.**
- Press '>' arrow at bottom right area of screen.

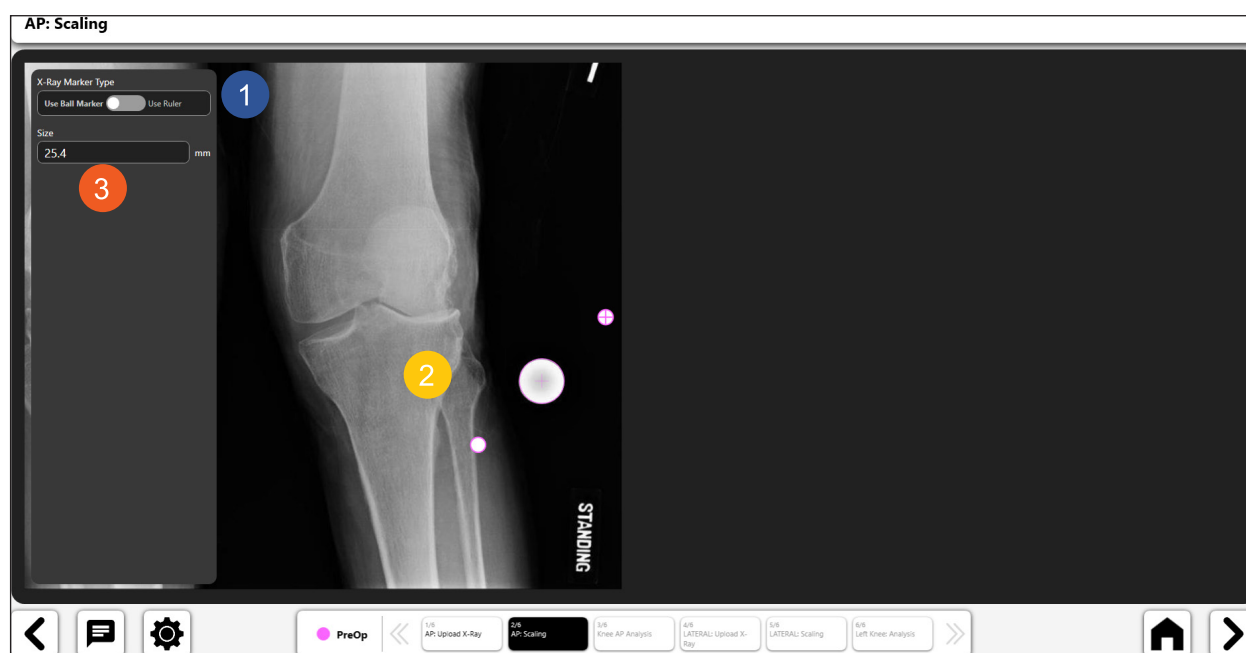
Figure 103: Scale AP X-Ray

#### 4. Add Optional Lateral X-Ray

You may choose to add a lateral X-ray. If you choose not to add a lateral X-ray, then you will skip to step 7 “Create the Preoperative Knee Template”.

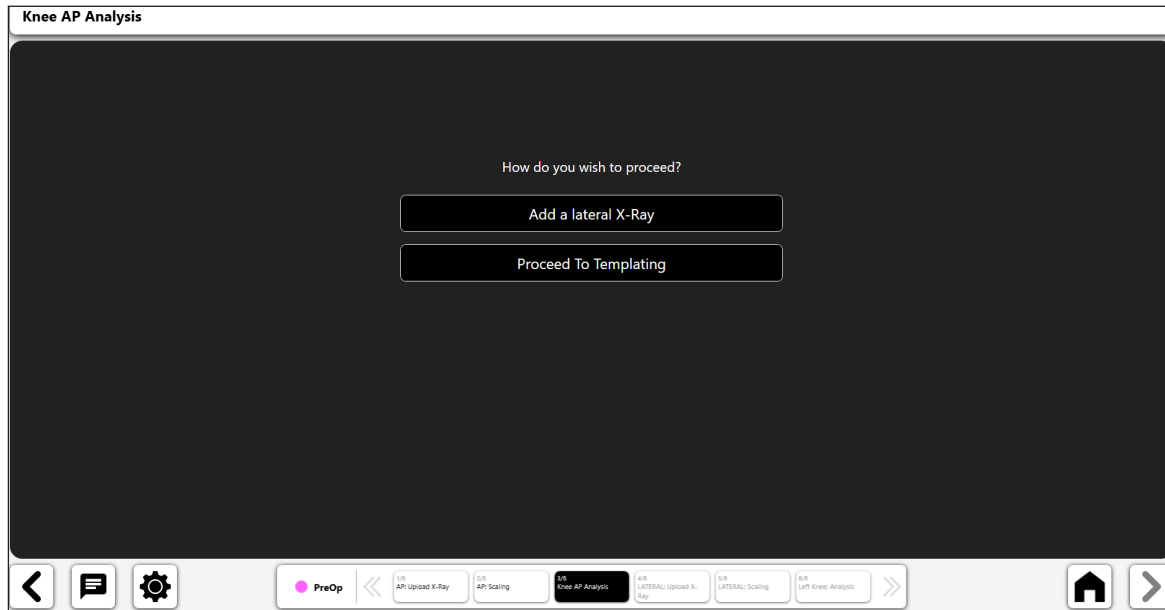
Figure 104: Knee AP Analysis

#### 5. Import Lateral X-Ray

Capture a lateral X-ray from the camera, Direct Import, an existing X-ray on the case or from a DICOM or image file.

Figure 105: Import Lateral X-Ray

## 6. Scale the Lateral X-Ray

Begin the preoperative templating process by scaling the lateral image using an object of known size.

### If you've included a lateral image in your template:

- Choose whether you will use a ball marker or a straight line using the Toggle control (No. 1).
- **If you use a ball marker to calibrate**, a digital circle will appear in the screen (No. 2). Image recognition will try to auto-identify a ball marker on the image and position the circle around it.
- **If you use a ruler to calibrate**, a digital line will appear (instead of the circle) which you can manipulate to measure an object of known size.
- Modify the position and size of the circle using the control handles so that it matches the object used to calibrate the image.
- Enter the known size of the object in the data entry box in the top left of the screen (No. 3). This is a diameter measurement if using a ball marker, and the actual line size if using 'ruler' calibration.
- **TIP: The standard size of a ball marker is 25.4 mm, but you should verify this.**
- Press '>' arrow at the bottom right of the screen.

Figure 106: Scale Lateral X-Ray

## 7. Create the Preoperative Knee Template

You can create a preoperative knee template after completing calibration. VELYS™ Knee Templating provides the following preop knee tools:

- Knee Tibial Component Template
- Knee Femoral Component Template
- Digital Line
- Digital Circle

*Tibial Component Template Tool:*

- Select the tibial component template icon.
- The template will display above the image (No. 1 and No. 2). If you use both an AP and a Lateral image, the template will display in both images, and can be moved independently. Move each template by dragging it and use the rotation handle to rotate them.
- Update size and model information for the template using the input boxes on the top of the screen (No. 3). **If you are using an AP and Lateral image, changing the template size will update the template in both images.**
- (No. 4) shows a selected 'toggle icon' for the tibial component template, showing that it is activated. You can delete the template by selecting the 'trash can' displayed within the toggle icon.
- **TIP: You can change the active object by selecting the toggle icon for the object you want to activate.**

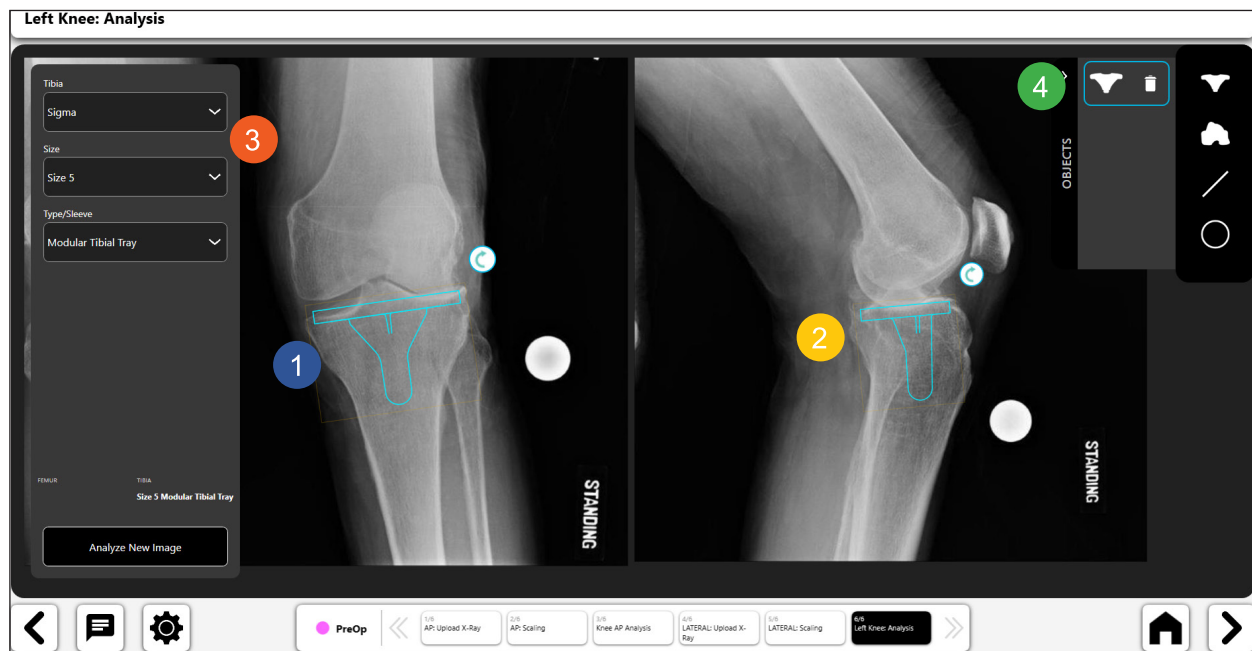
Figure 107: Tibial Component Template Tool

### Femoral Component Template Tool:

- Select the femoral component template icon.
- The template will display above the image (No. 1 and No. 2). If you use both an AP and a Lateral image, the template will display in both images, and can be moved independently. Move each template by dragging it and use the rotation handle to rotate them.
- Update size and model information for the template using the input boxes on the top of the screen (No. 3). **If you are using an AP and Lateral image, changing the template size will update the template in both images.**
- (No. 4) shows a selected 'toggle icon' for the tibial component template, showing that it is activated. You can delete the template by selecting the 'trash can' displayed within the toggle icon.
- **TIP: You can change the active object by selecting the toggle icon for the object you want to activate.**

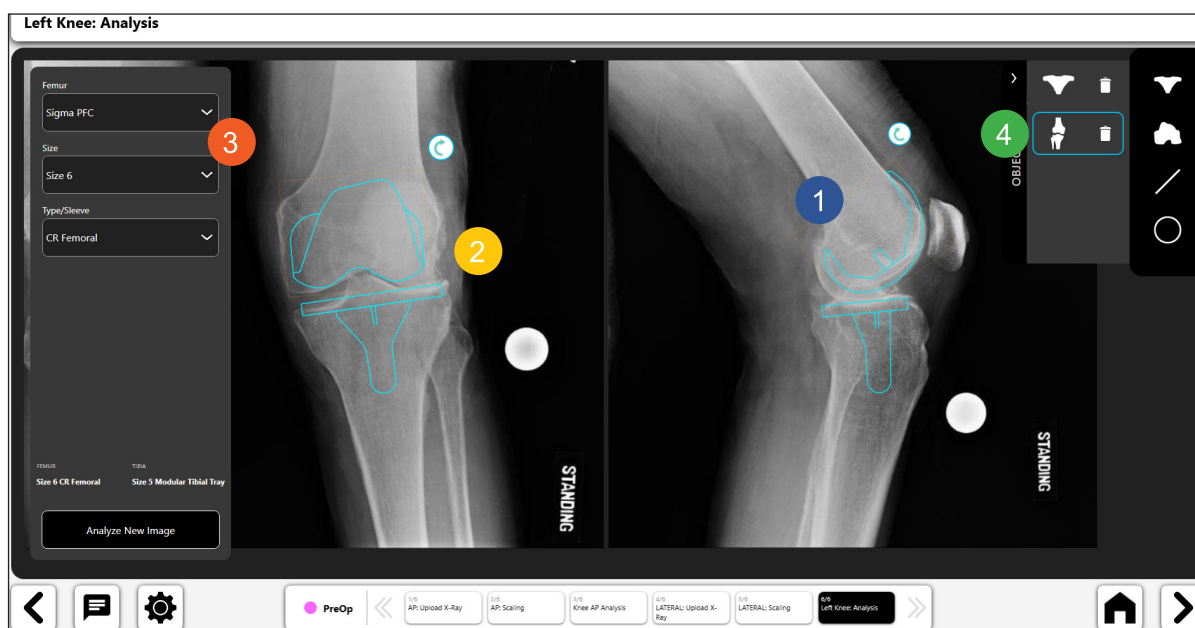
Figure 108: Femoral Component Template Tool

### Digital Line Tool:

- Select the digital line icon to create a digital line (No. 1). If you have both an AP and Lateral image, VELYS™ Hip Navigation will prompt you for which image to put the line on.
- You can manipulate the line using the navigation handles.
- The line length calculation will automatically update as you change the line length.

Figure 109: Knee Analysis - Template Side Selection Popup

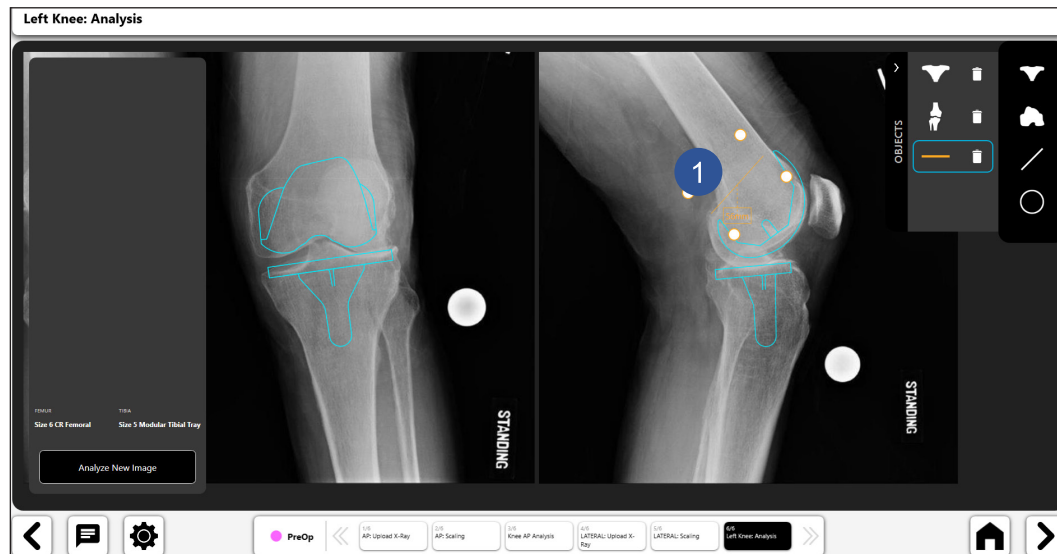
Figure 110: Knee Analysis - Digital Line Tool

### Digital Circle Tool:

- Select the digital circle icon to create a digital circle (No. 1). If you have both an AP and Lateral image, VELYST™ Hip Navigation will prompt you for which image to put the circle on.
- You can manipulate the size and position of the line using the navigation handles.
- The displayed circle size is the diameter of the circle.

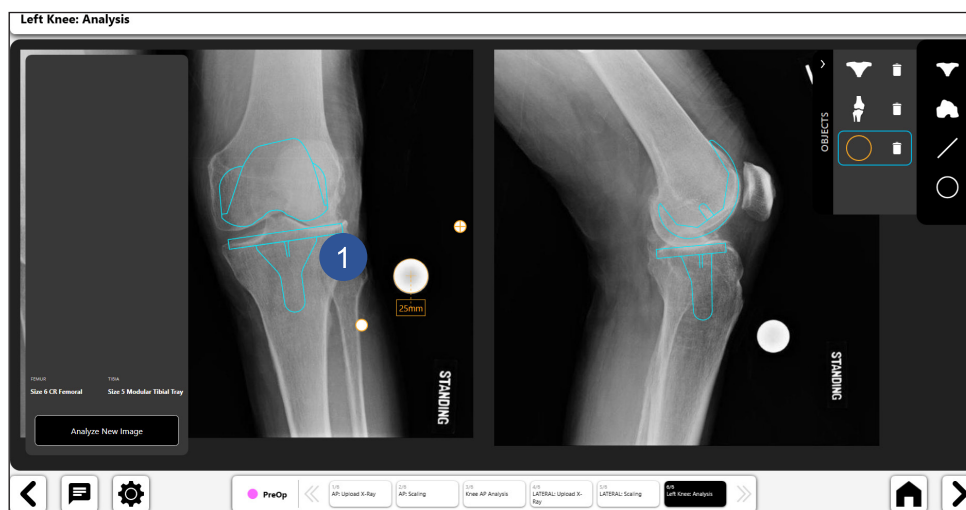
Figure 111: Knee Analysis - Digital Circle Tool

## 8. Final Preoperative Template

The Figure below shows an example of a completed template with associated information.

- Toggle icons (No. 1 and No. 2) allow you to activate controls on the page. Only one control can be active at a time. You can activate a control by selecting the toggle button for that control.
- The Heads-Up Display (“HUD”) shown in No. 3 shows the template data used in the preoperative template.
- You can save the template for later use by pressing the Save icon near the top right of the screen. The template can be opened later for viewing.

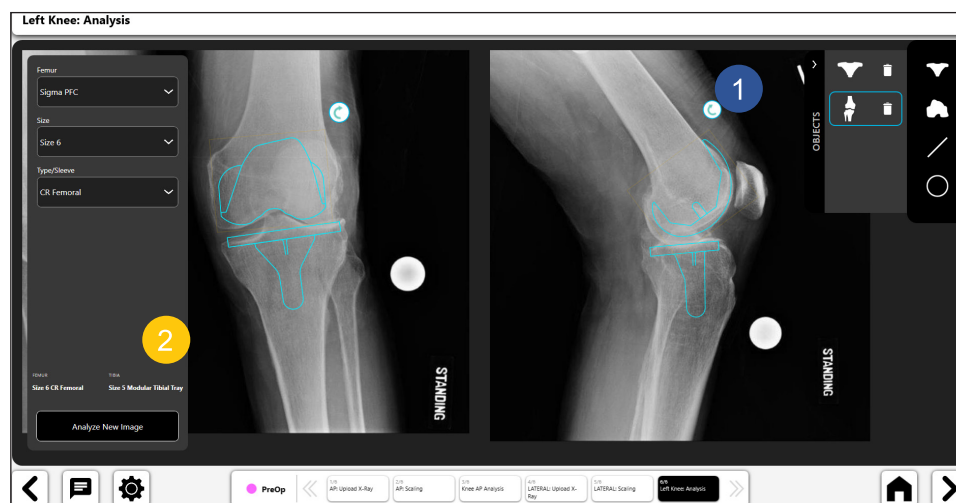
Figure 112: Knee Analysis - Final Preoperative Template

## Appendix A - Symbol Glossary

	MANUFACTURER
	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	DISTRIBUTED PRODUCT IN THE EUROPEAN UNION

IFU-0902-00-905 Rev. B

iPad is a trademark of Apple Inc.

iOS is a trademark of Cisco Technology, Inc. and is used under license by Apple, Inc.

WINDOWS is a trademark of Microsoft Corporation.

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iPad è un marchio di Apple Inc.
iOS è un marchio di Cisco Technology, Inc. ed è utilizzato su licenza di Apple Inc.
WINDOWS è un marchio di Microsoft Corporation.
DePuy Orthopaedics, Inc.