**GENERAL REUSABLE  
SURGICAL  
INSTRUMENTS**

**P/N: 108410**

**Rev: E Revised 04/13**

**© 2005-2013 DePuy Mitek**

# ENGLISH

## GENERAL REUSABLE SURGICAL INSTRUMENTS

### NON-STERILE – REUSABLE

#### INDICATIONS

DePuy Mitek General Reusable Surgical Instruments are intended to be used for general surgical use or implant insertion during orthopedic procedures.

The Instrument Sterilization Trays are intended to hold DePuy Mitek Instruments during the sterilization process and for storage.

#### DESCRIPTION

The DePuy Mitek General Reusable Surgical Instruments are a collection of non-sterile, reusable surgically invasive devices for transient use during orthopedic procedures.

#### INSTRUCTIONS FOR USE

Refer to the implant specific package insert for use of an instrument with the implant.

#### CONTRAINDICATIONS

None known

#### WARNINGS

Appropriate personal protective equipment and attire should be worn when cleaning and sterilizing soiled surgical instruments.

Instruments may have sharp cutting edges that will puncture skin.

#### PRECAUTIONS

It is imperative that the surgeon and operating theater staff are fully conversant with the appropriate surgical technique for the surgical instruments.

Inspect for damage prior to use. Replace a damaged, worn or bent instrument. Do not attempt to straighten, sharpen or repair.

Do not use a DePuy Mitek reusable instrument for any purpose other than its intended use as that may result in damage to the instrument.

**Follow the instructions and warnings issued by the suppliers of any cleaning and disinfection agents and equipment used.**

**Do not exceed 137°C during sterilization.**

Highly alkaline conditions can damage products with aluminum parts.

Complex devices, such as those with tubes, hinges, retractable features, matted surfaces, and textured surface finishes, require special attention during cleaning. Manual pre-cleaning of such device features is required before automated cleaning processing.

Avoid exposing instruments to hypochlorite solutions, as these will promote corrosion.

Clean delicate instruments separately from other instruments.

Repeated processing has minimal effects on instrument life and function.

End of useful instrument life is generally determined by wear or damage from handling or surgical use. Inspect instruments between uses to verify proper functioning. (See "Inspection and Functional Testing" instructions below.)

---

## **CLEANING AND STERILIZATION INSTRUCTIONS**

These cleaning and sterilization instructions have been validated for preparing reusable DePuy Mitek instruments for reuse. It is the responsibility of the end user to ensure that the cleaning and sterilization is actually performed using appropriate equipment, materials, and personnel to achieve the desired result. This normally requires validation and routine monitoring of the process. Any deviation from these instructions should be evaluated for effectiveness and potential adverse consequences.

### **PREPARATION**

- Clean instruments as soon as possible after use. If cleaning must be delayed, wet the instruments in a compatible liquid solution to prevent drying and encrustation of surgical soil.
- Avoid prolonged exposure to saline to minimize the chance of corrosion.
- Remove excessive soil with a disposable wipe.

Prior to sterilization, instruments should be cleaned by either automated or manual means described below.

### **AUTOMATED CLEANING**

Instrument types with such complex design features as cannulations, lumens, (i.e. tubes), holes, hinged joints, box locks, spring-loaded retractable features, or flexible shafts; must be manually pre-cleaned prior to automated cleaning to improve the removal of adherent soil.

#### Manual Pre-Cleaning Instructions

- Flush internal areas with a warm detergent solution. Pay close attention to lumens, cannulations, holes, threads, crevices, seams, and any hard to reach areas. While flushing, actuate any moveable mechanisms, (hinged joints, box locks, or spring-loaded features), to free trapped blood and debris. If the components of the instrument can be retracted, retract or open the part while flushing the area.
- For instruments with flexible shafts, flex the instrument while flushing.
- For instruments with lumens, brush the internal surfaces of the lumen.
- Rinse the instrument thoroughly with warm tap water.
- Ultrasonically clean the instrument for 10 minutes in neutral pH detergent, prepared in accordance with the manufacturer's instructions.

#### Loading Instructions

- Load instruments into washer disinfectant so that hinges are open and cannulations and holes can drain.

- Place heavier instruments on the bottom of containers. Do not place heavy instruments on top of delicate instruments.
- For instruments with concave surfaces, such as curettes, place instrument with the concave surface facing downward to facilitate draining.

#### Automated Cleaning Instructions

- Clean in a validated washer disinfecter using the “INSTRUMENTS” cycle and a pH neutral cleaning agent intended for use in automated cleaning. Effective automated cleaning can be achieved using the following parameters: two cold water pre-washes (minimum hold of two minutes each); a hot water enzyme wash (minimum hold of 4 minutes); followed by a hot water pH neutral detergent wash (minimum hold 3 minutes at 60°C); a hot water rinse (minimum time 20 seconds); a thermal rinse (minimum hold of 1 minute at 82.2°C) and dry (minimum hold 5 minutes at 95°C).
- After automated cleaning, proceed to inspection, instructions below.

### **MANUAL CLEANING**

#### Cleaning Instructions

- Prepare an enzymatic cleaning solution in accordance to the manufacturer’s instructions.
- Soak soiled instruments for a minimum of 10 minutes. Do not exceed 30°C.
- When cleaning, fully immerse the instrument in the cleaning solution to avoid aerosol generation. Use a soft bristle brush to remove all traces of blood and debris; pay close attention to any hard to reach areas, textured surfaces, or crevices. When cleaning Instruments with cannulations or lumens (i.e. tubes), or holes, use a tight-fitting, soft, non-metallic cleaning brush or pipe cleaner to scrub the cannula, lumen, or hole. Push in and out, using a twisting motion to remove debris. Use a syringe filled with enzymatic cleaning solution to flush hard to reach internal areas. When cleaning articulating instruments, (those with moveable parts), brush with a soft non-metallic bristle brush to remove all traces of blood and debris. Pay close attention to threads, crevices, seams, and any hard to reach areas. Actuate any moveable mechanisms, such as hinged joints, box locks, or spring-loaded features, to free trapped blood and debris. If the components of the instrument can be retracted, retract or open the part while cleaning the area. For instruments with flexible shafts, flex the instrument under the cleaning solution. Rinse the instrument thoroughly with warm water.
- Ultrasonically clean the instrument for 10 minutes in neutral pH detergent, prepared in accordance with the manufacturer’s instructions. Rinse the instrument thoroughly with warm water.

#### Rinsing Instructions

Rinse the instrument thoroughly with filtered water at 38–49°C, agitate for a minimum of 1 minute; repeat rinse at least two additional times. When rinsing, pay particular attention to flush the cannulations, lumens, or holes with warm tap water. Also pay particular attention to internal areas and moveable parts. Actuate moveable parts while rinsing. If the components of the instrument can be retracted, retract or open the part while rinsing the area.

For instruments with flexible shafts, flex the instrument under the rinse solution.

### Drying Instructions

Dry the instrument immediately after final rinse. Use filtered compressed air to dry internal areas. After drying proceed to Inspection instructions.

---

## **INSPECTION AFTER CLEANING**

- Inspect all instruments before sterilization or storage to ensure the complete removal of soil from surfaces, tubes and holes, moveable parts.
- If areas are difficult to inspect visually, check for blood by immersing or flushing the instrument in a 3% hydrogen peroxide solution. If bubbling is observed, blood is present. Rinse instruments thoroughly after using hydrogen peroxide solution.
- If soil is still present, reclean the instrument.

## **MAINTENANCE**

Between uses, lubricate moving parts with a water-soluble lubricant in accordance with the manufacturer's instructions.

## **INSPECTION AND FUNCTIONAL TESTING**

- Visually inspect the instrument and check for damage and wear.
- Cutting edges should be free of nicks and have a continuous edge.
- Jaws and teeth should align properly.
- Moveable parts should have smooth movement without excessive play.
- Locking mechanisms should fasten securely and close easily.
- Long, thin instruments should be free of bending and distortion.

## **PACKAGING**

- For instruments that are provided in sets, instrument trays may be used for containment during sterilization.
- Package trays/instruments with a barrier wrap material in accordance with local procedures, using standardized wrapping techniques such as those described in ANSI/AAMI ST46-1993.

## **STERILIZATION**

### Trays

- Place DePuy Mitek instrument in the appropriate location within the DePuy Mitek sterilization tray. If a specific location is not identified for an instrument, it may be placed in the general purpose area (pin-mat) ensuring that the sterilant has adequate access to all surfaces including difficult to reach areas, lumens, etc.
- Use a validated, properly maintained and calibrated steam sterilizer.
- Effective steam sterilization can be achieved using the following cycles:

| <b>Cycle Type</b>               | <b>Temperature</b> | <b>Minimum Exposure Time</b> |
|---------------------------------|--------------------|------------------------------|
| Prevacuum                       | 132°C (270°F)      | 4 min                        |
| Prevacuum<br>(outside the U.S.) | 134–137°C          | 3 min                        |

**Drying Time:** 30 minutes

## **STORAGE**

Store sterile packaged instruments in a manner that provides protection from dust, moisture, insects, vermin, and extremes of temperature and humidity.

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- SG** Johnson & Johnson Medical Singapore  
No. 2, International Business Park #07-01  
Tower One, The Strategy  
Singapore 609930
- Vietnam Johnson & Johnson Medical Vietnam  
Unit 8, Level 5, International Center  
17 Ngo Quyen street, Hoan Kiem district  
Hanoi, VIETNAM
- India Johnson & Johnson Medical India  
4th Floor, Building no. 9B  
Cyber City, DLF Phase III  
Gurgaon-122002  
Haryana  
India
- JP** Johnson & Johnson K.K. Medical Company  
5-2, Nishi-kanda 3-chome, Chiyoda-ku, Tokyo 101-0065  
Japan
- SI** Johnson & Johnson d.o.o.  
Šmartinska cesta 53  
1000 Ljubljana  
Slovenia
- HR** Johnson & Johnson S.E., d.o.o.  
Oreškovićeve 6h  
10010 Zagreb  
Croatia
- BIH** Johnson & Johnson d.o.o., Ljubljana, Republika  
Slovenija, Predstavništvo u Bosni i Hercegovini  
Fra Andjela Zavidovića br. 1  
71000 Sarajevo  
Bosnia and Herzegovina
- SR** Johnson & Johnson S.E., Inc.  
Airport City  
Omladinskih brigada 88b  
11000 Beograd  
Serbia
- RO** Johnson & Johnson Romania SRL.  
str. Tipografilor, nr. 11-15  
clădirea S-park A2, et. 5, sector 1  
013714 București  
Romania
- BG** Johnson & Johnson d.o.o.  
Business Park Sofia, Office Building 4, Floor 4  
Al Malinov blvd  
Sofia  
1766  
Bulgaria
- AL** JOHNSON & JOHNSON.  
Ruga Elbasanit, pall. EDIL, AL-IT  
Nr 2, app 6D  
Tirana  
Albania

**QTY**

Number of Units  
Número de unidades  
Numero di unità  
Anzahl  
Nombre d'unités  
Aantal eenheden  
Número de unidades  
Antal enheder  
Antall enheter  
Yksikköjen lukumäärä  
Αριθμός μονάδων  
Antal enheter  
Počet kusů  
Počet kusov  
Ilość  
Число единиц  
Egységek száma  
Birim Adedi  
件数  
装置數量  
수량

**NON STERILE**

NON STERILE  
NO ESTÉRIL  
NON STERILE  
UNSTERIL  
NON STÉRILE  
NIET-STERIEL  
NÃO ESTERILIZADO  
IKKE STERIL  
IKKE-STERIL  
STERILOIMATON  
ΜΗ ΣΤΕΙΡΟ  
OSTERIL  
NESTERILNÍ  
NESTERILNÉ  
NIEJAŁOWY  
НЕСТЕРИЛЬНО  
NEM STERIL  
NON-STERIL  
非灭菌  
未滅菌  
비멸균

**DePuy Mitek, Inc.**  
325 Paramount Drive  
Raynham, MA 02767, USA  
1-800-356-4835

**Medos International SARL**  
Chemin-Blanc 38,  
2400 Le Locle, Switzerland

- \* For recognized manufacturer, refer to product label.
- \* Para el fabricante reconocido, refiérase a la etiqueta del producto.
- \* Per il produttore riconosciuto, fare riferimento all'etichetta del prodotto.
- \* Dem Produktetikett ist der anerkannte Hersteller zu entnehmen.
- \* Pour le fabricant reconnu, consulter l'étiquette du produit.
- \* Raadpleeg het etiket op het product voor informatie over de erkende fabrikant.
- \* Para conhecer o fabricante reconhecido, consulte a etiqueta do produto.
- \* Se produktets etiket for at få oplysninger om anerkendt producent.
- \* Du finner informasjon om godkjent produsent på produktetiketten.
- \* Katso valmistajan tiedot tuotteen pakkausmerkinnöistä.
- \* Για πληροφορίες σχετικά με τον αναγνωρισμένο κατασκευαστή, ανατρέξτε στην ετικέτα του προϊόντος.
- \* Information om produktens tillverkare finns på förpackningen.
- \* Informace o uznávaném výrobcí naleznete na štítku výrobku.
- \* Informácie o uznávanom výrobcovi nájdete na štítku výrobku.
- \* Informacje na temat autoryzowanego producenta, patrz etykieta produktu.
- \* Информацию о признанных изготовителях см. на этикетке изделия.
- \* A gyártóval kapcsolatos információkat lásd a termék címkéjén.
- \* Tanınan üretici için, ürün etiketine bakın.
- \* 关于认可制造商的信息，请参考产品标签。
- \* 關於公認的製造商，請參閱產品標籤。
- \* 공인 제조사는 제품 라벨에서.

**DePuy International Ltd.**  
St Anthony's Road,  
Leeds LS11 8DT England  
Tel: +44 113 270 0461  
Fax: +44 113 272 4101