

# USS<sup>®</sup> FRACTURE SYSTEM

## Surgical Technique



 Image intensifier control

This description alone does not provide sufficient background for direct use of DePuy Synthes products. Instruction by a surgeon experienced in handling these products is highly recommended.

**Processing, Reprocessing, Care and Maintenance**

For general guidelines, function control and dismantling of multi-part instruments, as well as processing guidelines for implants, please contact your local sales representative or refer to:

<http://emea.depuysynthes.com/hcp/reprocessing-care-maintenance>

For general information about reprocessing, care and maintenance of Synthes reusable devices, instrument trays and cases, as well as processing of Synthes non-sterile implants, please consult the Important Information leaflet (SE\_023827) or refer to:



<http://emea.depuysynthes.com/hcp/reprocessing-care-maintenance>

# Table of Contents

---

<b>Principle of Fracture Clamps</b>	5
<hr/>	
<b>Surgical Technique</b>	6
<hr/>	
<b>Fractures with Intact Posterior Wall</b>	8
<hr/>	
<b>Fractures with Fractured Posterior Wall and Rupture of Posterior Longitudinal Ligament</b>	10
<hr/>	
<b>Assembling the Cross-Link System</b>	15
<hr/>	
<b>Reduction of Spondylolisthesis</b>	20
<hr/>	
<b>Implant Removal</b>	22
<hr/>	
<b>Bibliography</b>	23
<hr/>	
<b>Indications/Contraindications</b>	24

For Product Catalogue contact your local DePuy Synthes representative.

-  Image Intensifier Control
-  Warnings/Precautions

# AO Spine Principles

The four principles to be considered as the foundation for proper spine patient management underpin the design and delivery of the Curriculum: Stability, Alignment, Biology, Function.<sup>1,2</sup>

## AO Principles<sup>1,2</sup>


1.



### Stability

Stabilization to achieve a specific therapeutic outcome.


2.

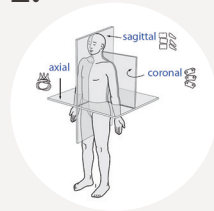

### Alignment

Balancing the spine in three dimensions.


3.

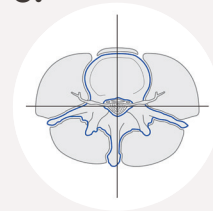

### Biology

Etiology, pathogenesis, neural protection, and tissue healing.

4.

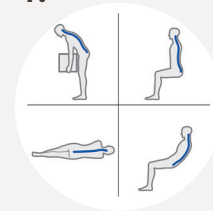


### Function

Preservations and restoration of function to prevent disability.

# Principle of Fracture Clamps

Fracture Clamps for Rods (Low Profile) can be angled  $\pm 15$ degs.

Fracture Clamps for Rods (Cranial End) can be angled  $+9$ degs/ $-15$ degs.

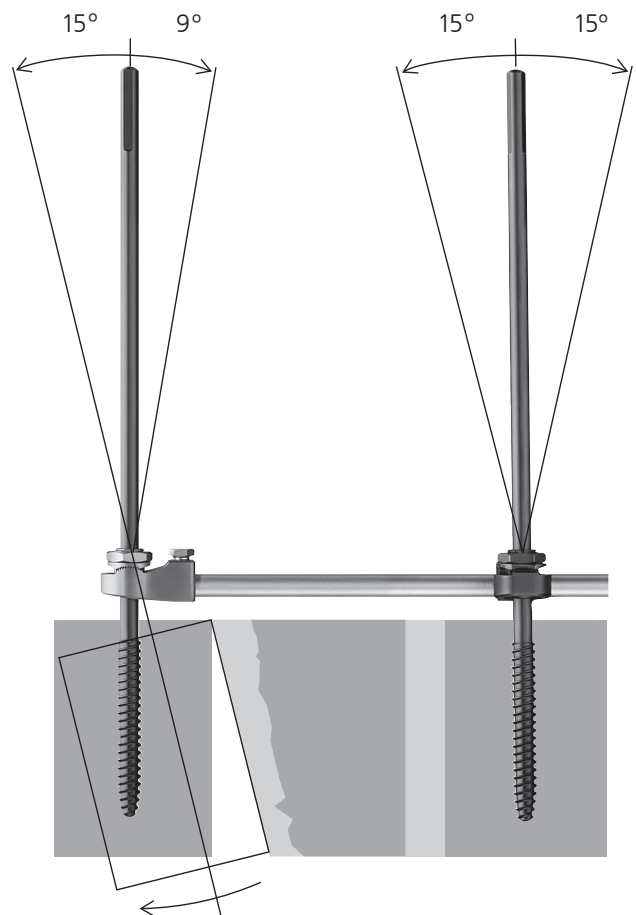


# Surgical Technique

## 1. Locate and open the pedicles

Locate and open the pedicles using the Pedicle Awl  $\text{\O}4.0$  mm (388.550) to a depth of 10 mm and the Pedicle Probe  $\text{\O}3.8$  mm (388.540). The pedicle probe has markings at 30, 40 and 50mm for checking the depth of the pedicle / vertebral body penetration.

### ▲ Warning:

- Do not penetrate the anterior wall of the vertebral body

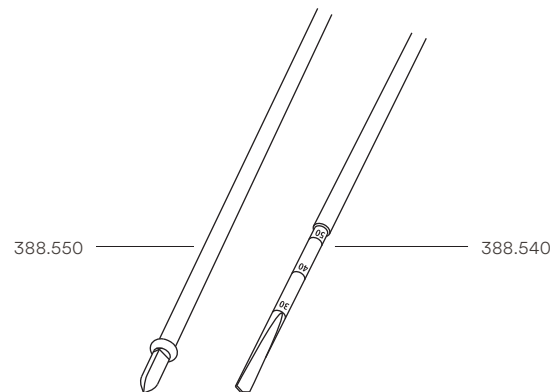

## 2. Verify Trajectory

- 1 Using the hook of a depth gauge or a ball tipped feeler, probe the drilled channel to check that the channel is fully intact and that the spinal canal has not been opened.


## 3. Insert Schanz Screws

Insert the Schanz screws using the T-Handle (395.380) or Universal Chuck (393.100).

- 1 The Schanz screws should be inserted under lateral image intensifier control.

### ▲ Warning:

The tips of the Schanz screws must not penetrate the anterior cortex.”

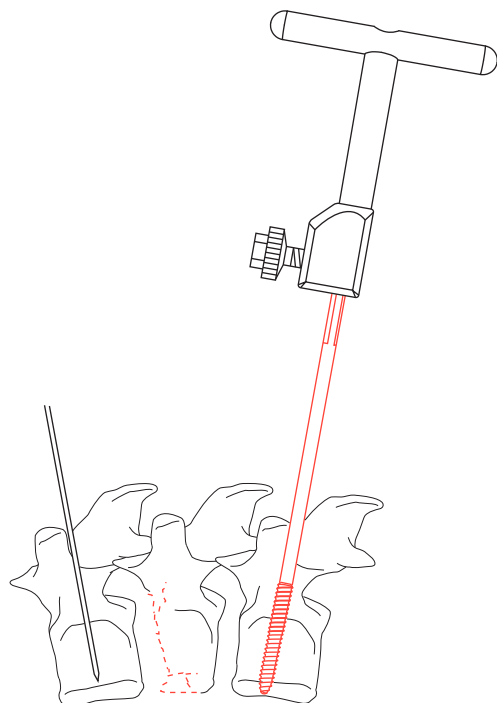

## 4. Assemble USS fracture clamps and rod

Select the appropriate rod length. Take any necessary distraction into account when determining the length of the rod.




Place the clamps on the Schanz screws, push the rod through both clamps and push the entire construction toward the spine.

A slight resection of the spinal process will cause the assembly to lie close to the lamina.

The rod comes to rest medially.

### 4a. Assembly with USS fracture clamp for the cranial end (optional)


The Fracture Clamp for Rods 498.833 can also be used for the cranial end. Since this clamp is firmly fixed to the rod, only one clamp can be used on each side. This clamp prevents the rod from jutting out at the cranial end, thereby protecting adjacent mobile segments. The cranial fracture clamp is fixed to the vertical rod using the Socket Wrench 6.0 mm (388.140).

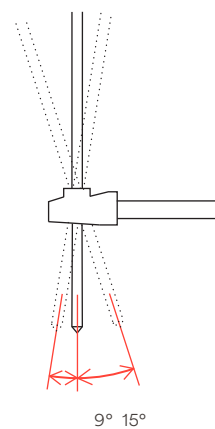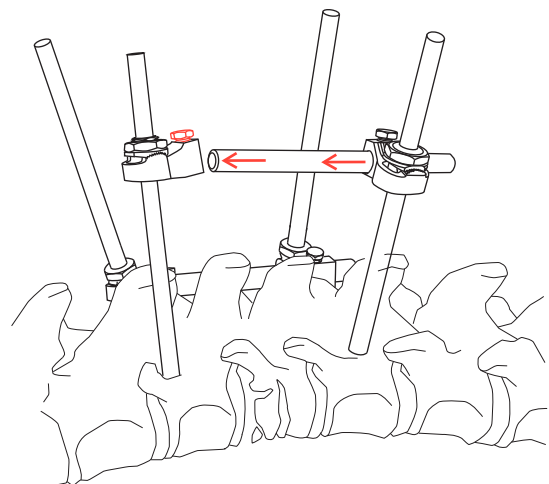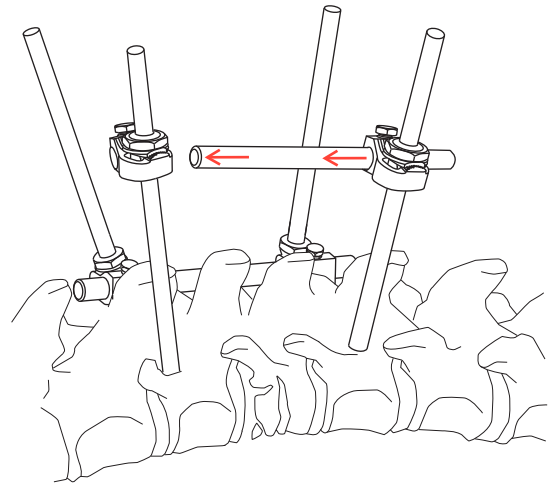

# Fractures with **Intact** Posterior Wall


## Principle of kyphosis correction with **intact** posterior wall

Pressing the Schanz screws together dorsally lordoses the adjacent vertebrae around the pivot point (red circle) of their facing posterior edges. The clamps on the rod move toward the centre. The fracture clamps must be able to slide freely along the rod, otherwise kyphosis correction will not be achieved.

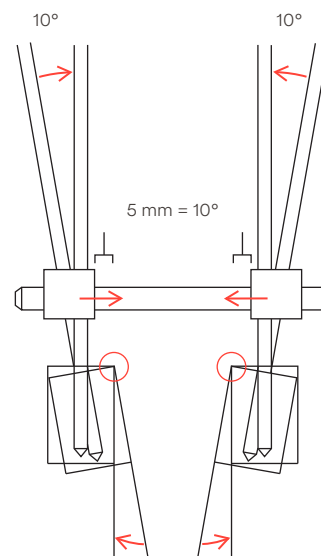

Principle of kyphosis correction with the cranial clamp with an **intact** posterior wall (optional)

The use of the cranial fracture clamp allows correction of 10° in each case by moving the caudal clamp 10 mm (guide distance).

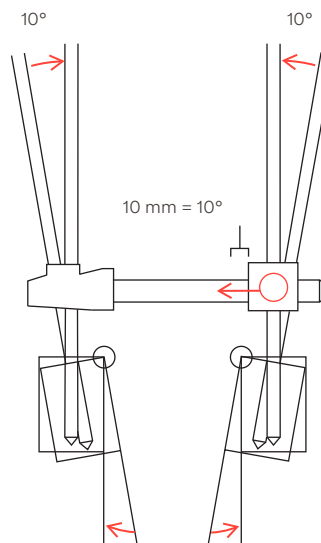




### 5a. Locate Socket Wrench on both caudal Schanz screws and lordose the spine

Tilt both posteriorly projecting caudal screws cranially to lordose the spine. Secure the clamps/Schanz screws in the desired position using the Socket Wrench Ø 11 mm (394.701).


▲ **Precaution:**

- It is absolutely essential that the blue-marked Socket Wrench Ø 11 mm (394.701) is used for the low-profile fracture clamps.

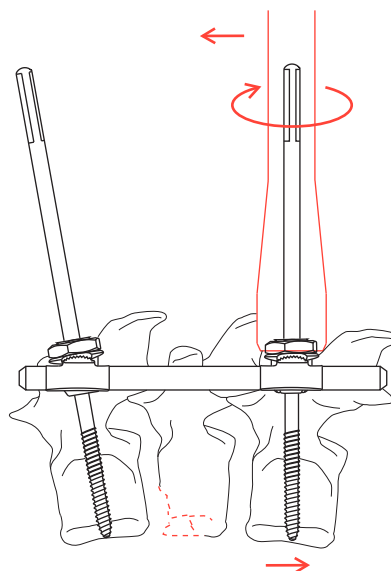

### 6a. Locate Socket Wrench on both cranial Schanz screws and lordose the spine

Repeat the above procedure for the cranial Schanz screws: Tilt in the caudal direction to complete the lordosing operation and secure in the desired position.

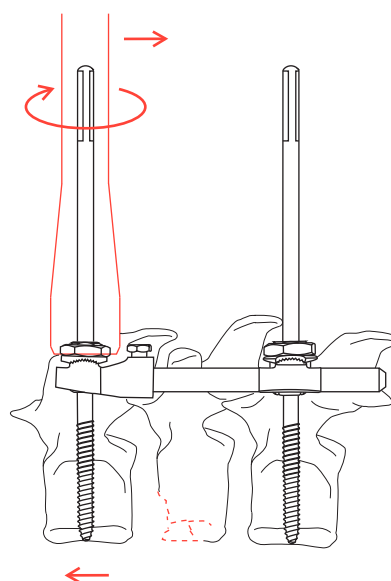


# Fractures with **Fractured** Posterior Wall and Rupture of Posterior Longitudinal Ligament

## ▲ Warning:


- Since any reduction produced by pressing the Schanz screw ends together may produce undesirable compression on the damaged posterior wall of the vertebral body, there may be an associated risk of fragment dislocation into the spinal canal. Therefore, every clamp on the rod must be secured by a Fixation Ring for Rods Ø6.0 mm (498.911). This shifts the centre of rotation (red circle) to the level of the rod. Similarly, pushing the screw ends away from each other may cause undesirable distraction in the situation where the Posterior Longitudinal Ligament is ruptured or damaged.

5 mm gaps between the fixation rings and the clamps allow kyphosis correction of 10 degree in each case (guide value).

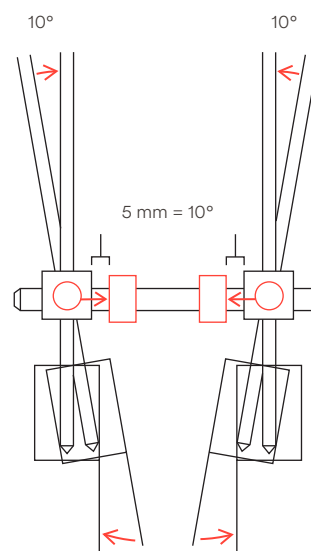


Principle of kyphosis correction with the cranial clamp with **fractured** posterior wall (optional)

The use of the cranial fracture clamp allows correction of 10° in each case by moving the caudal clamp 10 mm (guide distance). A fixation ring must be used as a stop.

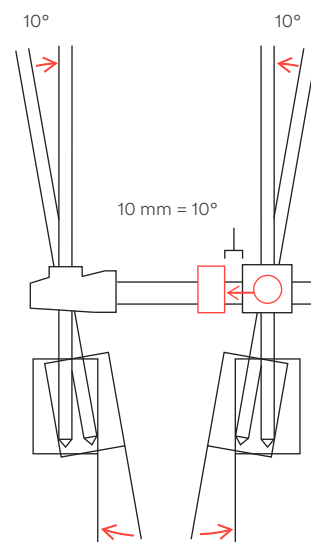


## 5b. Mount fixation rings according to the degree of lordosing

Pick up fixation rings using the Screwdriver, hexagonal (314.070) and the Holding Sleeve with Catches (388.363), locating the holding sleeve on the head of the set screw. Secure the fixation rings between the fracture clamps according to the desired degree of lordosing.

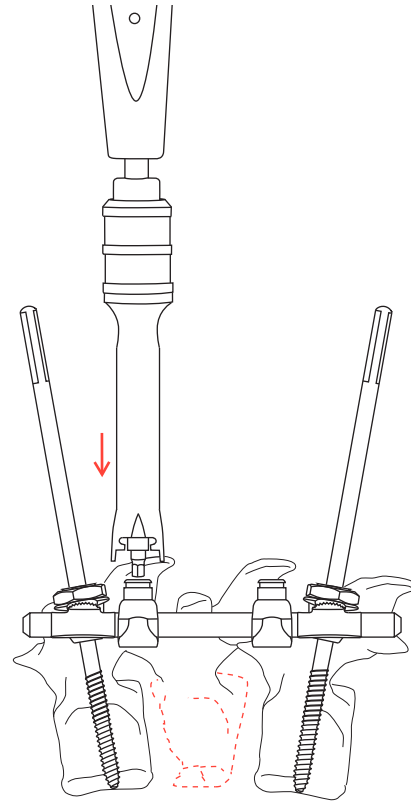

## 6b. Locate Socket Wrench and lordose the spine

Locate the Socket Wrench  $\varnothing$  11 mm and create the corresponding lordosis by tilting the Schanz screws as described under 5a and 6a.

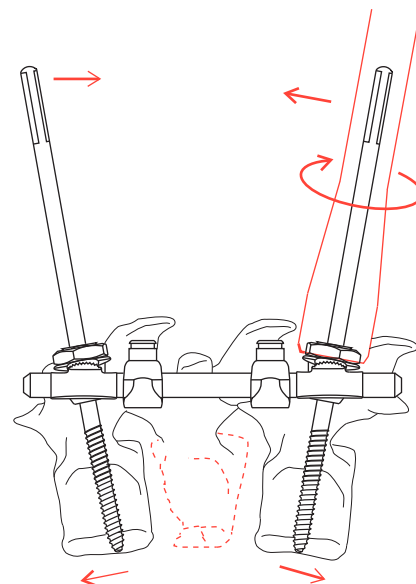


---

## 7. Fix the clamps on the rods

Using the Socket Wrench 6.0 mm (388.140), tighten the set screws to fix the fracture clamps on the vertical rods.



If the cranial clamp is used, the caudal fracture clamps are fixed to the vertical rods by tightening the set screws with the Socket Wrench 6.0 mm.

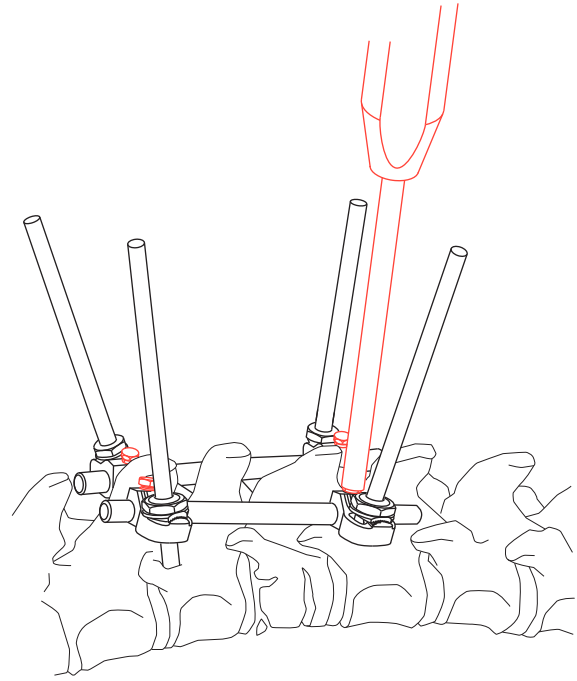

8. If required: Distraction with the Spreader Forceps under image intensifier control


Using the Socket Wrench 6.0 mm, loosen the set screws on the fracture clamps for the relevant vertebra and perform careful distraction if this is necessary to complete the anatomical reduction and restore the original level of the fractured vertebral body.

- If the cranial clamp is used, distraction can only be performed with the caudal clamp.

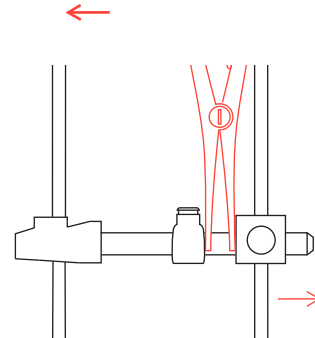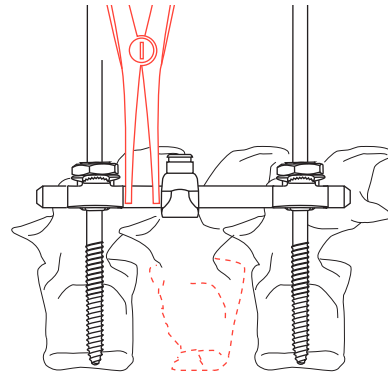

9. Remove fixation rings

When reduction is complete, tighten the set screws and remove the fixation rings.

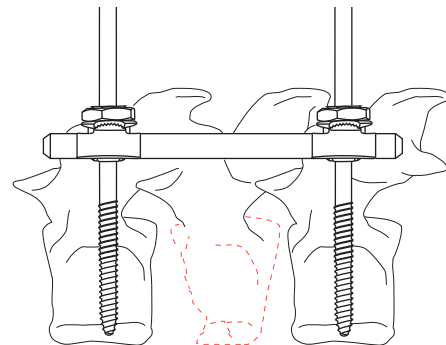

## 10. Trim Schanz screws using the Bolt Cutter

When reduction is complete and the assembly has been secured, trim the Schanz screws to the required length using the Bolt Cutter (Handle 391.780/790 and Bolt Cutting Head  $\varnothing$  5.0 mm 391.771).



### Using the Bolt Cutter

Assemble the Bolt Cutter and place in the neutral position (you should be able to see through the 5 mm hole). Position the handles, one on top of the other, on the bolt cutting head like the hands of a clock. Slide the bolt cutting head over the Schanz screw.

Pull the handles apart to an angle of approximately  $45^\circ$  until the Schanz screw audibly breaks.

Return the handles to the original position and move the bolt cutting head to the next Schanz screw. The previously cut screw shaft will fall out during this operation.

- If the cut screw shaft does not fall out of its own accord, it can be pushed out using the Cancellous Bone Impactor, straight (394.570) or the shaft of another Schanz screw.
- If this is not possible, the bolt cutting head will have to be dismantled and the screw shaft pushed out of the inner bolt.

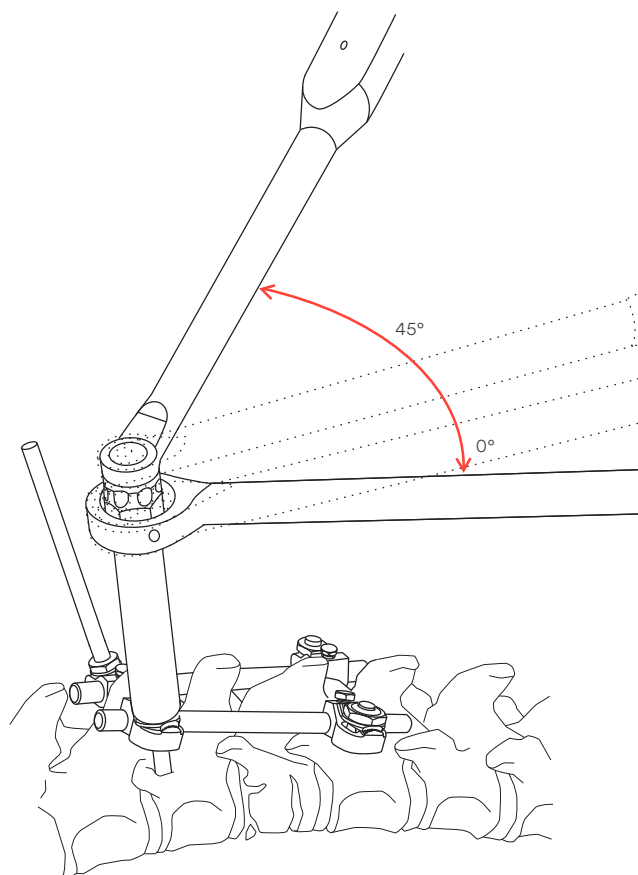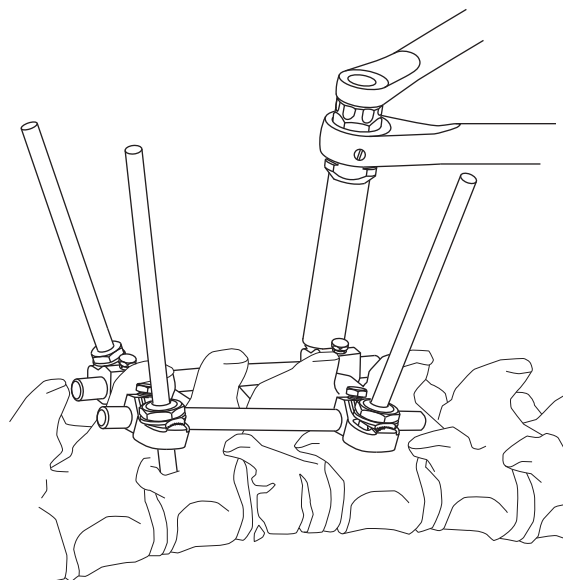


# Assembling the Cross-Link System

Cross-links are transverse stabilizers that link the two vertical rods, thereby increasing the stiffness of the construct. They are recommended for unstable fractures and multisegmental constructs.


## 1. Pick up first Cross-Link Clamp

Assemble the Small Screwdriver, hexagonal (314.070) and the Holding Sleeve with Catches (388.363). To pick up the preassembled Cross-Link Clamp for Rods  $\varnothing$  6.0 mm (498.813), insert the hexagonal screwdriver into the set screw on the clamp, push down the holding sleeve and clip the catches onto the sleeve of the preassembled clamp.

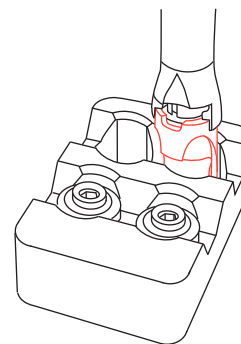

## 2. Mount first Cross-Link Clamp

Pull the holding sleeve back slightly, place the clamp onto the rod and release the holding sleeve.

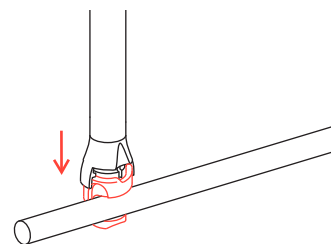


### 3. Insert cross-link rod

The special design of the cross-link sleeve with its two recesses on the top allows the cross-link rod to be angled up to  $\pm 20^\circ$  to suit the anatomical situation.

Determine the appropriate length of the B 3.5 mm cross-link rod. If necessary, cut the rod to length using the USS Rod Cutting and Bending Device (388.750).


Hold the clamp with the small hexagonal screwdriver and introduce the  $\varnothing$  3.5 mm cross-link rod through the hole in the Cross-Link Clamp (1). If necessary, use the Holding Forceps for USS Rods B 3.5/4.5 mm (388.450) to introduce the cross-link rod. Tighten the set screw of the Cross-Link Clamp with the small hexagonal screwdriver (2).

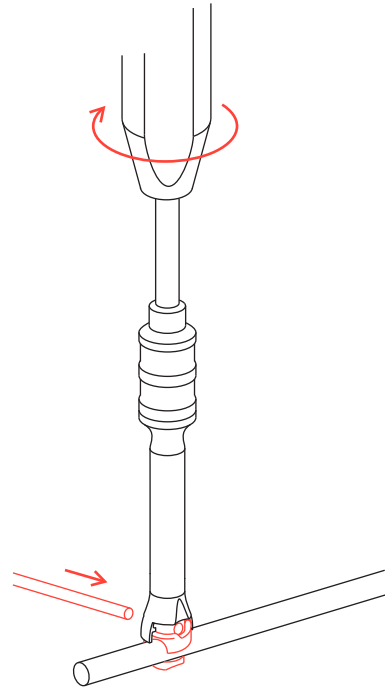




#### 4. Mount second Cross-Link Clamp

Repeat the procedure described in step 1 (page 14) for the second clamp on the opposite rod. Introduce the  $\varnothing$  3.5 mm cross-link rod through the second clamp so that it protrudes by 5 mm beyond the clamp. Tighten the set screw with the small hexagonal screwdriver.

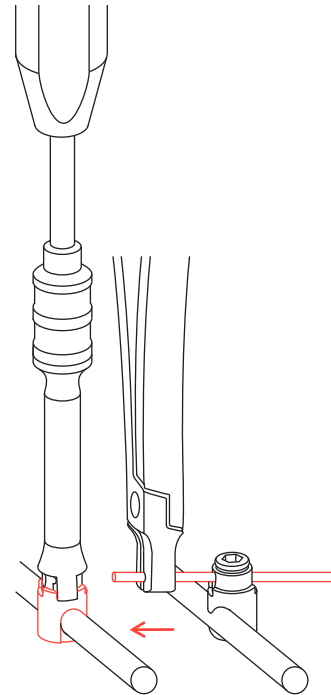

#### 5. Distract cross-link assembly (optional)



Loosen one of the set screws. Place the Holding Forceps for USS Rods  $\varnothing$  3.5/4.5 mm (388.450) next to the clamp and use the Spreader Forceps for Pedicle Screws (388.410) to exert distraction. Retighten the set screw with the small hexagonal screwdriver.

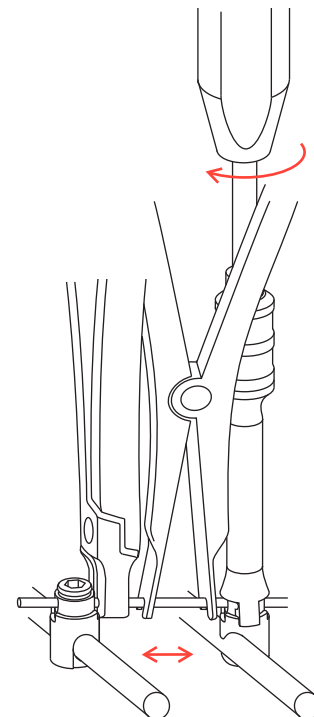

---

## 6. Check all set screws on the system

When the system is fully assembled, check that all screws are securely tightened.

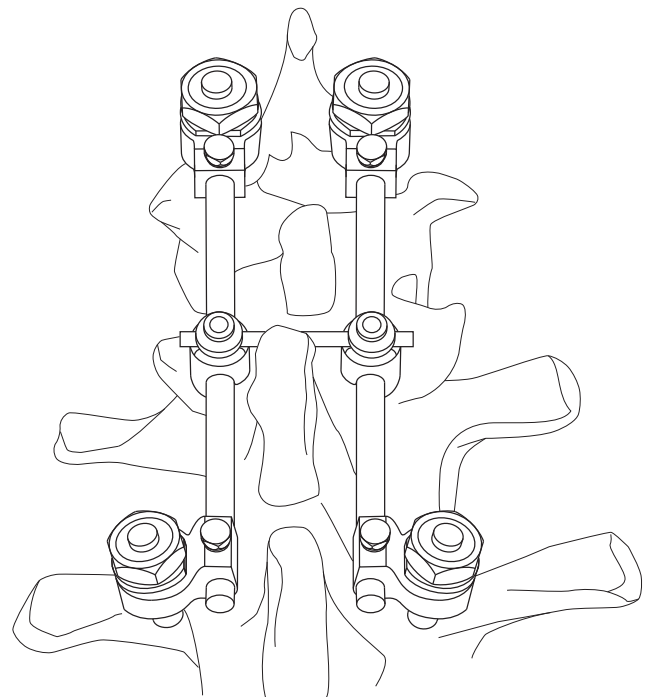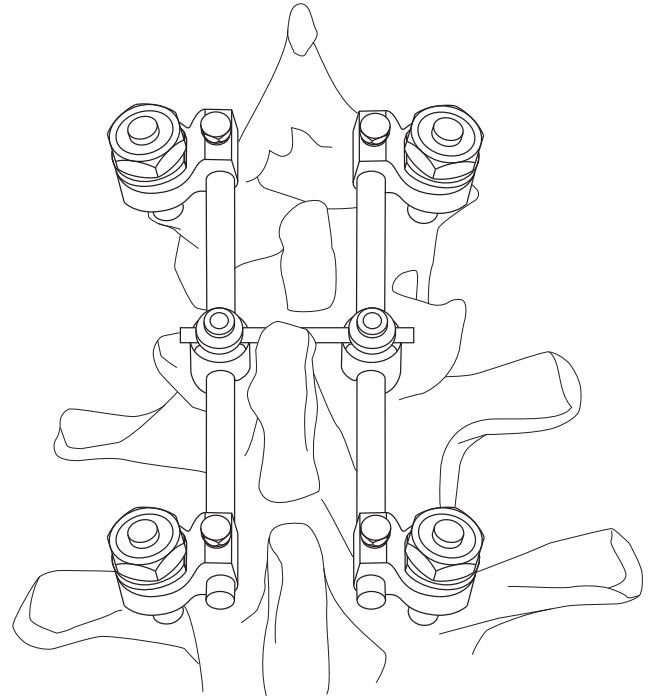

---

**Fracture of the posterior elements of the spine or disruption with distraction**

In these indications, the USS Fracture System is used as a tension-band wiring system. Reduce the fracture as described under 5a/6a, then perform appropriate compression using the fixation rings and the Compression Forceps (388.422).

**Complete disruption of the anterior and posterior elements of the spine with rotation**


In these indications, the USS Fracture System is used as a neutralization system. If necessary, perform compression using the fixation rings and the Compression Forceps (388.422).

The additional use of one or two cross-link stabilizers to produce a frame construction is recommended.

# Reduction of Spondylolisthesis


## 1. Insert Transpedicular Schanz Screws

Insert the Transpedicular Schanz Screws with dual core and double thread (496.776–778 or 496.796–798) into the displaced vertebra (cranial) as described in steps 1–3 on page 5. Normal Schanz screws are inserted into the caudal vertebra. Assemble USS fracture clamps and rods as described in step 4 (page 6). Secure caudal fracture clamps to the rod.

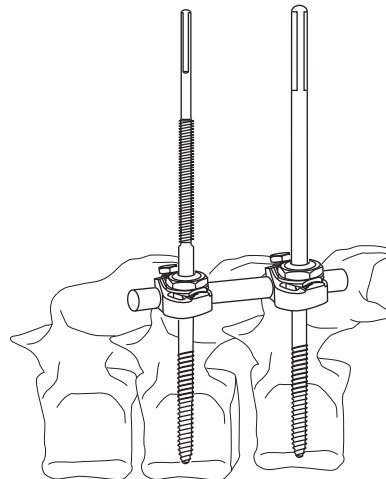

## 2. Perform reduction


Slide the USS Reduction Sleeve (388.931) and USS Nut, knurled (388.932) over the Schanz screws with double thread. Turn the nuts on both sides until the desired reduction is achieved.

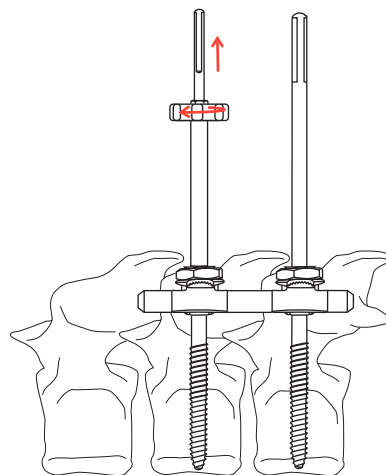

---

### 3. Tighten fracture clamps

Remove the USS Nuts, knurled and tighten the fracture clamps using the Socket Wrench Ø11 mm (394.701).

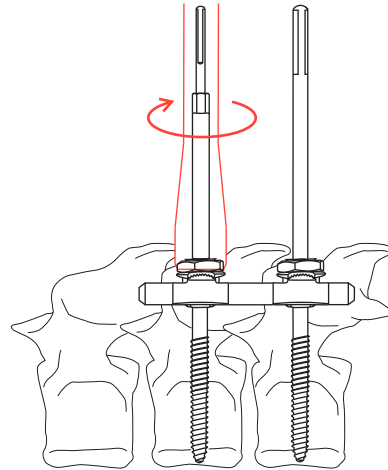

### 4. Fix fracture clamps on the rods and trim Schanz screws

Remove the USS reduction sleeves. Fix the USS fracture clamps using the Socket Wrench 6.0 mm as described in step 7 (page 11). Trim the Schanz screws with the Bolt Cutter as described in step 10 (page 13).

# Implant Removal

---

After fracture consolidation (9–12 months), removal of the implant is recommended in order to minimize any impairment of the paravertebral muscles. The implant should not be removed if tumours are present.

The clamps are loosened using the Socket Wrench Ø 11 mm (394.701), while the set screws are loosened with the Socket Wrench 6.0 mm (388.140). The rod and clamps can then be removed from the Schanz screws.

Next, grasp the ends of the Schanz screws with the screw forceps or the T-handle and pull the screws out.

# Bibliography

---

1. Aebi M, Thalgott JS, Webb JK (1998) AO/ASIF Principles in Spine Surgery. Springer-Verlag, Germany.
2. Aebi M, Arlet V, Webb JK (2007): AOSPINE Manual (2 vols), Stuttgart, New York: Thieme.

# Indications/Contraindications

---

Please refer to the corresponding Instructions for Use for specific information on Intended use, Indications, Contraindications, Warnings and Precautions, Potential Adverse Events, Undesirable Side Effects and Residual Risks.

Instructions for Use are available at [www.e-ifu.com](http://www.e-ifu.com) and/or [www.depuysynthes.com/ifu](http://www.depuysynthes.com/ifu)



Not all products are currently available in all markets.  
This publication is not intended for distribution in the USA.  
Surgical techniques are available as PDF files at [www.depuysynthes.com/ifu](http://www.depuysynthes.com/ifu)



**Synthes GmbH**  
Eimattstrasse 3  
4436 Oberdorf  
Switzerland  
Tel: +41 61 965 61 11

[www.jnjmedicaldevices.com](http://www.jnjmedicaldevices.com)