

Care and Maintenance Power Tools. Pictogram Key.



Disassembling



Spray for maximum 1 second with Synthes Maintenance Spray. If possible choose the Maintenance Unit alternative for lubrication of Electric Pen Drive and Air Pen Drive



Spray with disinfectant and wipe off*



Immersion in disinfectant possible*



Lubricate the designated points. See product-specific "Instructions for Use" for more detailed information



Wipe off with an absorbent cloth soaked in disinfectant*

The number of drops is shown on the Care and Maintenance posters



Manual cleaning**



Immerse in instrument milk (non-silicone based medical lubricant)



Immerse in cleaning solution



Steam sterilization**



Automated cleaning with manual pre-cleaning**



Steam sterilization only. Hot air, ethylene oxide, plasma and formaldehyde sterilization are not allowed



Immersing not allowed



Do not sterilize!



Immersing and automated cleaning not allowed



H₂O₂ plasma STERRAD sterilization (max. 60°C / 140°F)***



Flush the inner lumen with a syringe



H₂O₂ V-PRO sterilization (max. 50°C / 122°F)***



Check the function of parts marked in red



Recalibrate once a year



Lubricating with the Maintenance Unit should always be the first option of lubrication for Electric Pen Drive and Air Pen Drive



Do not bend a cable or hose



When a symbol is marked with a cross, the action is forbidden. Not following this indication may result in damage to the product and patient or user harm.



Caution: Use the mentioned article for this step

article #

* Follow disinfectant manufacturer's instruction

** Detailed instructions for manual cleaning, automated cleaning and sterilization can be found in the product-specific "Instructions for Use"

***Please follow the instruction provided in STERRAD® / V-PRO® sterilization guidelines (DSEM/PWT/0615/0068, DSEM/PWT/0915/0081)

STERRAD® is a registered mark of Advanced Sterilization Product.

V-PRO® is a registered mark of STERIS.

This publication is not intended for distribution in the USA.

For full user instructions and precautions please consult the product IFU. Please contact your DePuy Synthes sales representative for more information.

All surgical techniques are available as PDF files at www.depuysynthes.com/ifu